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

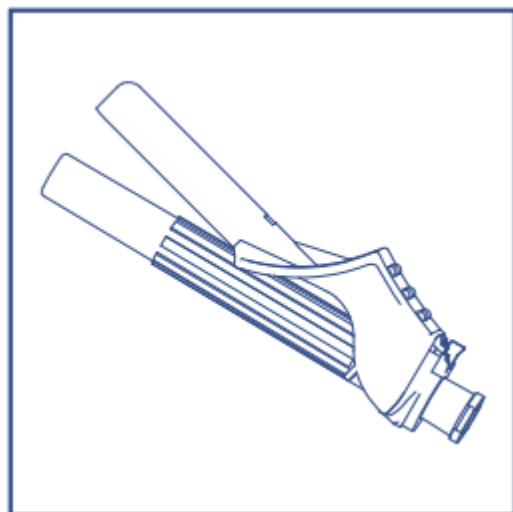
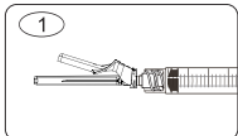
### DIRECTIONS FOR USE:

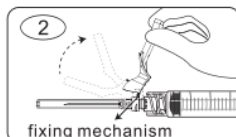
1. Carefully remove the safety needle from the package.

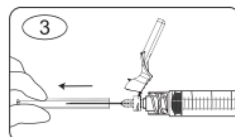
2. Securely attach the safety needle to a syringe with a push and clockwise twist motion. Move the safety sheath away from the needle and toward the syringe barrel. The safety sheath will remain in the position you set.

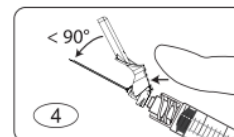
3. Remove the needle cover carefully to avoid damaging the needle. Perform puncture per standard clinical practice.

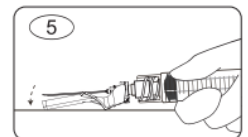
4. After puncture, Position sheath in preparation for device activation: Using a one-handed technique, push the tab forward with your finger or thumb so that the sheath is less than 90 degrees from the needle.

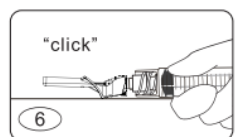5. Activate safety mechanism: position the safety protective cap against a flat surface at a lowest possible angle.

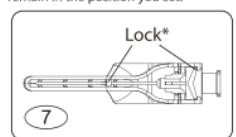6. Press down with a firm, quick motion until a distinct audible click is heard.

7. Visually confirm that the needle is fully engaged under the lock. Dispose of used needles and materials following your federal and local regulations for sharps disposal.

**Note:** the location of the lock will vary depending upon needle size. Audible click may not be heard on small needle sizes; visual confirmation is required per needle.

**100**  
PCS

Importé & Distribué par: Metze SAS  
10 rue de Penthièvre, 75008 Paris  
[commandes@metzecare.com](mailto:commandes@metzecare.com)