

hümiFlö®

DISPOSABLE PREFILLED BUBBLE HUMIDIFIER

CDH500

- Approximate volume 500ml
- Single-use only
- Non-pyrogenic distilled water
- **DO NOT USE** if seal is tampered
- Discard unused solution
- For flowmeter connection use only



CAUTION:

Only use between 0-6LPM

Any incident should be reported to the manufacturer or competent authority.

LOT



STERILE

SBS

MD



PP

CE

UDI



9 454 100 244 300 >

STAR MEDIK SDN BHD
200591005964

43, Jalan Tera Sentral 2, Nilai Utama Enterprise Park
71800 Nilai, Negeri Sembilan, Malaysia

EC REP

MT Promed Consulting GmbH Altenhofstrasse 60
60336 St. Ingbert, Germany

hümiFlö® is a registered trademark of Star Medik Sdn. Bhd.

INSTRUCTIONS FOR USE




1. PULL the nozzle tab upwards and discard.
DO NOT TWIST!


2. TWIST the air flow tab and discard.



3. Fasten the connector to hümiFlö®

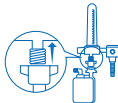

4. Mount connector to the oxygen outlet.

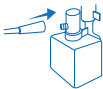

5. Connect the oxygen tubing.



6. Turn on the oxygen supply.

hümiFlö®

Should form bubbles. Check for leakages.