



---

**BRIGHTER SMILE GUARANTEED**

---

## **INSTRUCTIONS MANUAL**

Product of TLC Diagnostics Pty Ltd  
Medescan Australia  
1300 736 330  
[info@medescan.com.au](mailto:info@medescan.com.au)  
[www.medescan.com.au](http://www.medescan.com.au)

## INTRODUCTION

The Medescan Smile-Brite teeth whitening system utilises special cold light technology and the specially formulated, light activated, Carbemide peroxide to achieve maximum brightness in a short period of time. The powerful LED light reacts with the teeth whitening gel to remove stubborn surface and deeper stains.

The soft mouthpiece is ergonomically designed for a comfortable fit and complete coverage for whiter and brighter teeth.

# CERTIFICATION



**FDA**



**CE**



**CPSR**



## HOW TO GET THE BRIGHTER SMILE

1. Brush your teeth gently with toothpaste and rinse. (Figure 1)
2. Rinse and dry the mouth tray provided.
3. Insert batteries into the teeth whitening light unit. (Figure 2)
4. Attach the mouth tray to the teeth whitening light unit. (Figure 3)
5. Apply a small amount of Whitening Gel into both sides of mouth tray, top and bottom. Apply just enough to coat the frontal surfaces of your teeth but not your gums. Each syringe should last approximately 4 uses. (Figure 4)
6. Place the mouth tray over your teeth. Wipe off any excess gel that comes in contact with your gums. (Figure 5)
7. Press the start button to turn on the teeth whitening light unit and activate the teeth whitening gel. Close your mouth around around the mouth tray. (Figure 5)
8. Remove the mouth tray after 10 Minutes. Do not use for longer than 30 minutes. Once unit is removed, rinse your teeth with warm water. (Figure 6)
9. Detach the mouth tray from the teeth whitening light unit, and clean thoroughly the tray only with warm water .
10. Repeat this procedure once daily until desired result is achieved. This normally occurs within the first 7 days.



Figure 1



Figure 2



Figure 3



Figure 4



Figure 5



Figure 6

## GENERAL INFORMATION



### **Ingredients:**

Medicinal Hydrogen Peroxide, Carbomer, SiO<sub>2</sub>, WDJ-38, Glycerinum, Potassium Sorbate, Xylitol, Deionized Water.

### **Suitable for:**

1. Yellow teeth or badly stained teeth caused by smoking, drinking tea, coffee and or other reasons.
2. Age-related yellow teeth.
3. Light or moderate tetracycline pigmentation teeth and dental fluorosis.
4. Those who want a whiter and brighter smile.

### **Storage:**

Avoid direct sunlight and store in a cool area.

### **Precautions:**

1. Rinsing thoroughly. The water on the teeth's surface should be dried entirely before applying gel evenly to the teeth.
2. Excessive saliva will dilute the gel's concentration.
3. Everyone's teeth are different, few people will feel slight sensitivity, which is a normal sensation. You can apply your FREE whitening repair essence, provided in your MedeScan Smile-Brite kit, to ease discomfort. If your sensitivity is severe, please reduce the duration and the frequency the Smile-Brite gel is applied. If sensitivity is extreme or continues, please discontinue use and seek medical advice.



4. Avoid directly applying teeth whitening gel to gum area. Wipe off excess.
5. If the gel comes into contact with skin or clothing, please rinse immediately.
6. Please replace cap on the top of syringe after finishing teeth whitening treatment to avoid loss of active ingredients.
7. Please store MedeScan Smile-Brite out of children's reach.

### **Caution**

1. Not suitable for ceramic teeth and false teeth.
2. Not suitable for teeth discoloration caused by lesions or medicine.
3. Not suitable for severe tetracycline and decayed teeth.
4. Not suitable for defective enamel, exerted dentin and damaged teeth.
5. Not suitable for children under 12 years of age.
6. Not suitable for pregnant women.



---

**BRIGHTER SMILE GUARANTEED**

---

# **INSTRUCTIONS MANUAL**

Product of TLC Diagnostics Pty Ltd  
Medescan Australia  
1300 736 330  
[info@medescan.com.au](mailto:info@medescan.com.au)  
[www.medescan.com.au](http://www.medescan.com.au)