

HALO Retrofit Kit for OCTO Respirator Mask V 1.5

This HALO Strap Retrofit Kit converts your ORM version 1.5 to version 1.5 HALO. Installing it will improve comfort and allow for very quick donning and doffing.

This kit only fits ORM Version 1.5.
Do not install this on any other ORM version.

The kit consists of a bracket-and-strap-system.
Installing the kit requires permanent removal of your ORM's existing TPU straps.

Before installation, please carefully check the Retrofit Kit for any damage, as you will not be able to re-install the TPU straps.

To install, follow these simple steps:



Step 1

Cut both TPU straps with scissors and pull them out of your ORM.



Step 2

Your ORM should look like the image on the left now.



Step 3

Hold the bracket perpendicular to the ORM, and align the bracket's inside clips into the upper front-shield openings.



Step 4

Push one and then the other clip into the openings until the bracket surface is flat against the ORM, and then tilt down the bracket onto the ORM.

Confirm that the clips are fully inside the upper front-shield openings, and continue pushing down the bracket until it clicks into place.



Step 5

Finally, push down the bracket area on both sides of its upper section.



Step 6

Done.

Please read the enclosed instruction manual for the ORM v1.5 HALO to learn how to adjust and don and doff your respirator with the new HALO strap.