

3M™

Animalintex®

Hoof Poultice Pad Pre-cut hoof shape Aids in relief of equine foot injuries and conditions Aids in wound healing and relief of minor stiffness and soreness Convenient, no mess, easy to apply and remove. Ready to use poultice in a pad – just add water to pads

Designed especially for the equine foot.

Contains: Tragacanth 1.12g/dressing Boric Acid 0.39/dressing.

How To Use:

Can be used as a hot or cold wet poultice, or as a dry wound dressing. 1. Gently clean application area. 2. Place poultice in clean, shallow container (plastic backing on top). 3. For hot wet poultice, pour hot, potable water (38°C/100°F) around edges of poultice in container. 4. For cold wet poultice, use cool, potable water. 5. When saturated, gently squeeze out excess water. 6. Position poultice (plastic backing away from sole of foot). 7. Cover poultice completely with 3M™ Vetrap™ Bandaging Tape. Dry Wound Dressing 1. Gently clean application area. 2. Position poultice (plastic backing away from sole); cover poultice completely with 3M™ Vetrap™ Bandaging Tape.

Caution: Change Poultice At Least Once Every 12 Hours

For External Use only. For Animal Use Only.

Wash hands before and after use.

Do not apply too hot.

Apply poultice with the plastic backing away from animal.

Do not bandage too tightly.

If skin is broken, follow standard methods of wound care.

Indicated for use on horses and dogs. Not recommended for use on cats.

Made in England