



INNOVACYN, INC.

3546 N. RIVERSIDE AVE., RIALTO, CA, 92377

Toll-Free: 866-318-3116

Fax: 909-428-1947

Website: www.innovacyn.com

Website: www.vetericyn.com

Email: customerservice@innovacyn.com



THIS SERVICE AND DATA ARE PROVIDED "AS IS". DVMetrics assumes no liability, and each user assumes full risk, responsibility, and liability, related to its use of the DVMetrics service and data. See the Terms of Use for further details.

VETERICYN PLUS® ANTIMICROBIAL DRESSING GEL

Innovacyn

THICK BARRIER WOUND CARE

- + Use under dressings & bandages
- + Cuts & abrasions
- + Sores & irritations

Thick Wound Care Dressing Gel

Vetericyn Plus® Dressing Gel's thicker formula is best used with absorbent wound dressings & bandages for cuts & wounds. For best results, thoroughly clean with Vetericyn Plus® Wound Care.

Vetericyn Plus® Antimicrobial Dressing Gel adheres to the site of application, creating a wound protecting barrier.

- + For use under dressings & bandages
- + Helps coat, moisturize, & protect wounds, scrapes, abrasions, & minor wounds
- + Aids wound protection to help reduce the chance of potential infection
- + Safe for all animal species at all life-stages

Veterinarian recommended

Safe if licked or ingested

No antibiotics or steroids

Indications: Vetericyn Plus® Antimicrobial Dressing Gel is intended for the management of wounds including skin abrasions, lacerations, irritations, cuts, burns and intact skin; providing a proper moisture balance throughout the phases of wound healing. Relieves itch and pain associated with skin irritations. Optimal to use with dressings and bandages. Safe for all animal species at all life stages.

Directions for use: Shake well before use. Move excess hair from wound area. Clip if necessary. Open cap and invert bottle. Squeeze out gel and saturate the affected area and cover with a bandage, or wrap with dressing. For best results, flush and clean with Vetericyn Plus Wound Care spray if dirt or debris exist and use in conjunction with Vetericyn Plus Antimicrobial Dressing Gel for proper protection and moisture balance during healing stages. Repeat at each dressing or bandage change 3-4 times per day or as necessary. Faster healing is achieved when wounds are cleaned and maintained moist. No rinsing necessary. Safe for use around mouth, nose, ears, and eyes.

Use Precautions: External use only, not for injection. Animal use only, not for humans. Keep out of reach of children. Discontinue use if irritation or systemic condition develops and contact your veterinarian. Will not stain and contains no harsh chemicals or dyes.

Ingredients: Active: Hypochlorous Acid (0.012%) **Inactive:** Electrolyzed Water, Sodium Chloride, Sodium Hypochlorite, Lithium Magnesium Sodium Silicate, Sodium Bicarbonate, Phosphates, Sodium Sulfate

Storage & Disposal: Store at room temperature. Do not allow to freeze. Do not use after expiration date. After use, rinse nozzle with warm water to reduce chance of gel drying at the tip. Close Cap. No special disposal requirements.

Innovacyn products are manufactured in a FDA QSR compliant facility. Vetericyn Plus contains Hypochlorous Acid, a known antimicrobial, which serves as a preservative to inhibit the growth of microorganisms in the product.

Pet owners & vets love Vetericyn products!

b: Authentic Reviews

“I’ve trusted Vetericyn Plus® products to care for my horses for years. We always have a bottle available in our barn and in our trailer. It’s my first line of defense for everything wound care for my horses and dogs!”

Stevi Hillman, Professional Barrel Racer from Texas

Tell Us Your Story!

vetericyn.com/testimonial

Manufactured & Bottled by Innovacyn, Inc., 3546 N. Riverside Ave, Rialto CA 92377

vetericyn.com

866.318.3116

Made in the USA

	Part Number:
3 fl oz (89 ml)	1049; 10049B v1.0 1049; 10049 v1.0
8 fl oz (237 ml)	1041; 10041B v1.0 1041; 10041 v1.1

CPN: 1433052.0

THIS SERVICE AND INCLUDED DATA ARE PROVIDED "AS IS" WITHOUT ANY EXPRESS OR IMPLIED REPRESENTATION OR WARRANTY. Without limiting the foregoing, the service and data are based on third party data, and DVMetrics is not responsible or liable for such third-party data. Each user assumes full risk, responsibility, and liability related to use of the service and data. The service and data are further subject to the Terms of Use.

By using this site, you agree to the [Terms of Use](#) and [Privacy Policy](#).

[Contact Us](#) - [Logout](#)

Copyright 2023 - DVMetrics & Services LLC