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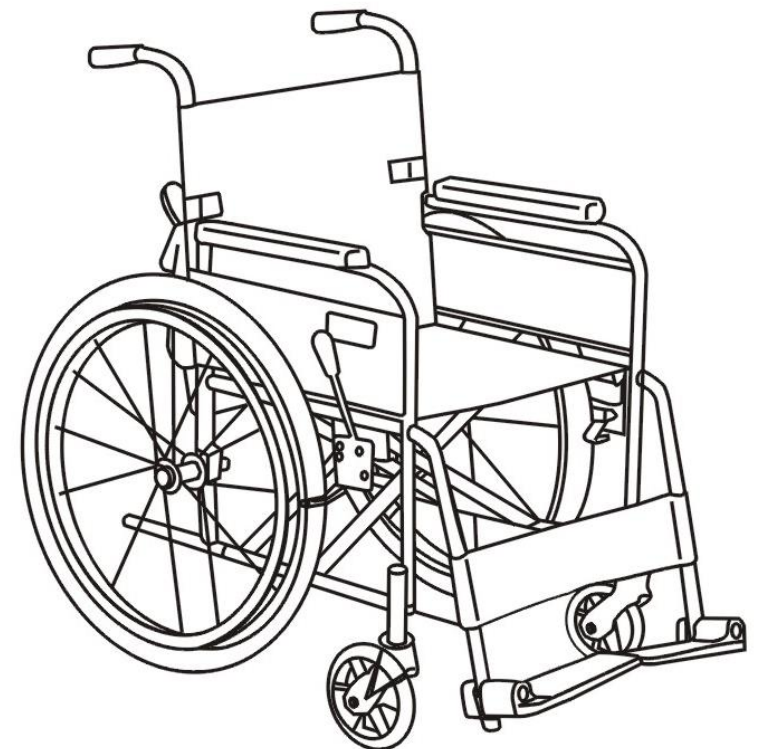
**Produttore:** Guangdong Dayang Medical Technology Co., Ltd.  
Changhongling Industrial Park (2nd Phase), Shishan, Nanhai District, Foshan, China. TEL: 0086-757-81821202 MOB.: 0086-13902809702  
EUDAMED ID/SRN: CN-MF-000003324

**EU Represent:** Sungo Europe B.V.  
Olympisch Stadion 24, 1076DE Amsterdam, NL  
ec.rep@sungoglobal.com +31(0)202111106  
EUDAMED ID/SRN: NL-AR-00000247

**Prodotti** Sedie a rotelle (codici prodotto e UDI-DI):  
AID04/DY01953LQ 6970006390969 DLO04/DY01909B 6970006390372  
KER07/DY01908LA 6970006390303 LIG07/DY01868LI 6970006390297  
MOV04/DY01868 6970006390082 OPE04/DY01953LQ 6970006390969  
PKA/DY01871LBJ 6970006390945 STS04/DY01904BD 6970006390952  
THI04/DY01951B 6970006390976 TOS25/DY01864LI 6970006390396

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This manual provides instructions for the use of self-propelled and transit wheelchairs. Read the following instructions carefully before using for the first time. The particular care taken in the design and manufacture of these products ensures a high safety and comfort.

Improper use of the wheelchair can cause damage, injury or accidents.

If the chair is used by other people, be sure to give them this instruction manual for reference.

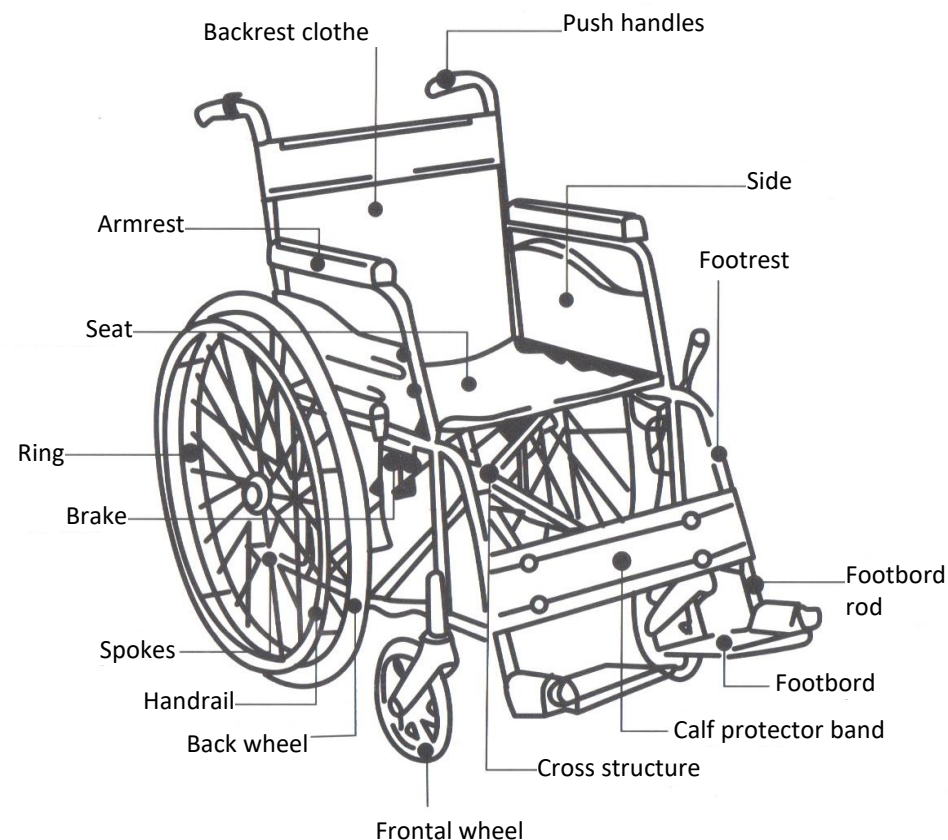
FloreoMed wheelchairs are suitable for adult patients who are able to independently carry out routes and transfers. The wheelchairs have been designed to be used both indoors and outdoors. They have the purpose of transporting a person on the seat with the leg rests and side parts mounted and do not serve as a means of traction, transport or other.

The wheels have solid or pneumatic tires for durability.

The foldable and lightweight design allows easy transport.

**INDEX**

Page 16 Components	Page 23 Footrest adjustment
Page 17 Legal Guarantee of Conformity	Page 24 Inclination
Page 18 Safety	Page 25 Maintenance
<i>Transportable weight</i>	<i>Repairs</i>
<i>Transfer from the wheelchair</i>	<i>Spare parts</i>
<i>Training</i>	<i>Disposal</i>
Page 19 Safety	Page 26 Warranty
<i>Danger of overturning</i>	Page 27 Information Notes
<i>Clamps</i>	<i>Intended use</i>
<i>Transport by car</i>	<i>Checks before use</i>
<i>Lifting the wheelchair</i>	<i>Washing instructions</i>
Page 20 Safety	<i>Device Life</i>
<i>Stairs and sidewalks</i>	<i>Label and CE marks</i>
Page 21 First use	Page 28 UE Declaration of Conformity
<i>Brake</i>	
Page 22 Opening and closing	
<i>Calf protector band</i>	
<i>Swing armrests</i>	

## LEGAL CONFORMITY GUARANTEE

(L.D. 206/2005 - Art. 128 Consumer Code)

Distributed by: Polirone Shop  
Via Ferri 138c - San Benedetto Po (MN) - Italy  
Store: Via dell'Alpino 7 Fraz. San Biagio – Bagnolo San Vito (MN) - Italy  
Tel. 0376 414066 Cell. 393 5305228 Cell. 393 2834704  
www.polironeshop.it info@polironeshop.it

Products: Wheelchairs models:

AID04/DY01953LQ	DLO04/DY01909B
KER07/DY01908LA	LIG07/DY01868LI
MOV04/DY01868	OPE04/DY01953LQ
PKA/DY01871LBJ	STS04/DY01904BD
THI04/DY01951B	TOS25/DY01864LI

Brand: FloreoMed

We hereby declare that the products indicated in this document satisfies all the applicable provisions of the "Consumer Code" and is covered by a Legal Guarantee if there is a lack of conformity within 24 months from the date of purchase (the purchase document is valid as proof).

POLIRONE SHOP also declares that the indicated products

- fully correspond to what is described on the online sales portal;
- are sold without any lack of conformity;
- are suitable for the use for which goods of the same type are usually used, but also for the specific uses that may be indicated;
- have the usual quality and performance of a good of the same type, which the consumer can reasonably expect.

## SAFETY

Use wheelchairs only as provided in the instructions for use. To stop, get on or off, always choose a flat and stable floor, secure the wheelchair to prevent it from moving using both brakes. If exposed directly to the sun's rays, the upholstery and padding of the seats and armrests, legrests and handles can reach temperatures over 41°C. When these parts come into contact with unprotected skin there is a risk of burn. To prevent these parts from heating up, park your wheelchair in the shade. When activating the adjustment devices (e.g. of the legrest), be careful and never put your hands in the component orientation area. There is a risk of injury. When transporting objects, make sure that they do not impair or restrict the maneuverability of the wheelchair. Check the pressure and wear of the tires and check the brakes.

### Transportable weight

Don't overload the wheelchair with excessive weights and never use it to carry more than one person. The maximum permissible load is shown on the wheelchair label or in the "Product Sheets" available on the website [www.polironeshop.it](http://www.polironeshop.it).

### Transfer from the wheelchair

If done independently, getting out of the wheelchair to go to another place, is dangerous. So ask for help, study the operation together and think that unexpected dangers may arise, such as any stumbling points. Please explain the type of your handicap. When the wheelchair is not loaded, for example when moving from the wheelchair to the bed, toilet or car, never force on the platform (danger of overturning).

**IMPORTANT: Before making the transfer make sure the brakes are applied and, if possible, lean the back of the chair against a wall.**

### Training

For safe use of the wheelchair, both indoors and outdoors, training is required to gain safety in the various main situations, such as: pushing, braking, driving forward, cornering, reversing on flat surfaces, possibly with the help of a companion. Avoid steep climbs, bumpy roads or other similar situations initially.

### Danger of overturning

Avoid bowing excessively, forward, sideways or backwards, to pick up or place objects: you run the risk of overturning or losing balance of the wheelchair, especially if it is equipped with a seat of reduced width and high height. Before stepping off a sidewalk alone, take a good look at its height. If it is too high, there is a risk of overturning.

### Clamps

When driving on stretches with narrow carriageways, for example building entrances, doors, etc., be careful with your elbows, risk of injury. Lifts and elevators constitute a particular danger for the user; doors that close automatically involve the risk of injuring your hands. Don't push to cross an obstacle and don't use the sides of a door to push yourself through a passage.

### Transport by car

The wheelchair must never be used as a seat when using a car or other means of transport. Disassemble the wheelchair (as specified in the instructions for use) and store it inside the car, or otherwise secure it in ways to avoid accidents in the event of dangerous maneuvers. Occupy a suitable seat inside the car or means of transport and fasten your seat belts.

### Lifting the wheelchair

Two assistants are required for lifting, each capable of carrying at least half the total weight of the person and the wheelchair.

Each helper must bend his knees and grasp the main frame: the front assistant must place his hands just under the front of the seat, the rear assistant must place his hands just under the push handles that hold the frame.

**IMPORTANT: do not lift the wheelchair from any removable part (armrests, push handles, etc.)**

### Stairs and sidewalks

Do not attempt to climb or descend steps higher than 30-50 mm. When it is possible ask for help from an assistant.

To tackle a step, the assistant will have to:

- Check that both the user's feet are on the footrests and can't slip.
- Release both brakes
- Firmly grasp both push handles
- Place the strongest foot on the tip lever on the back of the chair
- Warn the person in the chair that it is about to tilt backwards
- Pull back with both hands and push down and forward with foot.

**IMPORTANT: When a chair is tilted approximately 25 degrees it will balance on the rear wheels and be easy to maintain. If the inclination increases, it will be difficult to maintain the position and the person in the chair will not feel safe.**

For your safety, we recommend that you ask for the help of two people to go up and down difficult stairs. To go down the stairs, the wheelchair must be overturned on the large rear wheels and accompanied step by step. To go up, the person who helps you, turning his back to the stair, has to pull the wheelchair towards him from step to step, keeping it overturned on the rear wheels.

**IMPORTANT: never use an escalator for transport, choose the lift**

## FIRST USE

Before using the wheelchair, check that all parts have been correctly assembled and tightened, especially the footrests. Check that the parking brakes are working correctly. By inserting the brakes on both wheels, and giving the chair a slight force forward and backward, the wheels must not rotate.

The wheelchair is intended for disabled people who carry out intense outdoor activities. It is ideal both for indoors (home, workplace) and for outdoor leisure time, as long as the support ground is solid and not slippery.

### Brake

The parking brake is one of the main safety elements of a wheelchair and is present as a pressure brake, which acts directly on the tires. Brake your wheelchair with great caution when you are going backwards. This caution should be taken in particular when going backwards on a slope. Abrupt braking on the handrims or using the brake levers can cause the wheelchair to tip over.

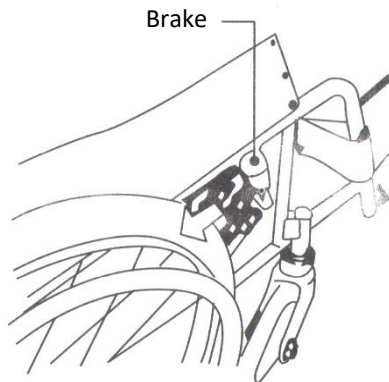
Brakes must always be kept in full efficiency: the braking effect of the pressure brakes can be reduced and not guaranteed if:

- the tire profile is worn
- the tires are wet
- the tire pressure is low


Check regularly and before each ride:

- the pressure of your wheels (7.0 bar).
- check that the spokes of the wheels are well pulled.

**IMPORTANT: Be careful that the brakes are blocked when going up / down. Brakes are designed to hold steady the chair. They must not be used as brakes during movement.**

## OPENING AND CLOSING

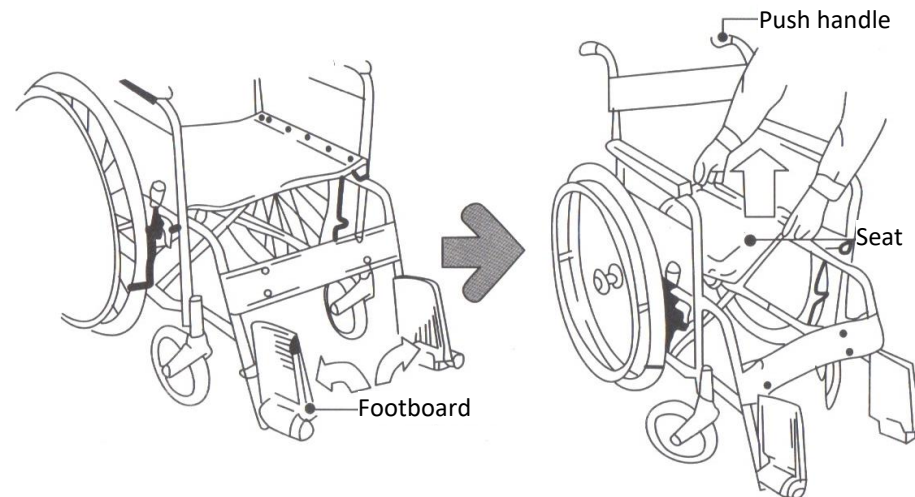Wheelchairs can be opened and closed easily, without the use of tools, with just a few manual operations.

Opening:

- spread the push handles
- push the central part of the seat down until it stops
- lower the footrests or mount the leg rests (where necessary)

Closure:

- raise the footrests
- raise the central part of the seat cover

### Calf protector band

The calf protection band prevents the legs from sliding off the footboard.

### Swing armrest (when available)

The swing armrests help the transfer from the wheelchair to the bed, chair, etc. and provide lateral body support. They allow greater adaptability and comfort when moving (sliding under tables, desks, etc.). The armrest can be moved backwards by pressing the release lever located at the front base of the armrest.

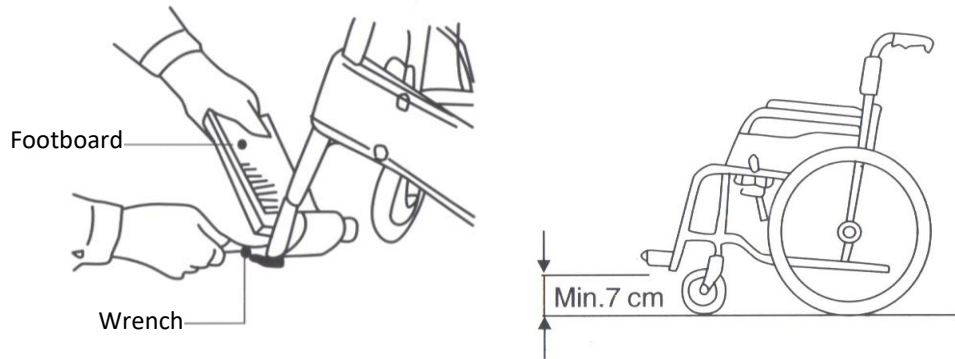
**IMPORTANT: never lift the chair by the armrests.**

## FOOTREST ADJUSTMENT


Height adjustment of the footboard:

- loosen the bolt and adjust it to the desired height
- screw the bolt in order to lock the footboard rod


Adjust the footboard to have a distance between the platform and the ground of about 7 cm

**IMPORTANT: When getting on and off the wheelchair, always raise the footrests and do not climb on them, they could break and cause falls.**

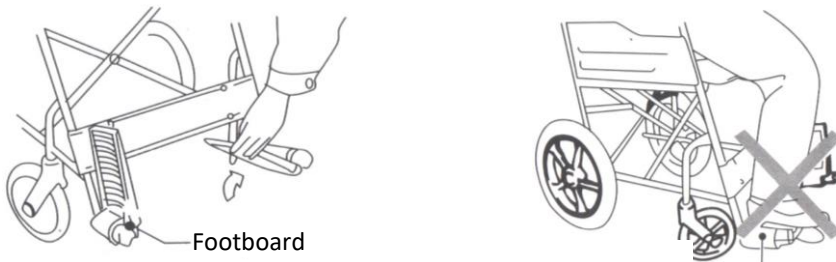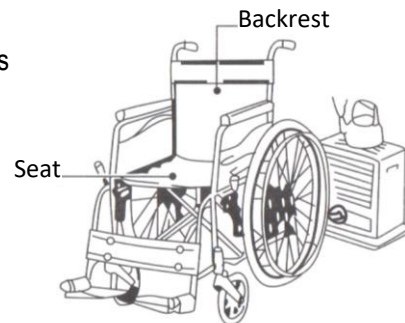Do not place the chair near heat sources; some parts could be damaged or overheated causing injury.


## INCLINATION

Do not lean over the edge of the wheelchair as this action could cause you to lose your balance and fall.

To go uphill, lean your body forward; for the descents, on the other hand, keep yourself well supported by braking the wheelchair. In the case of slopes with a gradient greater than 1%, it is recommended to be pushed by a second person.

**WARNING: on uphill roads do not carry bags or heavy bags behind the backrest. Danger of overturning**

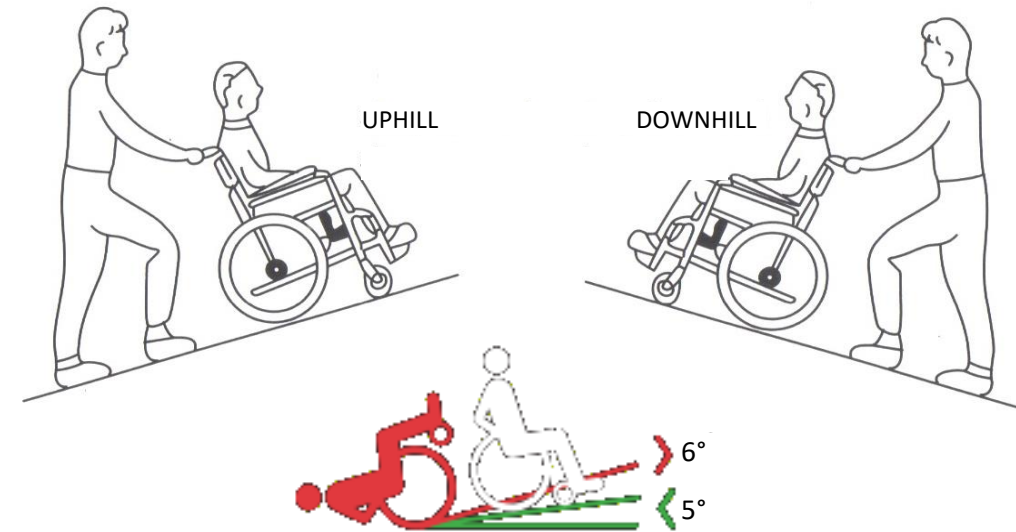When you get assistance continue straight up a slope. If you need to go downhill, walk backwards until you are on level..

Avoid standing on an inclined floor as this can cause you to lose control of your wheelchair. Do not use the brakes to slow down or stop the wheelchair, this could cause a sudden block or pull sideways, causing you to lose control of the wheelchair.

## MAINTENANCE

Maintenance must be done once a year. It is advisable to carefully check the condition of the bearings; sand, water and dust will damage them. Lightly grease with bicycle oil (resin-free).

To clean the frame, use warm water with a little soap or a household detergent. The parts in synthetic material must be cleaned with common detergents specific for this material.

Warning! Do not use high pressure cleaning devices! Danger of damage to the mechanical part caused by jets of water. The chairs must not be cleaned with aggressive detergents and cleaning items.

All components that may come into contact with the skin, if the wheelchair passes from one user to another, must be replaced or properly disinfected.

**IMPORTANT: Before each use, check that the tires are not worn or dirty, this would cause the brakes to malfunction. Check brake operation and tire inflation.**

### Repairs

For repairs, rely on your specialized dealer "PolirOne Shop", who has been trained in our factory according to our specifications, and is able to offer advice, assistance and repair services.

### Spare parts

Order spare parts from your "PolirOne Shop" dealer. To determine the correct spare part it is necessary to communicate the frame number of the wheelchair. For safety reasons, use spare parts and accessories authorized by PolirOne Shop.

### Disposal

The wheelchair packaging can be taken for material recycling. The metal parts are collected and recycled in the metal. The fabric and plastic parts are recycled. Disposal must take place in accordance with national and local laws.

## WARRANTY

The 24-month warranty we apply to our products reflects the confidence we place in the quality of our design, engineering and manufacturing, as well as performance.

We guarantee that this product has been manufactured in accordance with current European safety requirements and quality standards applicable to this product, and also that this product is free from defects related to workmanship or materials used at the time of purchase.

Our 24-month warranty covers any material and manufacturing defects if the product is used under regular conditions and in accordance with our instruction manual.

To request repairs or spare parts under warranty for any defects, proof of purchase must be presented (which must have been made no later than 24 months prior to the request for assistance).

The warranty does not cover damage caused by normal use, accidents, incorrect use, negligence or other consequences resulting from failure to comply with the instruction manual. Examples of normal use include parts such as: wheels and fabrics worn by regular use of the product, as well as the natural deterioration of colors and materials following prolonged use.

What to do in case of defects?

If any problems or defects occur, contact your dealer and ask for assistance.

Any damage not covered by the warranty can however be managed according to the price list.

## INFORMATION NOTES

The medical device described in this manual has the CE mark as it complies with the rules established by the EU Regulation 2017/745 MDR and is classified as non-invasive, non-sterile medical devices.

### Intended use

The device is supplied already assembled. Your wheelchair has been designed to be used both indoors and outdoors; uneven, sandy soils are not recommended. It is recommended to use it autonomously only for "active" people. Read the Instruction Manual carefully for correct use of the device.

### Checks before use

Before using your wheelchair, check that all parts have been correctly assembled and tightened, especially the footrests. If the product shows visible damage, the damaged parts must be replaced to ensure your safety.

### Washing instructions

Do not use high pressure cleaning devices! Danger of damage to the mechanical part caused by jets of water. The chairs must not be cleaned with aggressive detergents and cleaning items.

### Device life

The duration of the wheelchairs depends on the time of use and the intensity and use to which they are subjected. Correct use and adequate maintenance allows to maintain and extend the life of the device; on average it is estimated to last between 5 and 7 years.

### Label and CE marks

The CE label (see image) is stuck on the wheelchair.

## UE Declaration of Conformity

Medical Devices Regulation (EU) 2017/745 – MDR

**Importer:** PolirOne Shop di Cestaro Paola  
Via dell'Alpino 7 Bagnolo San Vito (MN) - Italy  
tel. 0376-414066 www.polironeshop.it info@polironeshop.it  
EUDAMED ID/SRN: not available yet

**Producer:** Guangdong Dayang Medical Technology Co., Ltd.  
Changhongling Industrial Park (2nd Phase), Shishan, Nanhai District,  
Foshan, China. TEL: 0086-757-81821202 MOB.: 0086-13902809702  
EUDAMED ID/SRN: CN-MF-000003324

**EU Represent:** Sungo Europe B.V.  
Olympisch Stadion 24, 1076DE Amsterdam, NL  
ec.rep@sungoglobal.com +31(0)202111106  
EUDAMED ID/SRN: NL-AR-000000247

**Products:** Wheelchairs (products codes and UDI-DI):  
AID04/DY01953LQ 6970006390969 DLO04/DY01909B 6970006390372  
KER07/DY01908LA 6970006390303 LIG07/DY01868LI 6970006390297  
MOV04/DY01868 6970006390082 OPE04/DY01953LQ 6970006390969  
PKA/DY01871LBJ 6970006390945 STS04/DY01904BD 6970006390952  
THI04/DY01951B 6970006390976 TOS25/DY01864LI 6970006390396

**Classification:** Medical Device - Class I, not-invasive, not-sterile, not intended for clinical investigations (according to the classification EU Regulation 2017/745 Annex VIII)

The manufacturer certifies and declares, under its own responsibility, that the products indicated in this document comply with the EU Regulation 2017/745 MDR on Medical Devices. The conformity assessment was carried out in accordance with Annex IV of the European Regulation EU 2017/745. The technical documentation proving compliance with the Directive is kept and made available by the importer to the Competent Authorities for a period of 8 years.

<sup>1</sup> San Benedetto Po, 30/03/2022

<sup>1</sup> This document is valid for 5 years from the date of issue

PolirOne Shop di Cestaro Paola  
Via Ferri, 138/c - 46027 S. Benedetto Po (MN)  
Partita IVA: 02252900200  
Cod. Fisc.: CST PLA 77B48 E897W  
www.polironeshop.it info@polironeshop.it