


## Instructions for ECO CUP, ECO II CUP & ECO III CUP

Step 1



Tear the foil bag open, remove test cup and disposable gloves provided for the donor. Label the device with the donor's information.

Step 2

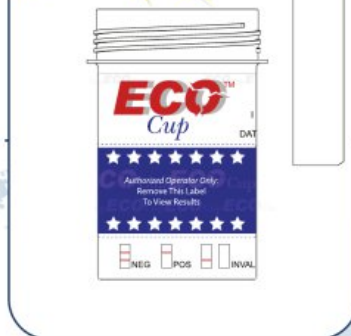

Wear disposable gloves to collect urine specimen. Open the test cup lid. Urinate directly into the test cup, filling between minimum and maximum as marked on the cup with the urine specimen.

Step 3



After the urine specimen has been collected, close the lid securely and return the cup to a collection official.

### FOR IN VITRO DIAGNOSTIC USE ONLY

AMP AMP AMP	Amphetamine Urine Test
BARB BARB BARB	Barbiturates Urine Test
BENZO BENZO	Benzodiazepine Urine Test
COC COC COC	Cocaine (Benzoyllecgonine) Urine Test
THC THC THC	Marijuana (THC) Urine Test
MDMA MDMA	MDMA (Ecstasy) Urine Test
METHADONE METHADONE	Methadone Urine Test
METH.AMP	Methamphetamine Urine Test
MOR MOR MOR	Morphine Urine Test
OXY OXY	Oxycodone Urine Test
OPI OPI OPI	Opiate Urine Test
PCP PCP PCP	Phencyclidine Urine Test
TCA TCA TCA	Tricyclic Antidepressant Urine Test

### FOR FORENSIC USE ONLY

BUP BUP BUP	Buprenorphine Urine Test
EDDP EDDP EDDP	EDDP Urine Test
PPX PPX PPX	Propoxyphene Urine Test

Step 4

The collection official, will use the gloves provided. Peel off the label to reveal the test results. Read test results at 5 mins. **DO NOT INTERPRET RESULTS AFTER 10 MINS.**

**RESULTS**

C T NEGATIVE POSITIVE INVALID