


EPC500 & ESC2 Pendant Quick Guide

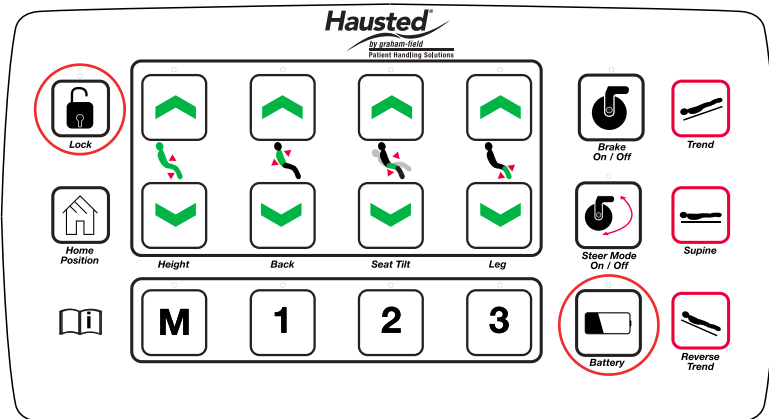


Lock

Pendant Lock

Press and hold. All UP-LED's will flash five times, then become solid and followed by three audible beeps indicating locked.



Lock

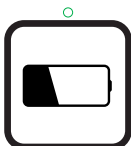

Battery

Pendant Unlock

Press and hold BOTH buttons at the same time. All UP-LED's will flash with three audible beeps indicating unlocked.

All UP-LED's will continue to flash until BOTH buttons are released.



**Home
Position**

Home Position

Press and hold. Chair automatically raises back section, lowers leg section, tilts seat section to 6° simultaneously, and adjusts chair height to lowest position (20").



To Pre-Set Position 1, 2, or 3 for OR, Pre-Op or Surgeon Specific

Step 1:

Have chair in the desired position


Step 2:

Press and hold M button, green light will begin to flash.

Holding the M button down, press 1, 2, or 3 and hold both down until the audible beeping stops and the green light on the M button goes out. Memory is now set for the selected number.

A solid green light confirms your position has been set.

This light will go out once both buttons are released.

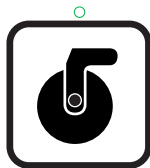


Brake
On / Off

Caster Brake

Press ONCE to lock all casters (audible beep and LED on)

Press again to unlock all casters (audible beep and LED off)

To prevent unintended movement, brakes lock automatically after chair is stationary for 30 consecutive seconds. Before brakes engage, a Brake alarm will sound (5 quick beeps) 5 seconds before.

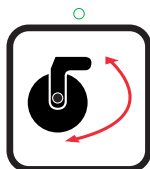


Steer Mode
On / Off

Caster Steer Mode

Press ONCE to STEER chair (audible beep and LED on)

Press again to return to neutral (audible beep and LED off)

Note: Engaging Steer mode through the pendant only locks the Foot-End casters into steer mode.

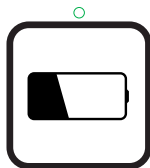

Battery

LOW Battery Indicator

Illuminates steady green when battery is 20% or less.

Chair will also emit beeping sound when raising or lowering