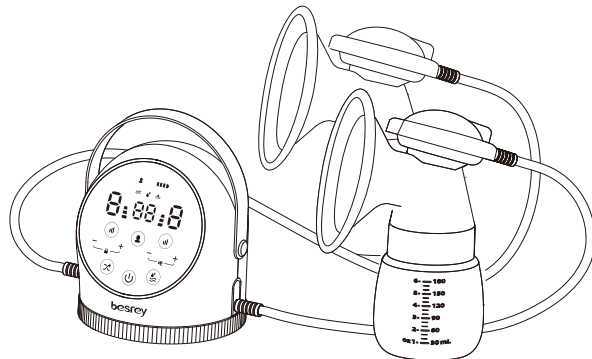
**besrey**<sup>®</sup>

**User Manual**  
**Electric Breast Pump**  
Model: XN/MS-2246A

**IMPORTANT! READ CAREFULLY AND KEEP  
IT FOR FUTURE REFERENCE**

# Contents

---

EN	-----	01-17
ES	-----	17-34

---

Overview -----	04
Specifications -----	04
Control Panel -----	05-07
Cleaning and Sterilizing Before Use -----	08-10
How to Assemble the Breast Pump -----	10-12
How to Use the Breast Pump -----	12-13
Using the Expressed Milk -----	14
Storing the Expressed Milk -----	14
Trouble Shooting -----	14-16
Care and Maintenance -----	17
Disposal -----	17

# **WARNING**

**WARNING!** Please read this manual carefully before using the product.

- It is recommended you consult with a licensed healthcare professional before using the pump if you have an active implanted device that would be affected by a magnetic field (e.g. pacemaker, implantable cardioverter defibrillator)
- Pumping can induce labor. Do not pump until after giving birth. If you become pregnant while breastfeeding or breast pumping, consult with a licensed healthcare professional before continuing.
- All breast pump components that come into contact with breast milk should be disassembled, washed, and sterilized for the first time use and before each use.
- Drinking water is recommended for cleaning breast pump components that come into contact with breast milk.
- The parts that come in contact with breasts and breast milk should be disassembled and cleaned as soon as possible after each use to prevent the growth of bacteria that may be caused by residual milk.
- All parts should NOT be exposed to ultraviolet light and should NOT be sterilized at excessively high temperatures (<100°C/212°F) for a long time (<5 minutes).
- Keep the pump dry. Clean the pump unit with a soft cloth, don't rinse/disinfect it.
- Take care when cleaning the silicone flange and diaphragm. The breast pump will not function properly if they get damaged.

- You should avoid rubbing or inserting objects when cleaning the silicone diaphragm. Otherwise, it may be damaged.
- Do not disassemble the pump unit; pump repairs can only be performed by an authorized service provider.
- Never use the pump if it has been damaged, is not working properly, or the pump unit has been dropped into water.
- This is a single-user pump and therefore should not be used by more than one individual.
- Sharing the pump among multiple users may present a health risk.
- Do not use the pump while sleeping or being overly drowsy.  
This product contains small parts and a cable that may present a hazard. Keep out of reach of children.
- The pump should not be used while lying down or engaging in strenuous exercise or activities with the risk of harm.

**Operation is subject to the following two conditions:**

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

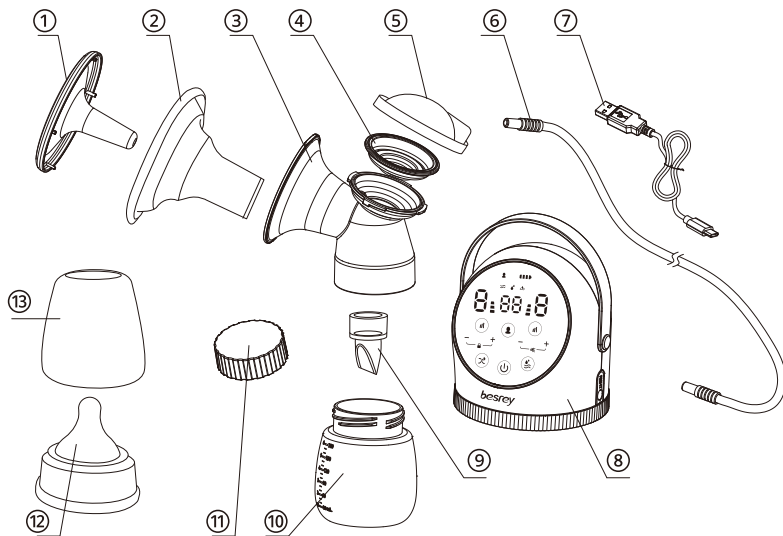
**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with

the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

## Overview


- ① Dust Cover x 2
- ② Flange x2
- ③ Breast Shield Body x2
- ④ Diaphragm x2
- ⑤ Diaphragm Cap x2
- ⑥ Tubing x2
- ⑦ USB-C Cable x1
- ⑧ Pump Unit x1
- ⑨ Duckbill Valve x2
- ⑩ Milk Bottle x2
- ⑪ Milk Bottle Lid x2
- ⑫ Teat x1
- ⑬ Teat Cap x1


## Specifications

Model: XN/MS-2246A	Battery: 3.7V	Suction Level: 9 Levels
SPU: BR-HL003	Charging Port: Type C	Suction Cycle: 3 Cycles (Slow/Middle/Fast)
Power Input: 5V=2A	Charging Time: 2.5 Hours	
Rated Power: 10W	Suction Mode: 3 Modes	
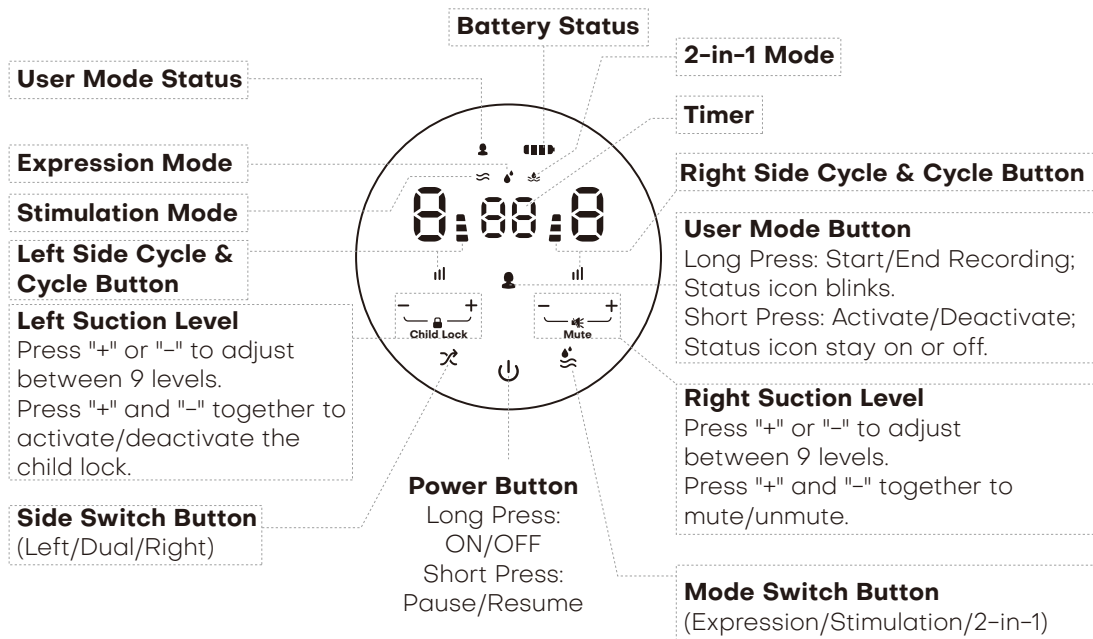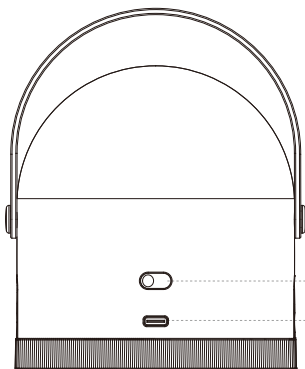
# Control Panel

**Note: This pump is powered by a built-in li-ion battery. When the battery status indicator shows one bar of power, it is recommended to charge the pump as soon as possible using a certified 5V=2A adapter (not included).**

Night Light Button  
Charging Port

1. Fully charge the device before first use. Long press the power button “⏻” to turn on the pump unit, and the battery status icon “🔋” will stay on to show the remaining battery. Charge the pump unit with the USB cable and power adapter (not included) when the battery is low (one bar of power).
2. When the pump is turned on, press the side switch button “↔” to choose which side to pump or to begin pumping on both sides; press the cycle button “⏮” to select the optimal suction cycle.
3. Press the mode switch button “👤” to select a preferred mode. The icon “≈” will remain visible while in stimulation mode. After 2 minutes, the pump will automatically switch to expression mode, and the icon “💧” will remain visible. The 2-in-1 mode “👤💧” scientifically combines the above two modes to simultaneously stimulate and express breasts; use this mode as needed.

4. When your nipples feel the suction power in any mode and at any suction level, press "+" or "-" to adjust the suction to a comfortable level. There are 9 suction levels to choose from.
5. Press the "+" and "-" buttons on the left side together to lock the buttons and prevent accidental activation or adjustment. To mute or unmute the control panel, press "+" and "-" on the right side simultaneously. When the child lock is enabled and the panel is muted, the status icons "🔒" and "🔇" remain visible; otherwise, these icons become invisible.
6. While the pump unit is pumping, long press the user mode button "👤" to begin recording; the status icon "👤" will blink to show that it is recording your pumping process. When you're done recording, long press the user mode button "👤" to stop recording. You can use this function to personalize and save your own pumping process. To repeat the saved pumping process, activate user mode by short pressing the user mode button "👤" and the pump will detect and run the saved pumping process automatically. If no saved pumping process can be detected, it will beep and exit user mode after short pressing the user mode button "👤".
7. When finished pumping, long press the power button "⏻" to turn off the pump. Alternatively, the pump will shut off automatically after 30 minutes of pumping if no further operation.

## Cleaning and Sterilizing Before Use

The components that come into contact with breast milk are not provided sterile and must be cleaned and sterilized for the first use. Moreover, these components must be cleaned after each use to avoid the buildup of dry milk residue which can lead to the growth of bacteria that can be harmful to you and your baby.

Clean and sterilize these milk-contact components before first use and after every use: flange, breast shield body, milk bottle, milk bottle lid, milk bottle teat, and duckbill valve.

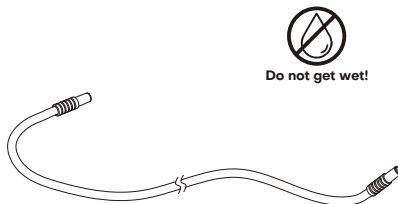
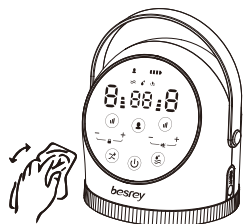
⚠ **WARNING:** Always take apart components before cleaning or sterilizing. After cleaning, make sure the components are completely dry.

⚠ **WARNING:** If the duckbill valve is clogged with milk, clean it gently with a cotton swab. Never rub or knead it with your fingers.


Clean Milk-contact Components	Sterilize Milk-contact Components
<ol style="list-style-type: none"> <li>1. Rinse in cold, clear water (approx. +20°C / +68°F)</li> <li>2. Wash in warm water (approx. +30°C / +86°F) using gentle baby dish detergent and wipe afterward with a clean cloth</li> <li>3. Rinse thoroughly with cold, clear drinkable water (approx. +20°C / +68°F) for 10 to 15 seconds</li> </ol>	<p>Boil in water for 5 minutes</p>
<b>OR</b>	<b>OR</b>
<p>Place on the top shelf of the dishwasher (but not for a long time)</p>	<p>Suitable for microwave, steam, or cold water sterilizing methods. Microwave or steam sterilizing should be less than 5 minutes</p>

Droplets can damage the electronics inside the pump unit. The pump unit, tubing, and the tubing air vents on both sides should always be kept dry.

**⚠ Warning:** Do not submerge the pump unit in water or any other liquid! Wipe the pump unit only with a clean, damp but not dripping cloth.

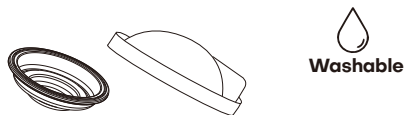The diaphragm and diaphragm cap do not come in contact with milk and should only be cleaned as needed.


Be careful not to get diaphragm and diaphragm cap to come in contact with milk. If there are milk droplets inside these parts, stop pumping immediately to avoid getting milk droplets inside the pump unit. Then clean these parts with water only and let them dry completely before using them again.


**⚠ Warning:** Do not use diaphragm, and diaphragm cap until completely dry.

## How to Assemble the Breast Pump


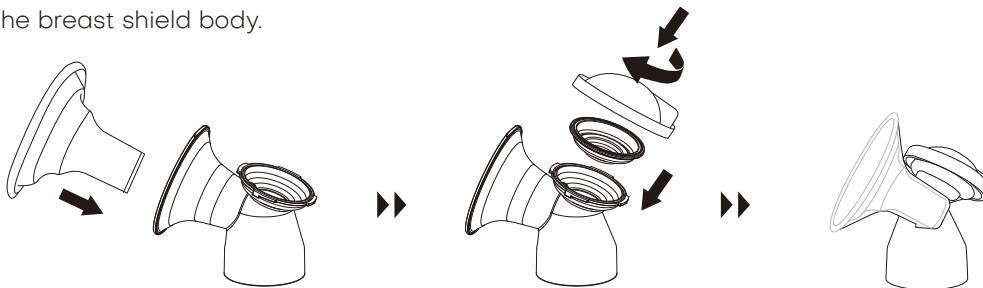
**Step 1.** Assemble components on the breast shield body: ① Insert the flange into the breast shield body. ② Put the diaphragm into the breast shield body, then secure the diaphragm cap onto the breast shield body.

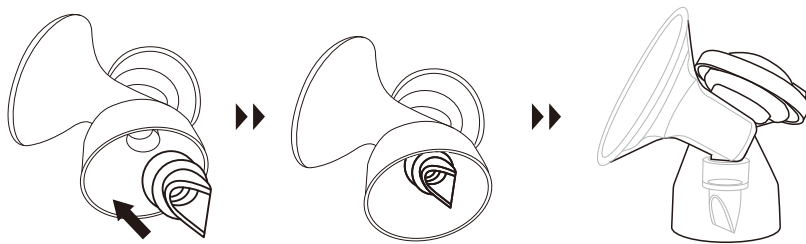
**Step 2.** Secure the duckbill valve on the breast shield body.

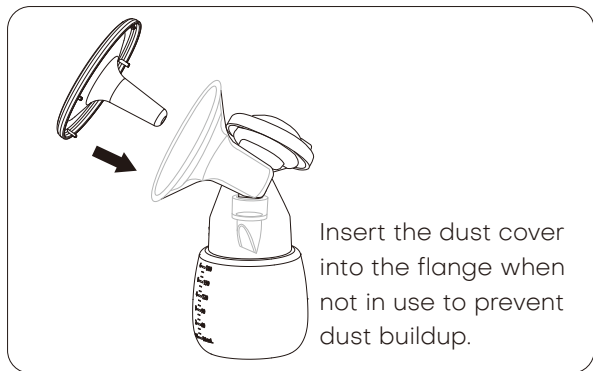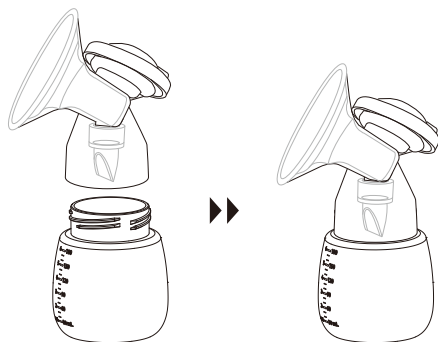**Step 3.** Carefully screw the milk bottle into the base of the breast shield body.

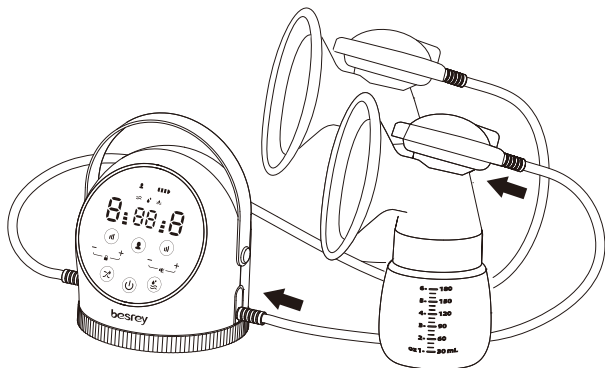
**Step 4.** Insert one end of the tubing into one tubing vent of the pump unit and the other end into the air vent on the diaphragm cap. Repeat the process to insert another tubing if double pumping.


**⚠ Warning:** Check that the tubing sets are securely attached to the diaphragm cap and the pump unit.

## How to Use the Breast Pump

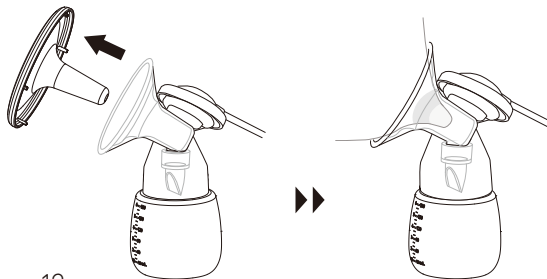**Step 1.** Remove the dust cover and bring the flange to your breast.

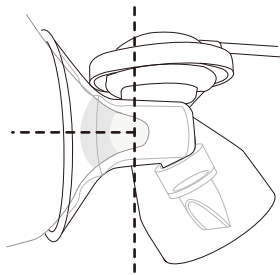
**Step 2.** Place the breast flange onto your breast. Check that the flange size fits your breast and nipple. Your nipple must be centered in the silicone flange tunnel before pumping to avoid discomfort.

**⚠ Warning:** Fit the flange tightly against your breast to prevent air leakage. Do not let the tubing get knotted when pumping.

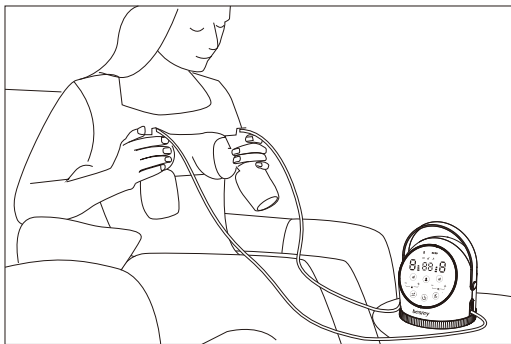**Step 3.** When you're ready, turn on the pump and set a comfortable suction level to begin pumping. Do not try to express with an intensity setting that is too high or uncomfortable. If excessive discomfort or pain is felt during pumping, stop pumping, detach the flange around your nipple and discontinue the session. Contact your healthcare professional if you express minimal or no milk, or if the expression is painful.


Note: To express milk when there is breast engorgement, use the lower suction level first to make the breasts unobstructed, then switch to the higher suction level, otherwise you may express minimal or no milk. In case of pain or discomfort during pumping, reduce the suction level or stop using the pump.

## Using the Expressed Milk

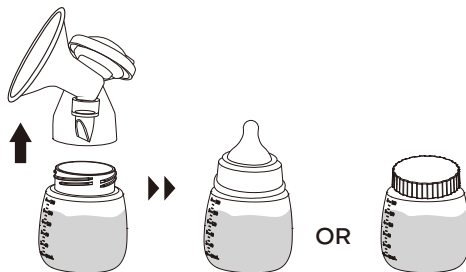Remove the breast shield cover, then attach the teat to feed the baby or attach the milk bottle lid to store the milk.

## Storing the Expressed Milk

When storing expressed milk, always follow the guidance of your lactation consultant or health professional. Here are some generic instructions for milk storage.

### Storing Breast Milk

- Use clean, food-grade, and sterilized breast milk storage bags or containers to store expressed breast milk. Make sure the containers are made of glass or BPA-free plastic and have tight-fitting lids.
- Never store breast milk in disposable bottle liners or plastic bags that are not intended for storing breast milk.
- Clearly label the breast milk with the date it was expressed. Storing breast milk in small quantities will help avoid waste.
- Do not store breast milk in the door of the refrigerator or freezer. This will help protect the breast milk from temperature changes from the door opening and closing.

- If you don't think you will use freshly expressed breast milk within 4 days, freeze it right away. This will help to protect the quality of breast milk.
- Leave about one inch of space at the top of the container because breast milk expands as it freezes.

### **Thawing Breast Milk**

- Always thaw the oldest breast milk first. Over time, the quality of breast milk can decrease.
- There are several ways to thaw your breast milk: a) In the refrigerator overnight. b) Set in a container of warm or lukewarm water. c) Under lukewarm running water.
- Never thaw or heat breast milk in a microwave. Microwaving can destroy nutrients in breast milk and create hot spots, which can burn a baby's mouth.
- If you thaw breast milk in the refrigerator, use it within 24 hours when the breast milk is completely thawed.
- Once breast milk is brought to room temperature or warmed, use it within 2 hours.
- Never refreeze breast milk after it has thawed.

### **Feeding Breast Milk**

- Place the sealed container into a bowl of warm water or hold it under warm, but not hot, running water for a few minutes.
- Breast milk should not be heated directly on the stove or in the microwave because the nutrients and antibodies in the milk will be destroyed.
- To avoid burns to the mouth of your baby, test the milk's temperature before feeding it to your baby by putting a few drops on your wrist.
- If your baby did not finish the bottle, use the leftover milk within 2 hours after the baby is finished feeding. After 2 hours, leftover breast milk should be discarded.

## Trouble Shooting

<b>Malfunctions</b>	<b>Cause</b>	<b>Solution</b>
Fail to turn on the pump.	<ol style="list-style-type: none"><li>1. The battery is damaged.</li><li>2. Low battery.</li></ol>	<ol style="list-style-type: none"><li>1. Contact the manufacturer.</li><li>2. Charge the pump unit.</li></ol>
Fail to charge the pump.	<ol style="list-style-type: none"><li>1. The charging port is damaged.</li><li>2. The USB cable is damaged.</li><li>3. The adapter is not compatible.</li></ol>	<ol style="list-style-type: none"><li>1. Contact the manufacturer.</li><li>2. Replace the USB cable.</li><li>3. Replace the adapter.</li></ol>
Low suction or no suction.	<ol style="list-style-type: none"><li>1. The diaphragm is damaged or not securely attached.</li><li>2. Didn't fit the flange tightly against your breast.</li><li>3. The tubing and the diaphragm cap are not securely attached.</li><li>4. The breast shield (flange) size does not match.</li></ol>	<ol style="list-style-type: none"><li>1. Contact the manufacturer or securely attach the diaphragm.</li><li>2. Always fit the flange tightly against your breast when pumping.</li><li>3. Ensure that tubing and the diaphragm cap are securely attached.</li><li>4. Before use, measure your nipples to ensure the breast shield (flange) is the right size.</li></ol>
Backflow	<ol style="list-style-type: none"><li>1. The diaphragm is damaged or not securely attached.</li><li>2. The duckbill valve is damaged or not securely attached.</li></ol>	<ol style="list-style-type: none"><li>1. Contact the manufacturer or securely attach the diaphragm.</li><li>2. Contact the manufacturer or securely attach the duckbill valve.</li></ol>

## Care and Maintenance

- Do not place the pump unit in the fridge or freezer.
- Do not immerse the pump unit in water or other liquids.
- Don't use a damaged charger, it may cause an electric shock.
- Do not clean the washable components in a dishwasher for an excessively long time.
- Do not store damp parts, as mold may develop. After cleaning, make sure the components are completely dry.
- Do not expose the pump unit to high temperatures and direct sunlight.
- Do not use harsh chemicals to clean the washable components. Use drinking-quality water and gentle baby dish detergent for cleaning components that come into contact with milk.
- Do not expose the device to high/low temperatures, and very high humidity levels. Store the device in a cool and dry place where it is protected from dust and damage.

## Disposal

This marking indicates that this product should not be disposed with other household wastes throughout the EU/UK. To prevent possible harm to the environment or human health from uncontrolled waste disposal, recycle it responsibly to promote the sustainable reuse of material resources. To return your used device, please use the return and collection systems or contact the retailer where the product was purchased. They can take this product for environmental safe recycling.

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Manufacturer: Guangdong Horigen Mother & Baby Products Co., Ltd.