Mölnlycke®

# Mepitel® Film

WITH SAFETAC® TECHNOLOGY

STERILE | EO

UK  
CA 0086

CE 2797




Manufacturer /Fabricant

 Mölnlycke Health Care AB  
Gamlestadvägen 3C, Box 13080  
SE-402 52 Göteborg, Sweden

[www.molnlycke.com](http://www.molnlycke.com)

# Mepitel® Film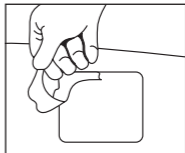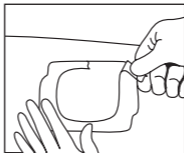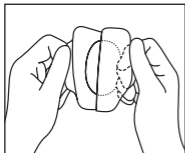

Patent: [www.molnlycke.com/virtual-patent-marking/](http://www.molnlycke.com/virtual-patent-marking/)

Single use  
Usage unique

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Mölnlycke Health Care AG  
Brandstrasse 24, 8952 Schlieren

UK Responsible Person:

Mölnlycke Health Care Ltd.,  
Medlock Street, Oldham, Lancashire, OL1 3HS, UK

Toll free number :

USA 1-800-882-4582  
Canada 1-800-494-5134

Australian sponsor address:

Mölnlycke Health Care Pty. Ltd.  
12 Narabang Way, Belrose, NSW 2085 AUSTRALIA

- en** Do not use if package is damaged
- de** Bei beschädigter Verp. verwerfen
- fr** Ne pas utiliser si l'emballage est endommagé
- es** No utilizar si el envase está dañado
- nl** Niet gebruiken indien verpakking beschadigd is
- sv** Skall inte användas om förpackningen är skadad
- it** Non utilizzare se la confezione è danneggiata
- fi** Älä käyttää jos pakkaus vaurioitunut
- pt** Não usar se embalagem danificada
- da** Brug ikke, hvis pakken er beskadiget
- el** Μην το χρησιμοποιήσετε αν η συσκευασία είναι κατεστραμμένη
- pl** Nie stosować, jeśli opakowanie jest uszkodzone
- cs** Nepoužívejte, je-li obal poškozen
- hu** Ne használja, ha a csomagolás sérült
- no** Må ikke brukes hvis forpakningen er skadet
- sl** Ne uporabite, če je ovoj poškodovan
- bg** Не използвайте, ако опаковката е увредена
- ro** Nu utilizați dacă ambalajul e deteriorat
- sk** Nepoužívajte, ak je obal poškodený
- tr** Ambalaj hasarlıysa kullanmayın
- lt** Nenaudoti, jei pakuotė pažeista
- lv** Nelietot, ja iepakojums ir bojāts
- et** Kahjustatud pakendi korral mitte kasutada
- ru** Не польз., если упак. поврежд.
- hr** Ne koristiti ako je pakiranje oštećeno
- sr** Ne koristiti ako je omot oštećen
- is** Notist ekki ef pakning er sködduð
- ar** لا يجوز استخدام الجهاز إذا كانت العبوة تالفة

## MD

<b>en</b>	Medical Device	<b>en</b>	Single sterile barrier system
<b>de</b>	Medizinprodukt	<b>de</b>	Einzelnes steriles Barriersystem
<b>fr</b>	Dispositif médical	<b>fr</b>	Système de barrière stérile unique
<b>es</b>	Producto sanitario	<b>es</b>	Sistema de barrera única estéril
<b>nl</b>	Medisch hulpmiddel	<b>nl</b>	Enkelvoudig steriel barrièresysteem
<b>sv</b>	Medicinteknisk produkt	<b>sv</b>	Sterilt singelbarriärsystem
<b>it</b>	Dispositivo medico	<b>it</b>	Sistema di barriera sterile singola
<b>fi</b>	Lääkinnällinen laite	<b>fi</b>	Yksinkertainen steriili sulkujärjestelmä
<b>pt</b>	Dispositivo médico	<b>pt</b>	Sistema único de esterilização de barreira
<b>da</b>	Medicinsk udstyr	<b>da</b>	Sterilt enkeltbarriersystem
<b>el</b>	ιατροτεχνολογικό προϊόν	<b>el</b>	Ενιαίο σύστημα αποστειρωμένων φραγμών
<b>pl</b>	Wyrób medyczny	<b>pl</b>	System pojedynczej bariery sterylnej
<b>cs</b>	Zdravotnický prostředek	<b>cs</b>	Systém jedné sterilní bariéry
<b>hu</b>	Orvostechikai eszköz	<b>hu</b>	Egyszeres sterilgát-rendszer
<b>no</b>	Medisinsk utstyr	<b>no</b>	Sterilt enkeltbarriersystem
<b>sl</b>	Medicinski pripomoček	<b>sl</b>	Enojni sterilni pregradni sistem
<b>bg</b>	Медицинско изделие	<b>bg</b>	Единична стерилна бариерна система
<b>ro</b>	Dispozitiv medical	<b>ro</b>	Sistem cu barieră sterilă simplă
<b>sk</b>	Zdravotnícka pomôcka	<b>sk</b>	Systém s jednou sterilnou bariérou
<b>tr</b>	Tıbbi cihaz	<b>tr</b>	Tekli steril bariyer sistemi
<b>lt</b>	Medicinos priemonė	<b>lt</b>	Vieno sterilaus barjero sistema
<b>lv</b>	Medicīniska ierīce	<b>lv</b>	Viena sterilās barjeras sistēma
<b>et</b>	Meditsiiniseade	<b>et</b>	Ühekordselt kasutatav steriilne barjäärisüsteem
<b>ru</b>	Медицинское изделие	<b>ru</b>	Одиночная стерильная барьерная система
<b>hr</b>	Medicinski proizvod	<b>hr</b>	Jednokratni sustav sterilne barijere
<b>sr</b>	Medicinsko sredstvo	<b>sr</b>	Jednokratni sistem sterilne barijere
<b>is</b>	Lækningatæki	<b>is</b>	Einfalt dauðhreinsað hindrunarkerfi
<b>ar</b>	جهاز طبي	<b>ar</b>	نظام حاجز معقم أحادي

# Mepitel® Film

Thin, transparent, adhesive, breathable soft silicone film dressing

## Product description

Mepitel® Film is a gentle, sterile, transparent, breathable film dressing. The film dressing is supported with a paper frame for ease of application.

Mepitel Film consists of:

- a silicone coated paper frame
- a breathable and waterproof polyurethane film
- a soft silicone (Safetac®)
- an embossed polypropylene film release liner

Material content:

Polyurethane, silicone, polypropylene and paper

## Indications for use

Mepitel Film is intended to protect fragile and sensitive skin and minimize risk of skin breakdown. It can be used as secondary dressing, fixating e.g. gauze or non-woven swabs or as a landing zone for aggressive tapes to secure devices to skin. It can be used as a protective dressing over IV sites when the IV device is secured through adhesive tape or sutures.

## Warning

Do not use as primary fixation of life supporting devices.

## Precautions

- Do not use Mepitel Film on patient or user with known hypersensitivity to the materials or components of the product.
- Do not reuse. If reused, performance of the product may deteriorate and may cause cross contamination.
- Mepitel Film is not a wound contact layer product, and does not allow the wound exudate to pass through the dressing to a secondary layer.

- If more than one product is used and they are overlapped, the vapour permeability goes down; which can lead to excess moisture not being allowed to pass away from the skin.
- Do not use for primary fixation for IV cannula, port or other devices.
- Sterile. Do not use, if inner package (sterile barrier) is damaged or opened prior to use. Do not re-sterilise.
- If you see any signs of infection, such as fever, or when the wound or surrounding skin becomes red, warm or swollen, consult a healthcare professional for appropriate treatment.

### **Instructions for use**

Mepitel Film can be used by lay persons under supervision of healthcare professionals.

1. Cleanse the skin in accordance with clinical practice.
2. Dry the skin thoroughly.
3. Remove the protection film (printed) and apply the adherent side to the fragile skin (or) primary dressing and smooth out Mepitel Film in place, ensuring a good seal. Do not stretch during application.
4. Remove the paper frame.
5. For best result, Mepitel Film should overlap the dry surrounding skin by at least 1–2 cm for the smaller sizes (sizes up to 10×12 cm) and 5 cm for the larger sizes in order to fixate the dressing securely. If the product edges start to curl, gently cut off these edges to increase the wear time.

Mepitel Film should be changed when it starts to come off, if damaged or as indicated by clinical practice.

Disposal should be handled according to local environmental procedures.

### **Other information**

If any serious incident has occurred in relation to the use of Mepitel Film it should be reported to Mölnlycke Health Care and to your local competent authority.

Mepitel® and Safetac® are registered trademark of Mölnlycke Health Care AB.

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