

# Tubifast® garments with TwoWay Stretch® technology

Ready-to-wear garments

SoftSeam technology®  
helps to reduce irritation  
of sensitive skin<sup>1</sup>



- Convenient, easy way to wet or dry wrap<sup>2-5</sup> with no requirement for pins, tapes, or ties<sup>1</sup>
- Sizes available for children aged 6 months to 14 years<sup>2-4</sup>
- Complete freedom of movement<sup>1</sup> through the two-way stretch construction<sup>1</sup>
- Comfortable to wear under nightwear and ordinary clothes<sup>1</sup>

Tubifast® garments with TwoWay Stretch® technology are a range of ready-to-wear<sup>1</sup> garments for use in wet or dry wrapping<sup>3-5</sup> and dressing retention<sup>5</sup>.

## How Tubifast® garments with TwoWay Stretch® technology work

Ready-made<sup>1</sup> Tubifast® garments with TwoWay Stretch® technology, for dressing retention<sup>5</sup> and wet or dry wrapping<sup>2-5</sup>, are available in full-sleeved vests, vests with integrated mittens (6-24 months only), tights, leggings, socks, gloves and Patch Wrap®. For children six months and over.

With no tapes, pins or ties<sup>2</sup>, Tubifast garments with TwoWay Stretch technology are easy to apply<sup>5</sup>, and their two-way stretch construction<sup>1</sup> ensures complete freedom of movement. Comfortable to wear under nightwear and ordinary clothes<sup>1</sup>, they also help to promote fully active days and restful nights.

## Benefits of Tubifast Garments with TwoWay Stretch technology

- Flat seams help prevent irritation of sensitive skin<sup>1</sup>
- Convenient way to wrap<sup>2-5</sup>
- Comfortable to wear under nightwear and ordinary clothes<sup>1</sup>
- Freedom of movement<sup>1</sup>
- Available in full-sleeved vests, vests with integrated mittens, tights, leggings, socks, gloves and Patch Wrap

## Material properties

Viscose, Elastane and Polyamide

## Washing instructions



**IMPORTANT INFORMATION:** Please note: It is recommended that Tubifast garments with TwoWay Stretch technology are sized according to dimensions as well as age. If in any doubt regarding the selection of size, the next largest size should be chosen (e.g. if the child is wearing a nappy with the Tubifast Garments with TwoWay Stretch technology, please allow for extra sizing).

References: 1. Mölnlycke Health Care. Data on file. 2. Choi J-W, Kang M-J, Lew B-L, Sim W-Y. Efficacy and utility of wet wrap dressing for patients with pediatric atopic dermatitis. Journal of the American Academy of Dermatology. 2018;79(3):AB122. 3. He H, Koh MJA, Lee HY, Ang SB. Pilot study of a customized nanotextile wet garment treatment on moderate and severe atopic dermatitis: A randomized clinical trial. Pediatric Dermatology. 2019. 4. Hon KLE, Wong KY, Cheung LK, Ha G, Lam MCA, Leung TF, et al. Efficacy and problems associated with using a wet-wrap garment for children with severe atopic dermatitis. Journal of Dermatological Treatment. 2007;18(5):301-5. 5. Cross-sectional observational study report Epidermolysis Bullosa. PD-566181 Rev 01. 2019.

## Areas of use

Wet wrapping is a therapy used in the treatment of atopic eczema, and involves the use of an emollient under retention bandaging. This can be done using the emollient Epaderm® Ointment together with Tubifast garments with TwoWay Stretch technology or Tubifast® TwoWay Stretch bandages. An emollient is applied to the affected skin, which is then covered with a damp Tubifast garment with TwoWay Stretch technology or bandage. Working together, the emollient and the damp layer keep the skin hydrated and moist, and help reduce inflammation.

The moistened Tubifast garment with TwoWay Stretch technology or bandage also acts as a retaining layer, ensuring that the emollient stays in contact with the skin. This layer also cools and soothes the skin, helping reduce the itch. A second, dry layer is then applied, which keeps the patient's clothing dry; preventing further scratching and potential injury or aggravation of the skin condition.

Dry wrapping is a similar process to wet wrapping. An emollient is applied to the affected skin, which is then covered with a single, dry layer of Tubifast garment with TwoWay Stretch technology or bandage.

## Tubifast garments with TwoWay Stretch technology Ordering Information

	Age	Art. No
Vests with integrated mittens	6-24 months	992007
Vests	2-5 years	992008
	5-8 years	992009
	8-11 years	992010
	11-14 years	992011
Tights	6-24 months	992012
Leggings	2-5 years	992013
	5-8 years	992014
	8-11 years	992015
	11-14 years	992016
Socks	1 size: 2-14 years	992017
Gloves	Child XS	5922
	Child Small	5923
	Child Medium-Large/ Adult Small	5921
	Adult Medium-Large	5920
Patch Wrap		
Size	Approx. Limb Circumference*	Art. No
Small	14-18cm	5924-00
Medium	19-24cm	5925-00
Large	25-31cm	5926-00
X Large	992010	5927-00

\*Measure at joint

Find out more at [www.molnlycke.com](http://www.molnlycke.com)

Mölnlycke Health Care AB, P.O. Box 13080, Gamlestadsvägen 3 C, SE-402 52 Göteborg, Sweden. Phone + 46 31 722 30 00. The Mölnlycke Tubifast, SoftSeam Technology, Patch Wrap and Epaderm trademarks, names and logos are registered globally to one or more of the Mölnlycke Health Care group of companies. © 2020 Mölnlycke Health Care AB. All rights reserved. HQIM000404

