VELPEAU®

VP0306,VP0306B
Arm Sling

FDA

CE

ISO
ISO9001

ISO
ISO13485

CONTENTS

WEAR STEPS	01-10
 ENGLISH	11-13
 FRANÇAIS	14-16
 DEUTSCH	17-19
 ESPAÑOL	20-22
 ITALIANO	23-25
 PORTUGUÊS	26-28

VP0306

EN Comfort Version
FR Version Confort
DE Komfort-Version

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ENGLISH

Step 1: Hang the product on your shoulder (the back strap ⑤ is behind you at this time, ignore it for now).

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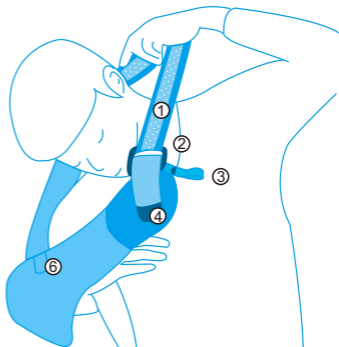
**ENGLISH**

Step2: Place your forearm into the arm pouch.

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ENGLISH

At this point, you can pull up from where the tag is sewn in and put your arm completely in like this.

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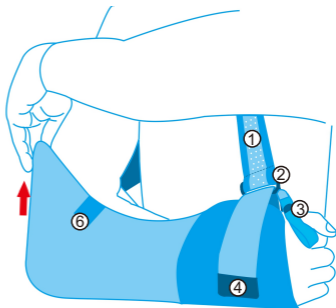
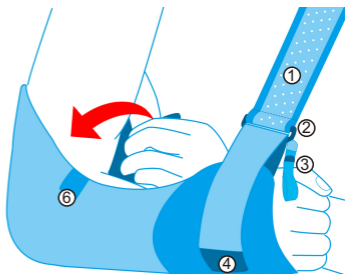
Passo 3: Fixar a Correia de braço⑥.

ENGLISH

Step 4: You can adjust the Velcro strap ④ to make sure your arm is at the most comfortable angle.

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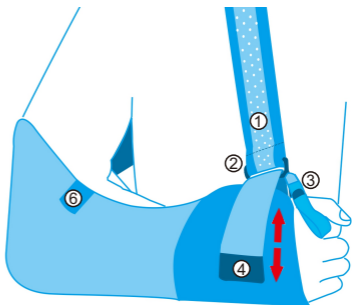
ENGLISH

In the diagram above, the Velcro strap ④ is attached on the outside of the body. If you find that the Velcro strap ④ is on the inside of your body after wearing it, you should adjust it on the inside. Or, you can pass the Velcro strap ④ through the Plastic buckle ② in the opposite direction and secure it on the outside.

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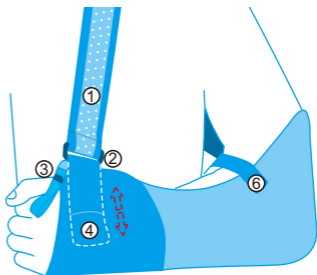
ENGLISH

Step 5: Hang the thumb loop ③ around the thumb and attach it to the product so that the thumb can hang on it.

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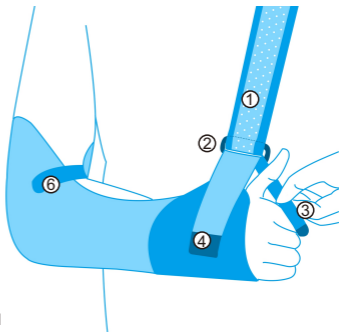
ENGLISH

Step 6: Pull the back strap ⑤ over waist to front and secure it ⑦ on the outside of your wrists to prevent arm swinging.

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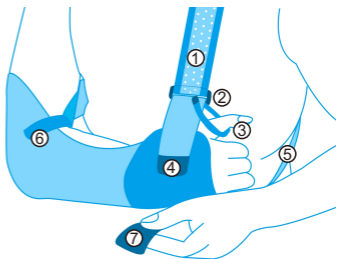
[PRODUCT NAME] : Arm Sling (Shoulder Immobilizer)

[PRODUCT CODE] :

[Comfort Version]

VP0306-S VP0306-M VP0306-L VP0306-XL VP0306-XXL

[Breathable Version]

VP0306B-S VP0306B-M VP0306B-L VP0306B-XL VP0306B-XXL

[PRODUCT SPECIFICATIONS] :

Size	Product Code	Version	Colour	Underbust (inch/cm)	Left/Right
S	VP0306-S	Comfort	Black	25 1/2"-31 1/2"/ 65-80cm	Ambidextrous
	VP0306B-S	Breathable	Blue		
M	VP0306-M	Comfort	Black	29 1/2"-37 1/2"/ 75-95cm	
	VP0306B-M	Breathable	Blue		
L	VP0306-L	Comfort	Black	35 1/2"-43 1/4"/ 90-110cm	
	VP0306B-L	Breathable	Blue		
XL	VP0306-XL	Comfort	Black	41 1/4"-49 1/4"/ 105-125cm	
	VP0306B-XL	Breathable	Blue		
XXL	VP0306-XXL	Comfort	Black	47 1/4"-55"/ 120-140cm	
	VP0306B-XXL	Breathable	Blue		

[FIBER CONTENT] :

Comfort Version:

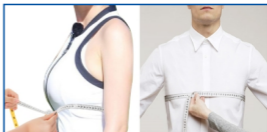
Polyester, Cotton, Nylon, Polypropylene, Polyurethane.

Breathable Version:

Lycell. Polyester, Nylon, Polypropylene.

[HOW TO CHOOSE THE RIGHT SIZE ?] :

Please measure your under-bust measurement to choose the right size. If the product is on the outside of the clothes, please measure your under-bust size.

**[PRODUCT PERFORMANCE] :**

- **Comfortable Version:** The whole product is soft and comfortable, filled with foam. Perforated design, warm and breathable, better suited for sleeping and winter.
 - **Breathable Version:** Lycell mesh fabric is soft and skin-friendly, and the 3D inner core is multi-dimensionally breathable, more suitable for hot summer.
 - Can be put on and taken off by oneself. (It depends on the specific condition)
 - Quick and convenient compression adjustment.
 - No strangling, no cutting off the circulation.
 - Perforated fabric allows good ventilation.
 - Open design allows you to wear it over clothing.
 - Built-in thumb loops support your hand to prevent fatigue.
 - Product contains no neoprene or latex, no skin irritation.
 - Fits on the left or right arm. (ambidextrous).
-

[INDICATIONS] :

- Acute shoulder instability.
 - Elbow injuries.
 - Muscle and/or tendon pain injury, shoulder disorders.
 - Immobilisation of the Shoulder & Elbow.
 - Trauma to the Shoulder & Elbow.
 - Postoperative care of rotator cuff surgery.
 - Distortion, contusion, luxation of the shoulder.
-

[INSTRUCTIONS] :

- Please use under the guidance of doctors and professional persons.
- Select the appropriate size and refer to the wear step diagram in the manual.
- Check the strap in case it is too tight or too loose, to ensure stability after wearing. If it is too tight, it will affect blood flow, which is not conducive to wound healing, or if it is too loose, it cannot be fixed.
- Easy to mount and dismount, just put on or take off the Velcro.

[PRECAUTIONS / WARNINGS] :

- To be used by adults and adolescents.
- Easy to use, it can be used with the help of medical personnel or healthy people when you have difficulty on your own.
- Place the fixed part as priority. If discomfort occurs in using process, please adjust the tightening.
- Wash it with water before or after using, store it in a ventilated dry place at normal temperature, and it's reusable.
- This product is for isolated use, with no other combination of product using, and its safety and effectiveness will not be influenced by other external conditions while using.
- No specific special skills, knowledge or training required for users, it does not contain materials with allergy or side effects to human body, which is in accordance with ROHS standard.
- If there is an open wound on the skin, use the product carefully or bandage the wound to avoid direct contact with it.

[TRANSPORTATION AND STORAGE CONDITIONS] :

It is allowed to be transported by general means of transportation and stored at normal temperature, but it must be fireproof, moisture-proof, dustproof, rodent proof, and insect proof.

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