

VELPEAU®



Ankle
Compression
Sleeve
VP1505



VP1505
Black



VP1505B
Khaki



[Product Name] : Ankle Compression Sleeve

[Product Code] :


☐ VP1505-Reg ☐ VP1505-XL
☐ VP1505B-Reg ☐ VP1505B-XL

[Product Specifications] :

Size	Product code	Color	Left/Right	Ankle Circumference
Reg	VP1505-Reg	Black	Fits Both Feet	6.3"-8.0"
	VP1505B-Reg	Khaki		
XL	VP1505-XL	Black	Fits Both Feet	8.0"-9.3"
	VP1505B-XL	Khaki		

How To Choose The Right Size?

Please choose the appropriate size based on the measurement above the ankle bone. Measure the circumference at 0.8 inch above the ankle bone.

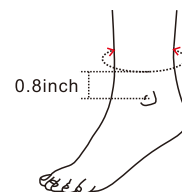

[Product Compounds] :

Polyamide, Spandex, Nylon, Filament, Polyester, Low Melt Yarn, Microfiber leather, etc.

[Application] :


Helps provide reliable support to stiff, weak or injured ankles.

www.velpeau.com

[Product Features] :

- The ankle sleeve is crafted with seamless technology for a durable and smooth texture to ensure long-lasting wear without irritating the skin.
- The single strap Figure 8 wrapping method provides superior comfort, support, and a precise fit for maximum comfort.
- Provides enhanced sweat-absorbing, breathable, and quick-drying attributes compared to neoprene, due to its thin material with light support.
- It exhibits high elasticity, maintaining its shape even with prolonged use.
- Comfort panel reduces bunching behind the heel
- Fit left or right ankle.
- Easily fits inside shoe.


[Product Display] :

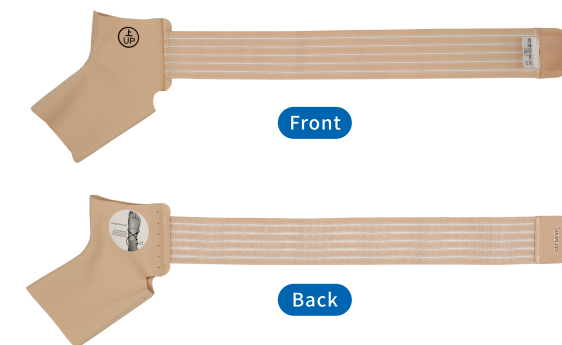

Front

Back

www.velpeau.com

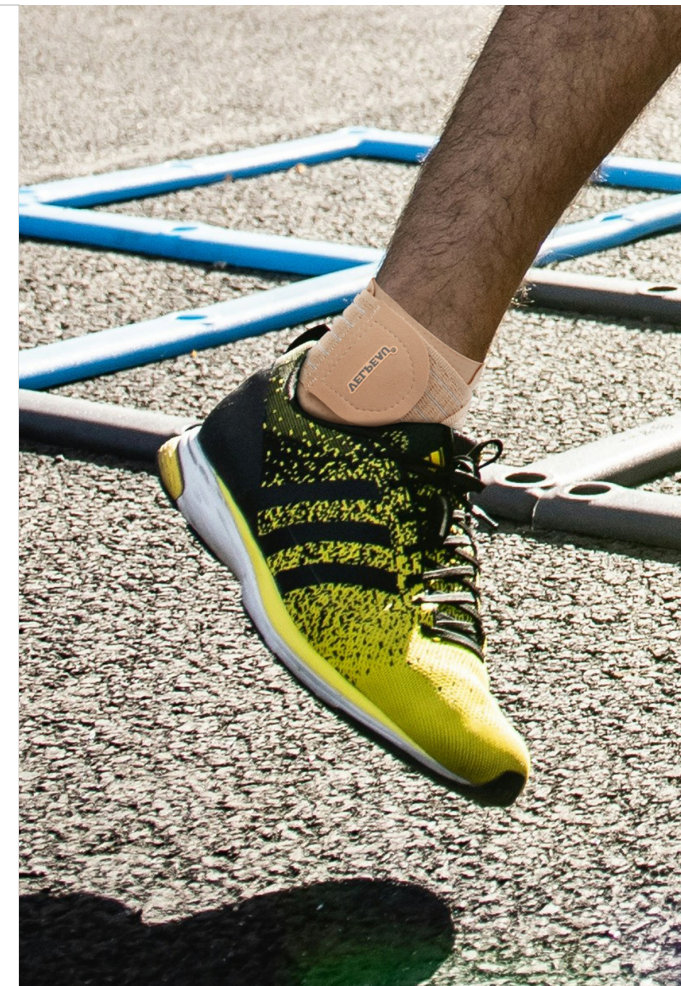

VELPEAU®



Shenzhen Weprotex Technology Product Co.,Ltd
Add: 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China
Tel: 0755-89505533 E-mail: sales88@velpeau.com
www.velpeau.com

EC REP: No. 38, Boulevard Renard Benoit, 78680 Epone, France



[How to wear] :

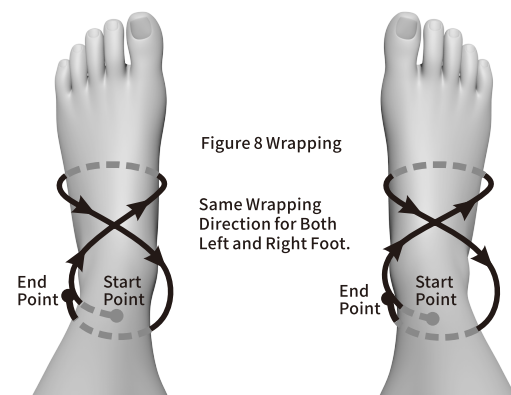


Figure 8 Wrapping

Same Wrapping
Direction for Both
Left and Right Foot.



Before putting on the ankle sleeve, ensure that the end marked with the word "UP" faces upwards, and align the opening with the back of the heel.



Scan the QR code
for detailed product information.

www.velpeau.com



Apply the Ankle Sleeve onto the healing foot.

The strap should start from the back of the foot towards the left side.

Wrap the strap around the ankle from the left side, then pass through the sole from the right side.

Then wrap the strap around the ankle from the left side to the right ankle.

Lastly, wrap the strap around the back of the ankle, and secure the hook & loop fastener onto the strap on the left side of the ankle to complete the wearing process.

www.velpeau.com

[Warning] :

- Avoid using this product if you have an open wound, eczema, or any lumps or bumps on the affected area.
- Discontinue use immediately if you experience any discomfort, such as tightness or numbness, while wearing the support.
- Clean the product with water before and after use, and store it in a well-ventilated, dry area at room temperature. It can be reused multiple times.

[Instructions] :


- Proper sizing can be achieved by following the wear step diagram in the manual or watching the detailed wearing video on the official website www.velpeau.com.
- It is important to check the strap tightness to ensure stability. Avoid making it too tight, which can restrict blood flow and hinder wound healing, or loose, which may result in inadequate support.
- The product is designed for easy assembly and disassembly, requiring simple attachment or detachment of the hook and loop fasteners.

[Transportation and Storage Conditions] :

The product can be transported using general means of transportation and stored at room temperature. However, it must be fireproof, moisture-proof, dustproof, rodent-proof, and insect-proof.

www.velpeau.com

[Washing] :

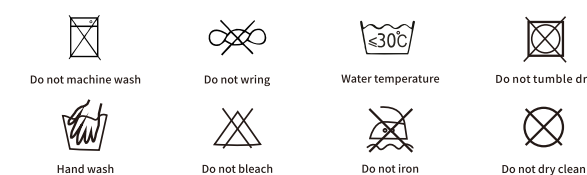

[Term of Validity] : 5 years

[Production Date] : See the package

[Manufacturer] : Shenzhen Weprotex Technology Product Co., Ltd.

[Production Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[After-sale Service] : Shenzhen Weprotex Technology Product Co., Ltd.

[After-sale Service Address] : 1710, Building 11, Phase II of Tian'an Yungu Industrial Park, Gangtou Community, Bantian Street, Longgang District, Shenzhen, China

[Postcode] : 518129

[Tel] : +86 - 755-89505533

[E-mail] : sales88@velpeau.com

[Website] : www.velpeau.com

[Date of Preparation] : Apr.3,2024

[Modification Date] : Apr.7,2025

www.velpeau.com