



The Scooter Accessories

Assembly instructions for the "TSARC"

"Retractable Canopy"

Assembly Tools Required

Please use the specific tools for assembly of the Retractable Canopy

(2) 5MM Allen Wrenches (1) 6MM Allen Wrench (1) Phillips Head Screwdriver



Step 1.

Please check to ensure all hardware is present for the installation



Step 1.2

Please ensure there is no damage and all additional frame components are provided.



Step 1.2 Continued

Additional Parts....



Assembly Step 1.

Start by attaching the Canopy to Hitch Bar using (1) Clevis Pin and (1) Screw Knob. (fig. 1)

Then attach the Rear Canopy Support Frame, using the 3rd hole from the bottom. (fig. 2)

Next you will Install the Headrest Support and "S" Bar to the top of the Canopy Support Frame. (fig.3)

(If you do not have a seat with a headrest, you are not required to install the Headrest support and "S" bar).

(fig. 1) (fig. 2) (fig.3)









Assembly Step 2.

Start by attaching the Rear Canopy Cover to the Rear Canopy Support Frame using the (8) Long M8X16 Female bolts along with the (8) Short M8X30 Screws Male.

(Please be sure to notice that the zipper will face forward when installing)

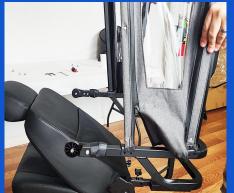

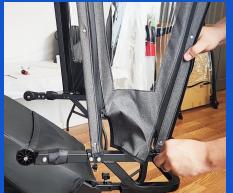





Assembly Step 3.

"Clear Retractable Cover Assembly"


In this Step you will insert the rectangle bar into the "Y" bracket. Next you will insert the (4) Short M8X16 Female and tighten together with the remaining (4) Short M8X30 Screws Male






Assembly Step 4.

Join the Clear Retractable Cover onto the Rear Canopy Cover by first removing the M10x1.25x40 bolt and nut then reinsert the M10x1.25x40 bolt and nut and securely fasten together.

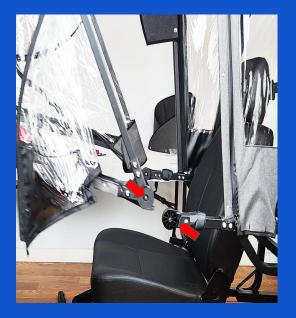

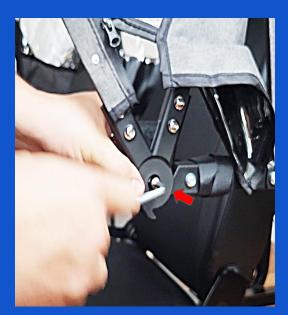



Assembly Step 5.

This next step provides Sunshade Effect when the Clear Retractable Canopy is not installed.

Simply install the Sunshade Gray Visor using the (8) Short M8X16 Screws using your "P2 bit Phillips Head Screwdriver"





Assembly Step 6.

This is the final step in completing and installing the Retractable Canopy to your scooter.

(Before using, ensure that all hardware has been tightened and securely strap the Canopy to your armrests to avoid any side to side movement when operating).





AutoMobility Store

Thank you for choosing AutoMobility Store. We greatly appreciate your business.

Please visit us at www.automobilitystore.com for all of your favorite accessories and mobility equipment.

For an Installation & Assembly video, follow this link to our YouTube channel: https://youtu.be/51hcn2JJdqM