Houdini™

Houdini Instructions

Thank you for purchasing the Houdini CPAP Hose Support System.

- 1 Place Houdini base on a flat surface.
- 2 Insert large thread end of the Telescopic Pole into Houdini base and turn clockwise until tight.
- 3 Expand the Telescopic Pole to full height by grasping the solid metal ring (not the spiral ring) at the top of the pole and pull straight up. Continue to expand all the sections; you may need to grasp the bottom portion of the telescopic pole near the base and pull up once more to ensure it is fully expanded. When the Houdini is at full height, there is 1 flexible and 4 solid portions, and will measure 38.25".
- 4 With Houdini now assembled, insert the base of the Houdini between the box spring and mattress. This can be inserted on the side or top of the bed, depending on your preference.
To collapse the Houdini, straighten the flexible portion at the top of the pole and carefully push back into the telescopic portion of the pole. Once this piece is inserted, keep pushing down from the top until the rest of the pole is back to its original size.

To see more CPAPology products, please visit, www.cpapology.com

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 **CPAPology™**

www.cpapology.com

CPAPology is a brand of KEGO Corporation

Tel: 833.817.7027

Parts/Pièces/Partes