

AAOCARE.COM
22732 Granite Way
Suite B
Laguna hills CA 92653
support@aaocare.com

Register your device today at aaocare.com
Receive our one year extended warranty.

Warranty will cover manufacturing defects and
we will repair or replace your device at no cost.



INFRARED AND LED FACIAL LIGHT THERAPY WITH STEAMER

NANO-PRO

Red Light: 640nm

Blue Light: 423nm

Yellow Light: 583nm

Infrared Light: 810nm (Light is not visible to the naked eye)

***THE SYSTEM WILL BEEP CONSTANTLY
WHEN LOW ON WATER***

INTENDED USE

LED and infrared light therapy can dramatically revitalize your skin without any downtime or discomfort. Our innovative aesthetic treatment helps counteract years of sun damage, environmental damage, aging, acne and acne scarring. Lumalux Plus provides noticeable results, often in a couple of treatments.

Red Light: 640nm

Skin penetration depth: 1-4mm

Red Light works from the inside-out to enhance mitochondrial function in cells. It decreases skin inflammation, smooths the skin tone, repairs sun damage, fades scars, reduces stretch marks and builds collagen in the skin, reducing wrinkles. It will also treat dermatitis or ringworm.

Blue Light: 423nm

Skin penetration depth: 0.8-1.2mm

Blue Light has an antimicrobial effect, making it effective at killing several types of bacteria that can collect in your pores and oil glands causing breakouts. It also reduces sebum production and minimizes pores.

Yellow Light: 583nm

Skin penetration depth: 1-2mm

Yellow light is used for inflammation reduction, stimulates lymphocyte production, activates toxin elimination, helps to heal psoriasis and eczema, and reduces redness from rosacea.

Infrared Light: 810nm (Light spot is Invisible)

Skin penetration depth: 4-6mm

Infrared light is similar to red light, however, infrared light is invisible to the naked eye and penetrates much deeper into the skin. It improves the circulation of oxygen-rich blood in the body, promoting faster healing of deep tissues, relieving pain.

Facial steaming

Facial steaming can help hydrate and soften the outer skin layer, allowing for the it to more easily absorb skin care products and to help counteract signs of aging.

IMPORTANT SAFETY INSTRUCTION

Read all instructions before using this device; save these instructions for future reference.

This manual contains important safety symbols and instructions. Pay attention to these symbols and follow all instructions provided.



This symbol indicates instructions for use; consult accompanying documents.



General Warning or Caution

This symbol indicates personal injury or damage to the equipment is possible.



Electric Shock Symbol

This symbol indicates a voltage hazard; any mishandling could result in irreparable damage to the equipment, personal injury or death.



Keep Dry-Symbol

This symbol indicates the device cannot be used outdoors or where it can become wet.

WARNINGS AND PRECAUTIONS

Note: When making changes to your health and wellness routines, it may be advisable to consult with your physician before proceeding.



Do not use if the patient is pregnant or breastfeeding.



Do not use on infants or children.



Do not use if the patient or administrator has epilepsy or a history of seizures.



Avoid using when the patient is taking cortisone injections or any other kind of steroid injections.



Avoid use if the patient is using photosensitive drugs.

PRODUCT WARNING AND SAFETY INFORMATION

	<p>Warning Do not attempt to operate this device if there is evidence of shipping damage or you suspect the unit is damaged. Contact AAO Care before attempting to plug in and operate damaged equipment.</p>
	<p>Warning Only authorized personnel may service this equipment. Contact www.aocare.com for servicing or repair.</p>
 	<p>Do not lie on or lean against the LED Device. The device is not designed to withstand full body weight or extreme pressure. Do not place the LED Light under clothing, blankets, sheets or towels.</p> <p>Failure to follow this precaution could result in shock!</p>
	<p>Warning Avoid looking directly into the light and wear the goggles provided or use appropriate eye protection during treatment.</p>

WARNINGS AND PRODUCT SAFETY INFORMATION CONTINUED

1. Follow instructions and heed all warnings. Install in accordance with the manufacturer's instructions.
2. Make sure that the power supply is unplugged from wall outlet before cleaning the apparatus.
3. Keep dry. Do not use this device near water.
4. Clean with a soft dry cloth only. Do not use any cleaning solvents as they could damage the device.
5. If the housing or control surface is dirty, use a soft cloth dampened with a non-alcohol and non-chlorine disinfectant to clean the device. Wring out the cloth before cleaning to prevent any fluid from penetrating into the housing unit. Wipe the device with a clean, dry cloth after cleaning.
6. Do not install near any heat source such as radiators, stoves or other apparatuses that produce heat.
7. Only use attachments/accessories specified in this manual.
8. There are no user-serviceable parts inside this device. Refer all servicing to the manufacturer, AAO Care.
9. Service is required when the device has been damaged in any way, such as power cord damage, water damage or if objects have fallen on to the device or if it is dropped, causing it to malfunction.
10. Store in a dry, non-humid location out of direct sunlight.

PROTOCOLS FOR USE

When should I use the Blue light?

Use NANO-PRO blue light to treat an acne breakout. The treatment consists of 12 sessions over 4 weeks, of 3-5 sessions per week.

When Should I use the Red light?


Use NANO-PRO red light to treat wrinkles. A treatment consists of 16 sessions over 4 weeks, of 4-6 sessions per week.

When Should I use Yellow light?


Use the NANO-PRO yellow light to treat skin erythema, and redness. A treatment consists of 18 sessions over 6 weeks, of 3-5 sessions per week.

DEVICE ASSEMBLY


1. Place the 3 panels on the table and connect plugs on both sides.

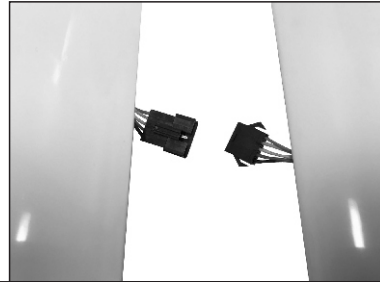


2. Insert the metal tab to side panel.
Repeat on another side.

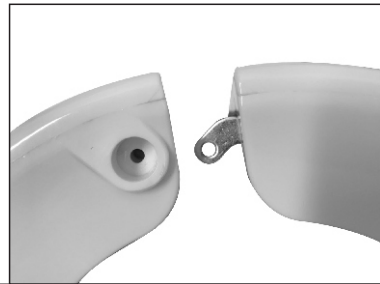


3. Insert screw in the side panels.

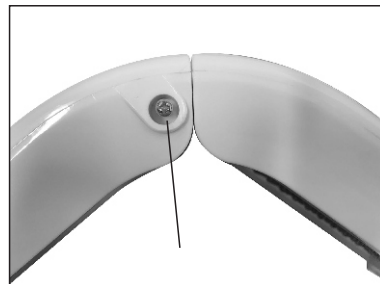

4. Plug in the adapter to the top panel,
then connect to an electrical outlet.

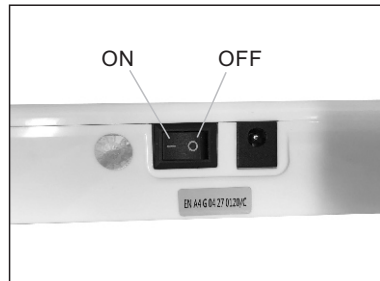

NANO-PRO OPERATION PANEL

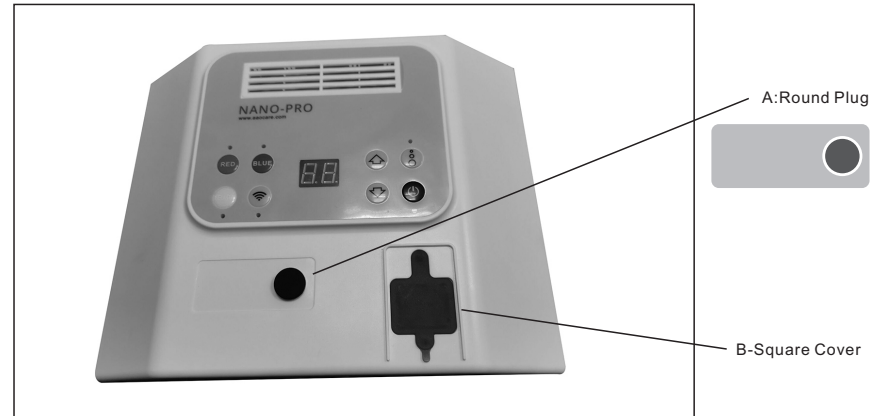

WARNING: This machine must have water to correctly function.

Steps to fill the water reservoir:

- 1) Remove both the rounded plug and square covering (figure A and B).
- 2) Fill water into the exposed square opening. The tank volume is 8 Oz.
A full tank will last 1.5 hours.
- 3) Reinsert both the rounded plug and square covering.

***THE SYSTEM WILL BEEP CONSTANTLY WHEN LOW ON WATER!
YOU CAN FILL WITH COOL OR WARM WATER!***

2. Control panel



WARNING: THIS MACHINE MUST HAVE WATER TO FUNCTION CORRECTLY

You can select any color and infrared in combination with each other. Press desired color button or infrared button. Use the UP and DOWN button to adjust the timer to your desired length, then press the ON and OFF button. If you do not want to adjust the timer, the default is 20 minutes. You can press the steamer button to start or stop at any time. The steamer does not have a timer.

USEFUL TIPS AND FREQUENTLY ASKED QUESTIONS

Q: Does the NANO-PRO system emit UV light?

A: No. UV light has a wavelength less than 400nm and can damage the skin. The Nano-Pro uses safe wavelengths between 423nm (blue light) and 810nm (Infrared light)

Q: Can I use multiple colors and the infrared light in a combination?

A: Yes.

Q: How long should a treatment be using the NANO-PRO system?

A: For your first treatment, it is recommended to start with 20 minutes. As you get comfortable with the light, you can increase treatment times to 30 minutes per day. Infrared light treatments can be combined with each of the 3 light sources.

Q: Are there any special preparations required before using the NANO-PRO system?

A: NANO-PRO system should be used in close contact with the skin. Treatment area should be free of clothing, make-up and anything else that may deflect light away from the skin. Do not look directly at the light and wear the eye protection goggles included.

Q: Why can't I see the Infrared Light?

A: Infrared light is invisible to the eye. Although you will not be able to see infrared light it is the deepest penetrating light source.

Q: How long do I need to wait before applying makeup after treatment?

A: There is no downtime needed after using the NANO-PRO. However, if you are combining it with other treatments such as chemical peels, microneedling or microdermabrasion etc., follow your skin care professional's advice.

Q: Can I use the machine without water if I don't want to use the steamer function?

A: No, the machine will beep when low on water. You may choose to only use the LEDs, but water must still be present in the machine.