




HO DENTAL COMPANY

Usage Instructions

The Sandtrap Intra-Oral Sandblasting Device

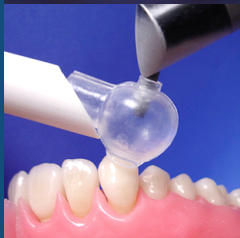

The Sandtrap is a patented, clear-plastic tool that confines and suctions away aluminum oxide particulates during intra-oral or chairside sandblasting.

It is designed to fit standard high-volume micro-etching suction tips. Insert the micro-etcher tip into the side hole of The Sandtrap. The Sandtrap should fit snugly.

Note: The Sandtrap can be trimmed at the base for better adaptation, angulation, or simply to bring the micro-etcher tip closer to the surface to be sandblasted.

HoDentalCompany.com

ph: (702) 430-9847

fax: (702) 430-8112