PEPE MOBILITY

1) COMPONENTES / COMPONENTS / KOMPONENTEN / COMPOSANTS / COMPONENTI

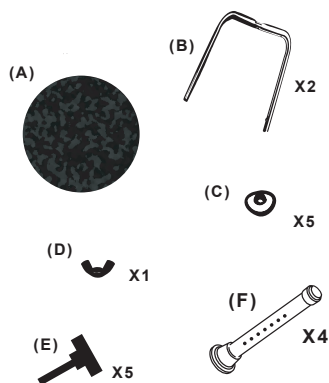
FIG. 1

2) MONTAJE / ASSEMBLY / VERSAMMLUNG / MONTAGE / MONTAGGIO

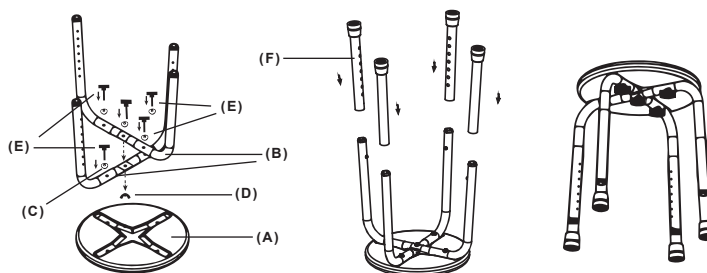
FIG. 2

EN

MANUAL

PEPE-STOOL 1

P30001

STOOL

Manual provided for the user of the product. BEFORE using this product, read this manual and keep it.

INDICATIONS OF USE

The PEPE Round Shower Stool should be used as a sitting aid in the bathroom or shower for people with sufficient torso stability. Its capacity is 220 lb.

FEATURES

- Mounting without the use of tools
- Quick and easy adjustment from 16.15" to 22" high in 7 positions.
- Ergonomic and comfortable seating surface.
- Safe and stable thanks to its sloping tips.
- Free of corrosion thanks to its aluminum chassis.
- Weight: 3 lb.

ASSEMBLY (Fig. 2)

1. Put the two U-shaped structures (B) in an X-shape and screwing from the center with screw (E), washer (C) and thumbscrew (D).
2. Turn the seat (A) over and screw the front frame onto it, with the remaining four screws (E). Use the washers (C) with the curved part towards the tube.
3. Put the legs (F) one by one and always at the same height, fitting the holes until you hear a click of the safety system of the structure tubes (B).

⚠ WARNINGS!

- Make sure all four legs are adjusted to the same height.
- Make sure the spring buttons click into the holes in the legs.
- Do not stand on this product or tilt it.
- Do not use it as a climbing aid.
- Do not use for side transfers or part load.
- Do not sit on it unless all four legs are fully in contact with the ground.
- Do not use outdoors.
- Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with products available on the market. Always keep the product clean and wash it regularly.

1. Remove the legs.
2. Use water and a mild non-abrasive detergent to clean the separated parts.
3. Disinfect by cleaning all accessible surfaces with disinfectant.

CERTIFICATION

This product has the corresponding CE marked, in compliance with the Regulation (EU) 2017/745 on medical devices.

WARRANTY INFORMATION

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

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