PEPE PEPE MOBILITY

1) COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI

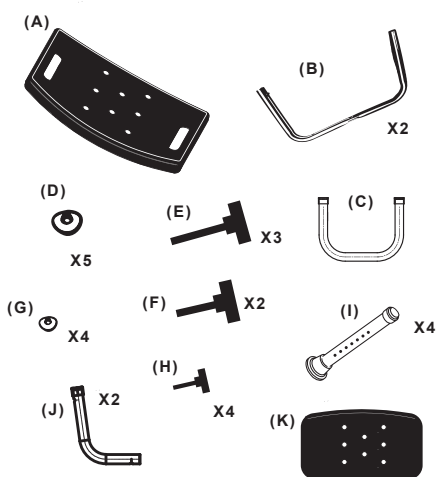
FIG. 1

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

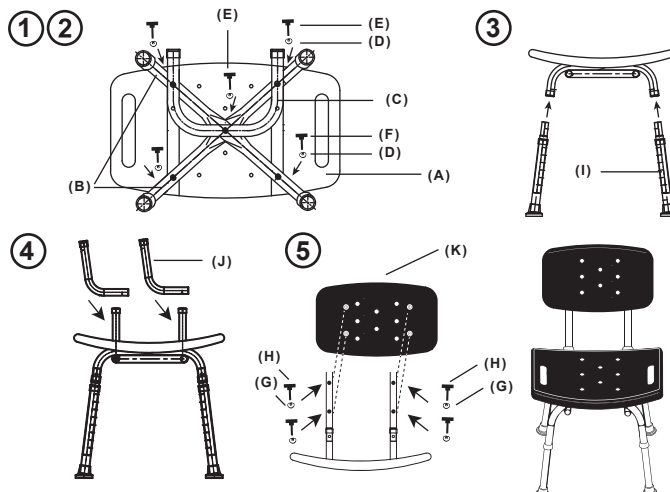
FIG. 2

EN

MANUAL

PEPE-CHAIR

P30003

CHAIR

Manual provided for the user of the product. BEFORE using this product, read this manual and keep it.

INDICATIONS OF USE

The PEPE-CHAIR should be used as a sitting aid in the bathroom or in the shower for people with sufficient torso stability. Its capacity is 220 lb.

FEATURES

- Mounting without the use of tools
- Quick and easy adjustment from 4.15" to 20" high in 7 positions.
- Ergonomic and comfortable seating surface.
- Safe and stable thanks to its sloping tips.
- Free of corrosion thanks to its aluminum chassis.
- Weight: 6 lb.

ASSEMBLY (Fig. 2)

1. Turn the seat (A) over and position the two U-shaped frames (B) by snapping them into an X shape.
2. Once fitted, add the U-shaped base frame of the backrest (C) and using the washers (D) with the curved part towards the tube, tighten the 3 longer screw wrenches (E). Do the same for the remaining two legs with the 2 medium size wrenches (F) and washers (E). In this way the structures (B) (C) will be secured and we will have the seat (A) assembled without legs.
3. Position the legs (I) one by one and always at the same height, fitting the holes until you hear a click in the tubes of the frame (B).
4. Take the upper L-shaped back frames (J) and position them by fitting the holes until you hear a click in the tubes of (C).
5. Use the small washers (G) with the curved part toward the backrest upper frame tube (J) and tighten the remaining 4 screw wrenches (H) to the backrest (K).

⚠ WARNINGS!

- Make sure all four legs are adjusted to the same height.
- Make sure the spring buttons click into the holes in the legs.
- Do not stand on this product or tilt it.
- Do not use it as a climbing aid.
- Do not use for side transfers or part load.
- Do not sit on it unless all four legs are fully in contact with the ground.
- Do not use outdoors.
- Any serious incident related to the product should be reported to the manufacturer and the competent authority of the Member State in which they are user and patient set.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with products available on the market. Always keep the product clean and wash it regularly.

1. Remove the legs.
2. Use water and a mild non-abrasive detergent to clean the separated parts.
3. Disinfect by cleaning all accessible surfaces with disinfectant.

CERTIFICATION

This product bears the appropriate CE marking in compliance with Regulation (EU) 2017/745 on medical devices.

WARRANTY INFORMATION

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

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