CODE	
PEPE - BATH RAIL WHITE 12"	P30006
PEPE - BATH RAIL BLACK 12"	P30007

The PEPE - BATH RAIL is perfect for people with difficulty getting up, risk of falling or with reduced mobility, providing help and security. Don't use this product without reading and understanding the instructions. If necessary, contact a healthcare professional or technical personnel to avoid possible injury.

ASSEMBLY

The box contains screws (6), dowells (6) and rubber gaskets (2), to be installed on a wall. Examine the bath rail kit. Take everything out of the box and check that the bath rail is in good condition, as well as all the parts.

1. Set the installation location. In general, the bath rail has to be at the waist of the user.
2. Mark mounting holes using a pencil through the holes of each flange.
3. Use a drill bit to drill holes and insert the screws in to the dowells for fixing.
4. Fix the screws through the flange holes positioning the bath rail on the wall and tight the 6 screws in the holes.
5. Pull the bar hard to test its supporting.
6. You can apply a little silicone caulk behind and around the decks to prevent water from seeping behind the walls.

CARE AND CLEANING

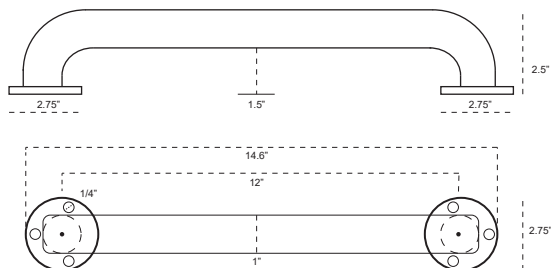
- Cleaning with a microfiber cloth with hot soapy water.
- For the most stubborn stains you can use a sponge, but it is essential not to use scourers, bleaches or abrasive products.

GENERAL WARNINGS

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. **Don't use the product if it is damaged.**
- The bath rail supports 250 lb.
- We recommend using only one user at a time.
- Use the screws that come with the bath rail, check it monthly and retighten if necessary.
- Use the device for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet lift, or for injuries or damage caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

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