EN Insert the joint into the hole and slide it until it completely covers the bandage, as shown in steps 1 and 2. Make sure that the silicone completely covers the bandage and that it is well positioned so that no liquid leaks out. Before removing, wipe the silicone dry with a cloth.

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PEPE - FULL LEG - CAST SHOWER COVER

P30033

The **PEPE - Cast Shower Covers** have been designed to protect the cast from water when bathing and prevent it from getting wet. They are intended for patients in plaster casts or bandaged due to an injury, operation or other reason that requires their use. Materials: bag: PVC; ring: PP; cover: silicone.

WARNINGS

- This product is not intended to be submerged in water. It is a splash guard to prevent splashing.
- Do not exert weight on the plastic ring.
- Do not attempt to insert any body part other than the extremities for which each product is designed into the hole.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

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