The KMINA COMFORT walking stick is designed for people who need extra support when walking. It has a secure grip handle that adapts to the shape of the hand. The wide, flexible tip increases stability and adapts to all surfaces.

HEIGHT ADJUSTMENT

1. Stand on a flat surface and let your arm hang down naturally by your side.
2. Remove the height adjustment clip and slide the aluminium tube.
3. Set the cane height so the top part of the handle touches the inside part of the wrist.
4. Place the clip.

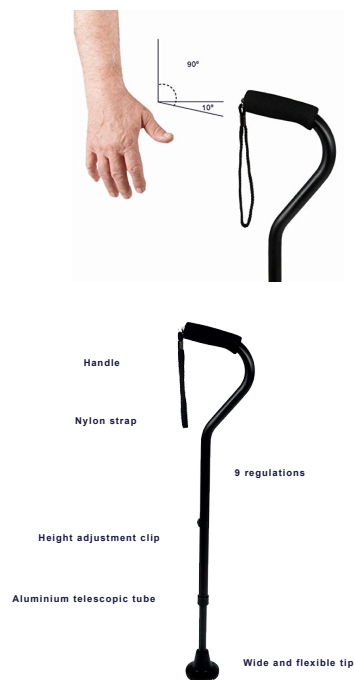
When using the cane, grip the handle **with the curve forward**.

WARNINGS

- Do not use this equipment without first reading and understanding these instructions. If needed, contact health care professional, dealer or technical personnel before attempting to use, otherwise injury may occur.
- KMINA Comfort Cane provides support, increases stability and assists while walking.
- It is designed with 10° extra inclination (physiological angle of the wrist) to avoid pain in hands and wrists.
- It provides ambulatory assistance for an individual weighing up to 330 lbs.
- It has 9 adjustments.
- Before using the cane, ensure that the height adjustment clip is securely locked in place or a fall could result, causing bodily injury and/or damage to the user.
- Inspect cane tips to check any wear and damage.
- Replace any broken, damaged or worn items immediately.
- Always consult your doctor to determine proper adjustments and use.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your cane clean by using a damp cloth. Do not use abrasive cleaning materials.

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