**KMINA - ALL TERRAIN ROLLATOR
KMINA - ANDADOR TODOTERRENO
KMINA - GELÄNDEGÄNGIGER ROLLATOR
KMINA - DÉAMBULATEUR TOUT TERRAIN
KMINA - DEAMBULATORE TUTTI TERRENI**

The **KMINA All Terrain Rollator** has been designed as an aid for people with reduced mobility who need support for walking. It has a seat for temporary use if you need to rest. It can be easily folded for storage or transport. This rollator is a walking device that improves the restricted ability to move around indoors and outdoors.

ASSEMBLY

1. Remove the rollator from the box and pull the handles outward to unfold the rollator.
2. Place the backrest in the slots at the front of the walker.
3. Adjust the walker in one of its 6 settings to the desired height. To fix it, screw the nuts that come in the box onto the sides of the walker until they are completely fixed. Be sure to adjust both sides to the same height.
4. Hook the cane holder onto the tab on the left side of the walker.
5. Finally, insert the basket into the tubes at the front of the walker.

FOLDING SYSTEM

1. To unfold, pull out the handles until the walker opens fully and push down on the seat until it opens fully.
2. To fold the rollator, remove the basket from the walker and pull the strap on the seat upwards until the handles are together.

HEIGHT ADJUSTMENT

The handlebar has six adjustments every 1.2". To adjust the correct height:

1. Stand upright next to the walker, with shoulders relaxed and arms slightly positioned at 15°.
2. Adjust the handlebar height to the desired height and secure it with the adjusting nut.

BRAKE SYSTEM

When the walker is in motion the brakes are operated by simply pushing the brake levers upwards.

1. To lock the brakes, press the brake levers down until you hear a locking sound. The wheels will be locked until you pull the brake levers up again.
2. To unlock the brakes, push the brake levers upward.

Brake pressure is adjusted by turning the adjusting nut, located on the top of the brake shoe, on the rear wheel.

1. Turning the nut counterclockwise will increase the braking pressure.
2. Turning the nut clockwise decreases the pressure required for braking.

DEPTH	WIDTH	HEIGHT (6 regulations)	FOLDING DIMENSION
25.5"	27.5"	34" - 38.5"	25.5"x33"x8"
			
SEAT DIMENSIONS	SEAT HEIGHT	GRIP DISTANCE	WHEEL DIMENSIONS
17"x9"	23"	18"	10"x1.5"
BASKET DIMENSIONS	MAX. LOAD BASKET	MAX. WEIGHT USER	WEIGHT ROLLATOR
15"x10.2"x7.5"	11 lb	300 lb	20 lb

MATERIALS

Structure: aluminum. Basket and backrest: polyester + PVC. Wheels: rubber. Handles: PVC.

TIRE PRESSURE

The recommended tire pressure is 30-35 psi.

WARNINGS

- Make sure the walker is completely open and in the correct position before use.
- Be careful when folding and unfolding the walker to avoid hurting yourself.
- Make sure that the handle is properly adjusted to the correct height. Check the stability of the walker before use.
- Test the brake mechanism before use. If the wheels turn while the brakes are locked, refer back to how to adjust the brake pressure. When the brakes are released, the wheels should turn freely.
- Use only on even surfaces, never on a slope or uneven terrain. Using the walker in any other way than intended could cause damage or injury.
- Do not leave the walker unattended without locking the brakes.
- Do not stop or turn on a ramp or slope. Do not make sharp turns. On downhill slopes, apply the brakes to prevent the walker from going faster than you.
- Use the basket only to transport objects without exceeding the maximum load: 11 lb.
- Do not hang bags from the handles, the walker may tip over.
- Do not exceed the maximum user weight: 300 lb.
- Do not expose the product to temperatures above 40°C for a prolonged period of time; otherwise the aluminum tubes will heat up and could cause burns.
- According to the manufacturer's recommendations this product is suitable for recycling.
- Inspect the walker before each use to ensure that there is no damage or excessive wear.
- Before use, check that the walker is secure and stable; and all fastenings and fittings are tight and secure. If you find any signs of wear or damage to your walker, DO NOT USE IT.
- Retain these instructions for future reference.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your walker clean using a damp cloth. Do not use abrasive cleaning materials. Do not clean or disinfect the walker using steam.

KMINA warrants this walker to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels, handles and bags. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not responsible for accidental damage or damage to property.

In case of return, properly pack the walker preferably in its original packaging, otherwise KMINA will not be responsible for possible damages caused during shipment.

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