# KMINA

**USER MANUAL  
MANUAL DE USUARIO  
BENUTZERHANDBUCH  
MANUEL D'UTILISATION  
MANUALE UTENTE**

**KMINA - UPRIGHT ROLLATOR  
KMINA - ANDADOR ANTEBRAZO  
KMINA - ARTHRITIS ROLLATOR  
KMINA - DÉAMBULATEUR VERTICAL  
KMINA - DEAMBULATORE VERTICALE**

EN

REFERENCE

KMINA - UPRIGHT ROLLATOR

K10063

The **KMINA Upright Rollator** is a walker designed for those who have balance problems, leg weakness or difficulty maintaining an upright posture while walking. Its major point of support is the forearm supports, which provide a higher level of stability by allowing the forearms to rest while the user walks. It also allows weight to be distributed more evenly, reducing pressure on the hands and wrists. It has been designed for both indoor and outdoor use and can be easily folded for easy transport. It has a space to store the cane and a lever to help lift the walker.

#### ASSEMBLY

1. Remove the walker from the box and place it with the wheels facing upwards to begin assembling.
2. Unscrew the nuts on the 4 casters, unscrew them and retighten the 4 lock nuts. Then open the frame of the walker until you hear a click to indicate that it is fully open.
3. Insert the backrest through the bottom of the handlebars and then snap both handlebars into the central frame by adjusting it to the desired height in one of its 8 settings. The forearm padding should be positioned at the rear of the walker.
4. Place the brakes on the handlebar tube and adjust it to one of the 4 settings (leaving a space of 3" - 6") that best matches the length of your arms. Tighten the thread located at the bottom of the forearm padding for a secure brake attachment.
5. Hook the cane holder into the tab on the left side of the walker.
6. Finally, insert the basket into the tubes at the front of the walker.

#### FOLDING SYSTEM

1. To unfold, pull outward on the seat ends until the seat opens fully and push down to fully engage.
2. To fold, remove the basket from the walker and pull the strap on the seat upwards.

#### HEIGHT ADJUSTMENT

The walker can be adjusted to 8 different heights between 43" - 50" by means of a push button on the side of the aluminum frame.

#### BRAKE SYSTEM

When the walker is in motion, the brakes are operated by simply pushing the brake levers inward.

1. To lock the brakes, press the brake levers outward until you hear a locking sound. The wheels will remain locked until you pull the brake levers inward again.
2. To unlock the brakes, push the brake levers inward.

The brake pressure is adjusted by turning the adjusting nut, located on the top of the brake shoe, on the rear wheel.

1. Turning the nut counterclockwise will increase the braking pressure.
2. Turning the nut clockwise decreases the pressure required for braking.

KMINA

DEPTH	WIDTH	HEIGHT (8 regulations)	FOLDING DIMENSION
26"	29.5"	43" - 50"	23"x43"x10"
			
SEAT DIMENSIONS	SEAT HEIGHT	GRIP DISTANCE	WHEEL DIMENSIONS
18"x10"	21.5"	18"	8"x1.2"
BASKET DIMENSIONS	MAX. LOAD BASKET	MAX. WEIGHT USER	WEIGHT ROLLATOR
14"x10.2"x7.5"	11 lb	300 lb	22 lb

**MATERIALS**

Structure: aluminum. Basket and backrest: Oxford cloth. Wheels: PP + PVC. Handles: PVC. Forearm padding: PU. Finishings: ABS + Nylon fiber.

**WARNINGS**

- Make sure the walker is completely open and in the correct position before use.
- Be careful when folding and unfolding the walker to avoid hurting yourself.
- Make sure that the handle is properly adjusted to the correct height. Check the stability of the walker before use.
- Test the brake mechanism before use. If the wheels turn while the brakes are locked, refer back to how to adjust the brake pressure. When the brakes are released, the wheels should turn freely.
- Use only on even surfaces, never on a slope or uneven terrain. Using the walker in any other way than intended could cause damage or injury.
- Do not leave the walker unattended without locking the brakes.
- Do not stop or turn on a ramp or slope. Do not make sharp turns. On downhill slopes, apply the brakes to prevent the walker from going faster than you.
- Use the basket only to transport objects without exceeding the maximum load: 11 lb.
- Do not hang bags from the handles, the walker may tip over.
- Do not exceed the maximum user weight: 300 lb.
- Do not expose the product to temperatures above 40°C for a prolonged period of time; otherwise the aluminum tubes will heat up and could cause burns.
- According to the manufacturer's recommendations this product is suitable for recycling.
- Inspect the walker before each use to ensure that there is no damage or excessive wear.
- Before use, check that the walker is secure and stable; and all fastenings and fittings are tight and secure. If you find any signs of wear or damage to your walker, DO NOT USE IT.
- Retain these instructions for future reference.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.
- Keep your walker clean using a damp cloth. Do not use abrasive cleaning materials. Do not clean or disinfect the walker using steam.

KMINA warrants this walker to be free from defects. The warranty will remain in effect for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement, such as brakes, wheels, handles and bags. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not responsible for accidental damage or damage to property.

In case of return, properly pack the walker preferably in its original packaging, otherwise KMINA will not be responsible for possible damages caused during shipment.

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