



1) COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI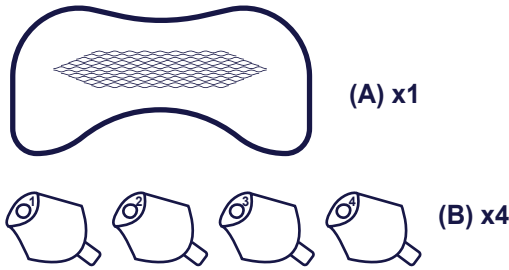

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO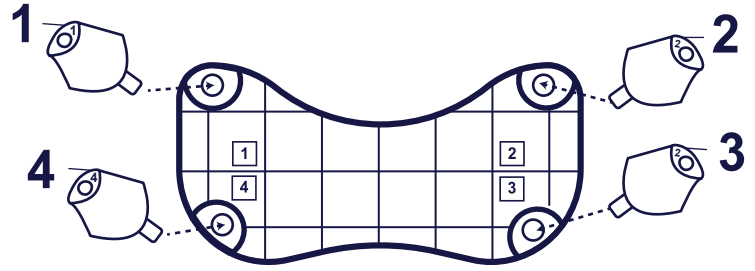

3) MEASUREMENTS / MEDIDAS / MASSE / MESURES / MISURE

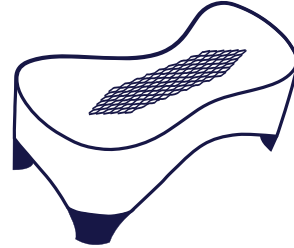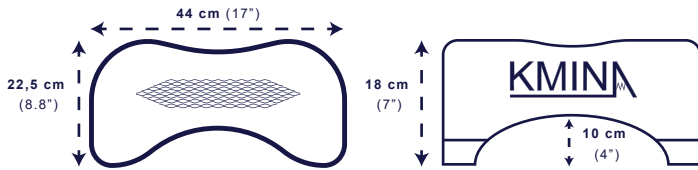**EN**

REFERENCE

KMINA - TOILET STOOL

K30006

The **KMINA TOILET STOOL** facilitates intestinal evacuation by being able to position the legs at a recommended angle of 35° in relation to the upper-body, which helps to avoid the appearance of hemorrhoids, constipation, irritation, flatulence, swelling and discomfort. It is characterized by its stable position, its rough base and non-slip feet.

**INDICATIONS OF USE**

Recommended to avoid the problems of chronic constipation, preventing injuries to the pelvic floor, prolapses or fissures. Use the toilet stool only for this purpose.

**FEATURES**

- Dimensions: 17" x 8.8" x 7"
- Material: Polypropylene; Legs - ABS plastic.
- Weight: 2.2 lb
- Maximum user weight: 330 lb
- Non-slip legs. Bamboo effect.
- Rough non-slip surface.
- Modern and functional design, it is elegant, compact, lightweight and durable.

**COMPONENTS**

Stool frame (A), legs (B) x4.

**ASSEMBLY**

1. Flip the stool frame (A) over.
2. Look at the bottom of the legs and note the numbers on each leg (B).
3. Note the numbers on each hole in the stool frame next to it.
4. Attach the legs (B) one by one in the order of the numbers, fitting the holes.

**CLEANING AND DISINFECTION**

The product can be cleaned and disinfected with products available on the market. Always keep the product clean and wash it regularly and after each use.

1. Use water and a mild non-abrasive detergent to clean.
2. Disinfect by cleaning all accessible surfaces with disinfectant.

**GENERAL WARNINGS**

- This toilet stool is designed to facilitate intestinal evacuation and defecation. Use the stool for this purpose, do not sit on it or stand up as it may cause falls and trips.
- Use the toilet stool only when you are seated.
- Make sure the product is not damaged and that you have all the components.
- Make sure all four legs are adjusted to the same height.
- Make sure the non-slip caps on the legs are tight.
- Do not use for side transfers or part load.
- Use the product only on even ground.
- It is not intended for use by children under 12 years of age.
- Consult a physician before use or doubts and discontinue use if you suffer back or leg discomfort as a result.

**WARRANTY INFORMATION**

We provide a manufacturer's warranty for the product in accordance with our general commercial terms and conditions in the respective countries.

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