**HANDLE ASSEMBLY / MONTAJE DE LAS ASAS / MONTAGE DER GRIFFE / MONTAGE DES POIGNÉES / MONTAGGIO DELLE MANIGLIE**

EN

REF.	K30012
KMINA - SOFT TOILET RISER WITH HANDLES	K30012

The KMINA - TOILET RISER SOFT WITH HANDLES 4" is ideal for people with difficulty sitting or getting up from the toilet, as it offers an extra height that makes it easier and safer. This toilet is designed for the elderly, people with knee or hip problems, people with reduced mobility or people who suffer from some kind of physical or mental disability. It adapts to most classic oval toilets.

**ASSEMBLY**

The box contains: A. Raised toilet seat with cover, B. Screw-in brackets (2) and C. Armrests.

**To install the handles:**

1. Insert each handle into the holes. Place the handle numbered 1 into the holes with the same number. Do the same with the handle numbered 2. The handles are correctly positioned when you hear a "click".

**To install the toilet riser:**

1. Place the lift upside down on a flat surface and insert the bolt-on brackets into the holes.  
2. Tighten and turn the screw handle clockwise to lock the brackets. Do not tighten the brackets all the way, otherwise it will not be possible to fit the seat.  
3. Lift the lid and seat off the toilet and place the booster seat on the toilet. Make sure that the inner rim is placed inside the toilet and the front rim is placed on the outside.  
4. Turn the head of the brackets clockwise until the raised toilet seat is fixed and immovable.

**CARE AND CLEANING**

■ All parts of the toilet riser can be cleaned with soap and hot water or disinfected with NON-ABRASIVE products.

**GENERAL WARNINGS**

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. Don't use the product if it is damaged.
- The raised toilet seat must not be used to support more than 220 lb.
- It is recommended for use by one user only. If it is to be used by another person, the device should be sanitized and checked for residual body fluids.
- Check the screw-in brackets and, if necessary, retighten.
- DO NOT allow children to play or operate the device.
- DO NOT use the cover as a seat.
- Use the toilet lift only for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet lift, or for injuries or damage caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

**WARRANTY**

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require natural wear and tear requiring periodic replacement.

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