


1) COMPONENTS / COMPONENTES / KOMPONENTEN / COMPOSANTS / COMPONENTI

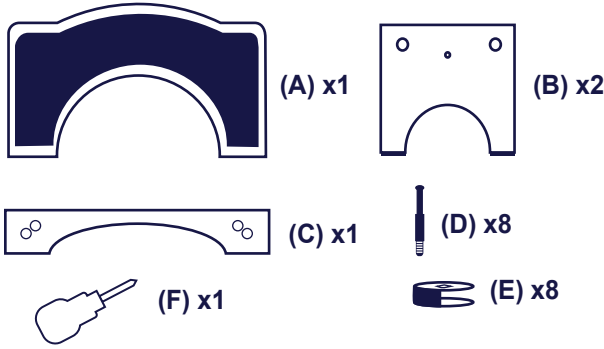3) MEASUREMENTS/MEDIDAS/MASSE/MESURES/MISURE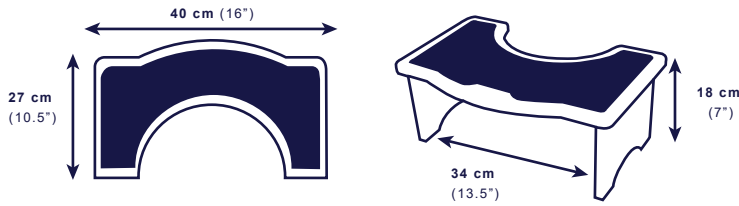

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

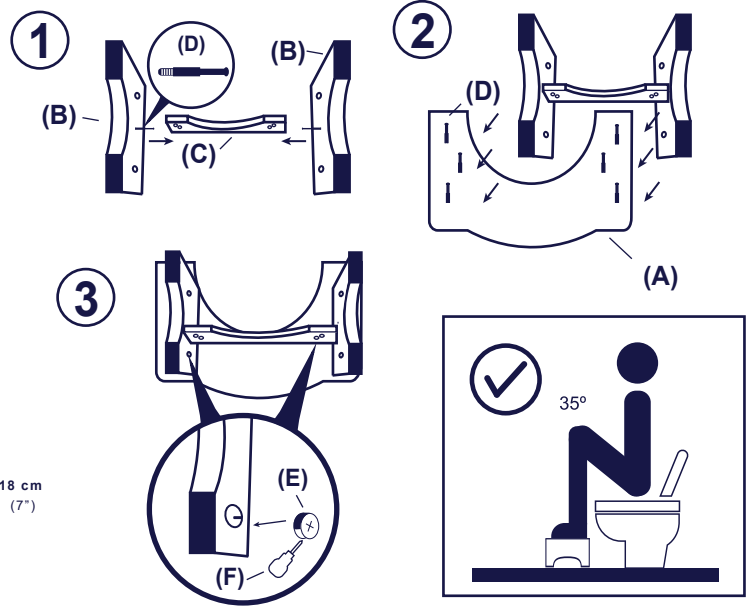EN

CODE

KMINA - WOODEN TOILET STOOL

K30015

The **KMINA WOODEN TOILET STOOL** facilitates bowel movements as the legs can be placed at a recommended angle of 35° in relation to the torso, which helps to prevent the appearance of haemorrhoids, constipation, irritation, flatulence, bloating and discomfort. It is characterised by its stable position and non-slip base and legs.

INDICATIONS FOR USE

Recommended to avoid chronic constipation problems, preventing pelvic floor injuries, prolapse or fissures. Use the stool for this purpose only.

FEATURES

- Dimensions: 16" x 10.5" x 7"
- Material: wood; Non-slip surface: rubber.
- Weight: 3.3 lb
- Maximum user weight: 330 lb
- Rough non-slip surface.

COMPONENTS

Anti-slip top board (A), legs (B), leg connection arc (C), screws (D), screw locking wheel (E), spanner (F).

ASSEMBLY

1. Take the legs (B) and place a screw (D) in the central hole of each one. Then insert the leg connection arc (C) forming a structure as shown in picture 1.
2. Turn the top board (A) upside down and insert the remaining 6 screws (D) into the holes indicated for this purpose. Then, fit the leg structure with the board as shown in picture 2.
3. Finally, screw the locking wheels (E) into each of the holes where the screws protrude using the included spanner (F).

CLEANING AND MAINTENANCE

The product can be cleaned and disinfected with commercially available products. Always keep the product clean and wash it regularly and after each use.

1. Use a dry cloth. If it gets wet, dry it afterwards to prevent the wood from swelling.
2. Disinfect by wiping all accessible surfaces with disinfectant.
3. Do not allow the stool to remain in very humid environments to prevent the wood from swelling.

GENERAL WARNINGS

- This Wooden Toilet Stool is designed to facilitate intestinal evacuation and defecation. Use the stool for this purpose, do not sit on it or stand up as it may cause falls and trips.
- Use the toilet stool only when you are seated.
- Make sure the product is not damaged and that you have all the components.
- Make sure that the two legs are correctly adjusted.
- Make sure the non-slip caps on the legs are tight.
- Do not use for side transfers or part load.
- Use the product only on even ground.
- It is not intended for use by children under 12 years of age.
- Consult a physician before use or doubts and discontinue use if you suffer back or leg discomfort as a result.

WARRANTY INFORMATION

We provide a 3-year manufacturer's warranty for the product in accordance with our general terms and conditions of business in the respective countries.

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