**USER MANUAL  
MANUAL DE USUARIO  
BENUTZERHANDBUCH  
MANUEL D'UTILISATION  
MANUALE UTENTE**

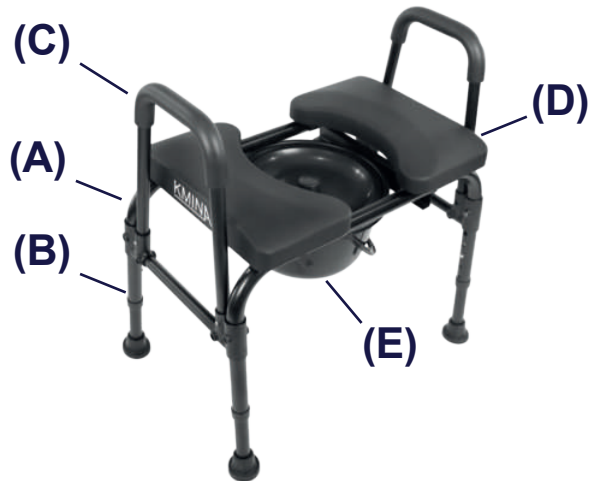
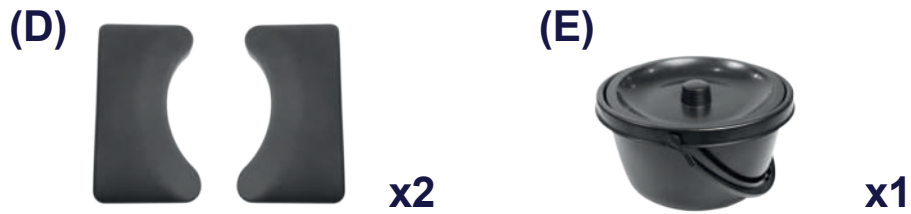
**KMINA - BEDSIDE COMMODE  
KMINA - SILLA INODORO  
KMINA - TOILETTENSTUHL  
KMINA - CHAISE PERCÉE TOILETTE  
KMINA - SEDIA COMODA WC**

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
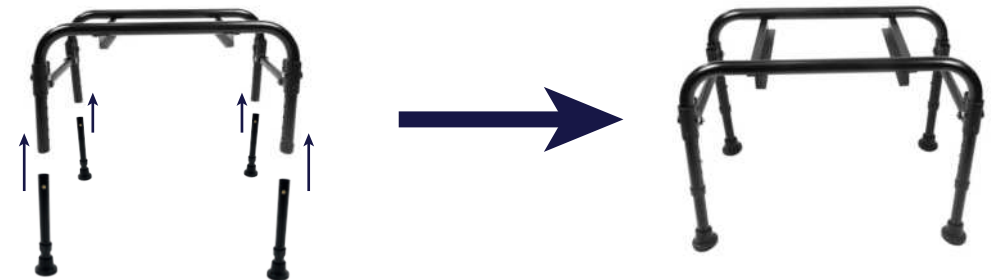
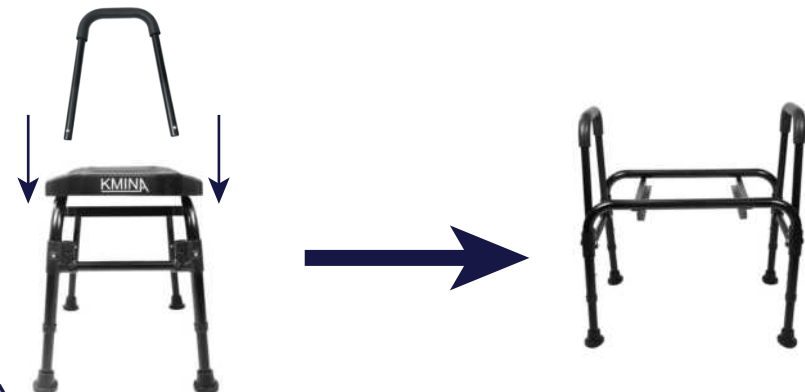
**EN 3. Seat assembly.** Place each part of the seat (D) on the frame (A) pressing until the clamp engages with the main frame tubes. The curved part should be on the inside of the frame. Open or close the seat to adjust it to the proper size.

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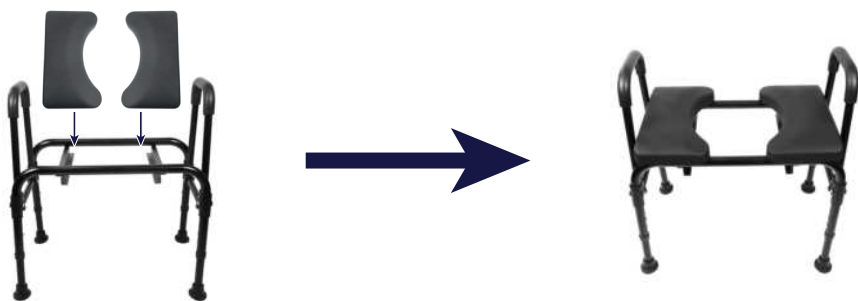
**EN 4. Assembly of the bucket.** Finally, place the bucket (E) on the lower frame rails (A) and slide it towards the center to make use of its portable toilet function.

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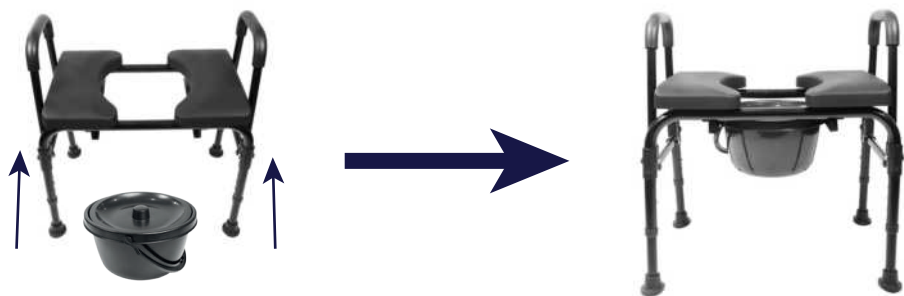**MATERIALS**

Frame and legs: aluminum. Seat: high density PU foam and wood. End cap: thermoplastic rubber (TPR). End caps: polypropylene.

**WARNINGS**

- Check that the seat is securely fastened.
- Exercise caution when removing the handles. You may have to exert some force and may hit yourself when removing them.
- Remove the bucket when it is used in its shower chair or lift function.
- Do not stand on this product.
- Do not use the product on ramps or stairs.
- Make sure that all four legs are at the same height and that the spring-loaded knobs engage all holes.
- Do not expose the commode chair to the sun to prevent the seat from cracking.
- The commode chair is intended for household use.
- Do not exceed the maximum weight supported by the product, which is 400 lb.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

**CLEANING AND MAINTENANCE**

Clean the product with soap and water. Use mild detergents, do not apply abrasive products.

**WARRANTY**

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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