RISER ASSEMBLY / MONTAJE ELEVADOR / MONTAGE TOILETTENSITZERHÖHUNG / MONTAGE REHAUSSEUR / MONTAGGIO RIALZO

EN

REF.

KMINA - PADDED TOILET RISER 2" | K30030

The **KMINA - PADDED TOILET RISER 2"** is ideal for people with difficulty sitting or getting up from the toilet, as it offers an extra height that makes it easier and safer. This toilet is designed for the elderly, people with knee or hip problems, people with reduced mobility or people who suffer from some kind of physical or mental disability. It adapts to most classic oval toilets.

FEATURES

- Thickness: 2" (approx.)
- Toilet riser weight: 1 lb.
- 100% water-resistant, non-slip material.
- Maximum user weight: 300 lb.

ASSEMBLY

To install the toilet riser place it on the toilet and, tightly fit the fins inside the toilet riser to the seat, so that it is fully engaged, as shown in the picture.

CARE AND CLEANING

■ Clean with water and neutral soap. The use of chlorine bleach, bleach, alcohol-based disinfectants or abrasive products is not recommended. Clean before each use if the user changes or has been in contact with body fluids.

GENERAL WARNINGS

- It is recommended for use by one user only. If it is to be used by another person, the device must be sanitized and checked to ensure that no traces of body fluids remain.
- Check before use that the toilet riser is correctly adapted to the toilet in which it is to be used.
- During the injection process the measurements may vary and damage may occur.
- Caustic products, desolvents and sharp or cutting objects in direct contact with the lifter are prohibited.
- Regularly examine the hoist for normal wear and tear: holes, cracks, changes in surface appearance, etc.
- Do not burn or dispose of in nature outside the places intended for this purpose.
- Use the toilet riser only for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet lift, or for injury or damage incurred as a result thereof.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require natural wear and tear requiring periodic replacement.

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