

K40003
K40004

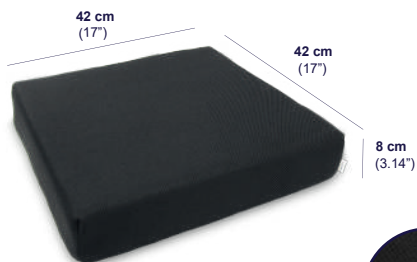K40011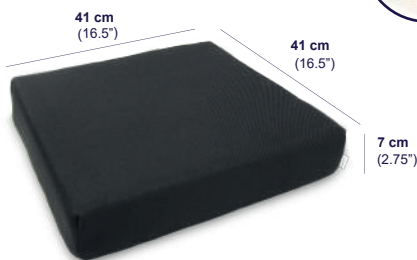

REF: K40003*
4 cm (1.57")
4 cm (1.57")

REF: K40004
8 cm (3.14")

REF: K40011
7 cm (2.75")

EN

CODE
K40003
K40004
K40011

ANTI-DECUBITUS CUSHION

The antidecubitus cushion prevent the suffering of bed sores and decubitus ulcers, typical of bodies that remain immobile for a long period of time. These injuries occur mainly in the body's support points. Excessive pressure of the bones deprives them of blood and oxygen, resulting in redness, skin tissue and deep wounds. To avoid the appearance it would be convenient to place appropriate products behind each support point and to change the patient in different body positions regularly. It is also crucial to avoid any type of humidity because this fact favors the formation of these bed scores.

INDICATIONS OF USE

Recommended for the prevention of bedsore and improve rest. Depending on the risk of bedsore and the user's weight, the use of one cushion or another will be recommended. The shape of the cushion is designed to fit wheelchairs and is recommended for use both by private users and in Hospitals, Residences and Clinics. We advise to position the cushion preferably on a flat and rigid base.

FEATURES

■ Dimensions:

PRODUCT	DIMENSIONS	MATERIAL	VISCO THICKNESS	PU THICKNESS	DENSITY	HARDNESS
K40003	17"x17"3.14"	Viscoelastic foam (VISCO) Polyurethane (PU)	1.57"	1.57"	1.43 lb/ft³	1,7 - 2,5 kPa
K40004	17"x17"3.14"	Viscoelastic foam (VISCO)	3.14"	-	2.80 lb/ft³	1,7 - 2,2 kPa
K40011	16.5"x16.5"x2.75"	Viscoelastic foam (VISCO)	2.75"	-	4.07 lb/ft³	2,9 kPa

- Maximum user weight: 220 lb.
- Cover with 3D fabric, soft, washable, breathable, waterproof and non-slip.
- Fabric Composition: 100% PES + Membrana PU.
- Certificate OEKO-TEX STANDARD 100.
- Barrier antibacterial, antifungal, antichinches and antimites. For its full effectiveness, the cushion cover must be completely closed.

CLEANING AND DISINFECTION

The cover of the cushion is removable in order to clean it:

- Clean only with water and neutral soap. It is not recommended to use: chlorine, bleach, disinfectants with alcohol or abrasive products.
- We recommend to pre-washed a small surface area be when using hospital cleaning products.
- Never use drying presses, the waterproof layer can be broken by not letting the water molecules out. Important not to wet the viscoelastic foam.
- Do not iron.

⚠ WARNINGS!

- Check that the cushion fits correctly on the seat where it is to be used.
- It is recommended that the cushion is 0.4"-0.8" smaller than the seat.
- *K40003: The visco-elastic part (white color) facing up, as in the image.
- Caustic products, paint strippers, solvents and sharp objects in direct contact with the cushion are prohibited.
- Check the cushion regularly for normal wear and tear: holes, cracks, change in surface appearance, etc.
- Store the cushions preferably in a flat position and protected from natural light and moisture.
- Do not burn or throw away in the wild, outside of the designated areas.
- The guarantee will remain in force for 3 years from the date of the consumer's original purchase. This warranty does not cover normal wear and tear of the product and does not replace legal warranties.
- Please observe the care and maintenance instructions. The manufacturer assumes no liability for damage resulting from improper care or maintenance.
- Please contact the company in the event of a problem to resolve it.
- This does not affect your legal rights.
- Please keep these instructions for future reference.
- According to the manufacturer's recommendations this product is suitable for recycling.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

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