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EN The KMINA - TABLET CRUSHER easily reduces tablets to powder to make swallowing easier. It contains a storage compartment for tablets on the top of the lid. *Overall size:* Ø 2.15", 1.8" height.

INSTRUCTIONS FOR USE

1. Open the screw top and put the tablet into the container.
2. Turn the screw top clockwise and anticlockwise continuously to crush the tablets into powder.
3. Spare tablets can be stored in a compartment in the lid.

MAINTENANCE

- It can be cleaned on the top shelf of dishwasher.

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WARNINGS

- Do not use for tablets prescribed to swallowed as a whole.
- DO NOT use UV light to disinfect the product.
- Periodically inspect for wear and damage.

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EN The **KMINA - TABLET SPLITTER** cuts tablets in half for easy swallowing or for storage in small doses. Stainless steel blade neatly cuts tablets in half. It includes a compartment to store tablets. *Overall Size:* 3.3"x1.8"x1".

INSTRUCTIONS FOR USE

1. Open the top of the splitter and place the tablet in the middle of the V shape notch.
2. Press down the top and the tablet will be cut in half.
3. Store the tablet which is not going to use in the compartment.

MAINTENANCE

- Wash with normal cleanser and water.
- DO NOT use in the dishwasher.
- DO NOT use UV light to disinfect the product.

WARNINGS

- Avoid direct contact your finger with the sharp blade.
- Capsule must be swallowed as a whole.
- Keep out of the reach of children.
- Replace a new one if you find the blade become rusty.
- Periodically inspect for wear and damage washed.

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