EN

REFERENCE

KMINA - MONTHLY PILL ORGANISER 3 TIMES A DAY

K40016

The **KMINA MONTHLY PILL ORGANISER 3 TIMES A DAY** is ideal for people who take a large number of tablets every day. It is suitable for three servings and therefore has clearly differentiated compartments for breakfast, lunch and dinner.

INSTRUCTIONS FOR USE

1. Place a coloured sticker on each of the 31 individual pill boxes. This way you can organise your pills in the way that suits you best: marking the weekend, distinguishing between odd and even days, etc.
2. Organise your monthly pills once to avoid forgetfulness.
3. Make sure you hear a click when you close the individual pill dispenser, to prevent pills from slipping out of their compartment.

WARNINGS

- Keep out of reach of children.
- DO NOT wash in dishwasher.
- DO NOT use ultraviolet light to disinfect the product.
- Inspect periodically for wear or damage.

CLEANING AND MAINTENANCE

Before first use, wash with water and dry before inserting tablets.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorised dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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