EN

REF.

KMINA - MONTHLY PILL ORGANISER 4 TIMES A DAY

K40049

The **KMINA MONTHLY PILL ORGANISER 4 TIMES A DAY** is ideal for people who take a large number of tablets every day. It is suitable for two doses and therefore has clearly differentiated compartments for day and night. Material: polypropylene (PP).

#### INSTRUCTIONS FOR USE

1. Place the white stickers with the numbers 1 to 31 on the side of each of the individual pill dispensers. Similarly, place the blue stickers with a white check on the opposite side. In this way, you can see at a glance which days you have taken your medication.
2. Organize your monthly pills in a single time to avoid forgetfulness.
3. Make sure you hear a click when you close the individual pill dispenser to prevent pills from slipping out of their compartment.

#### WARNINGS

- Keep out of reach of children.
- DO NOT wash in dishwasher.
- DO NOT use ultraviolet light to disinfect the product.
- Inspect periodically for wear or damage.

#### CLEANING AND MAINTENANCE

Before first use, wash with water and dry before inserting tablets.

#### WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorised dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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