



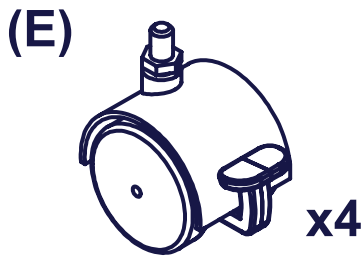
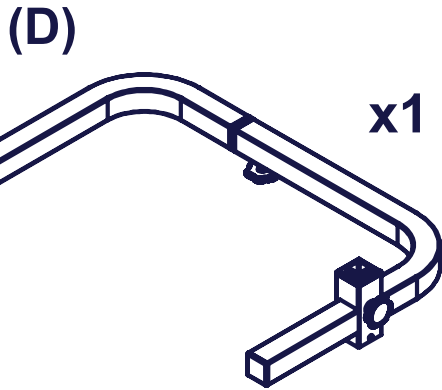
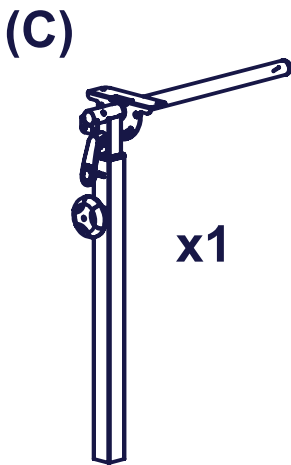
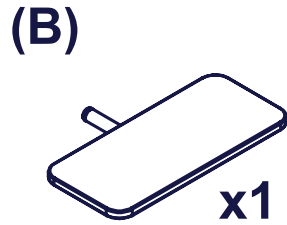
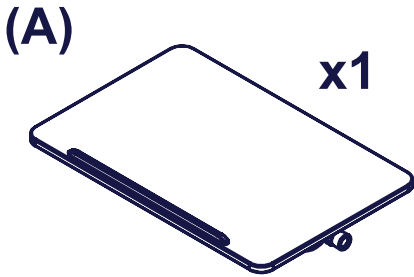
# KMINA





**USER MANUAL  
MANUAL DE USUARIO  
BENUTZERHANDBUCH  
MANUEL D'UTILISATION  
MANUALE UTENTE**

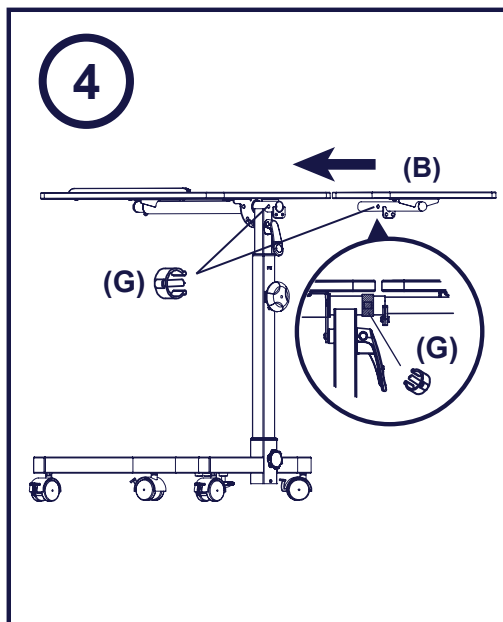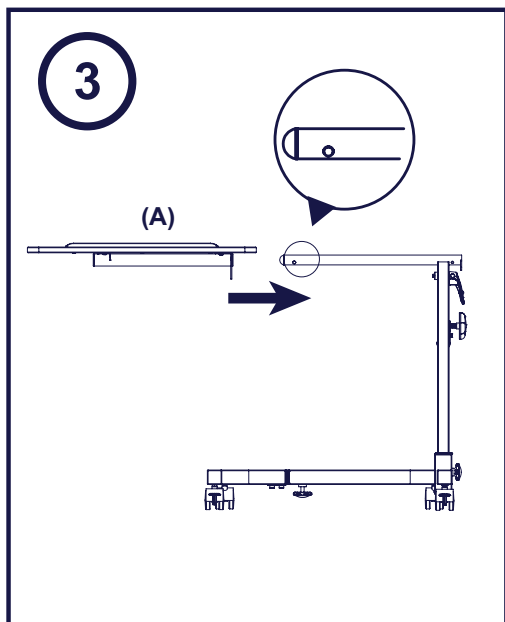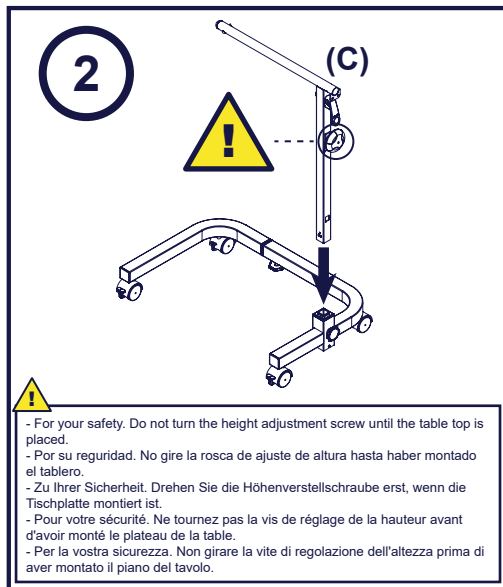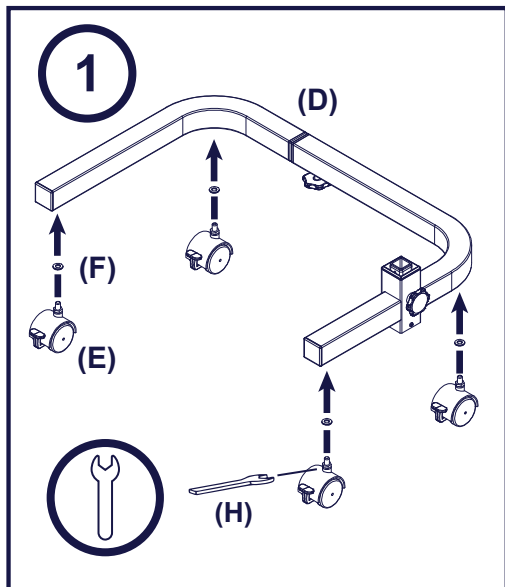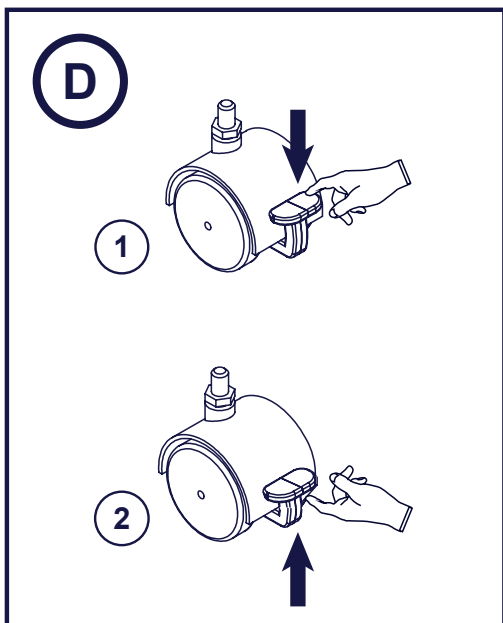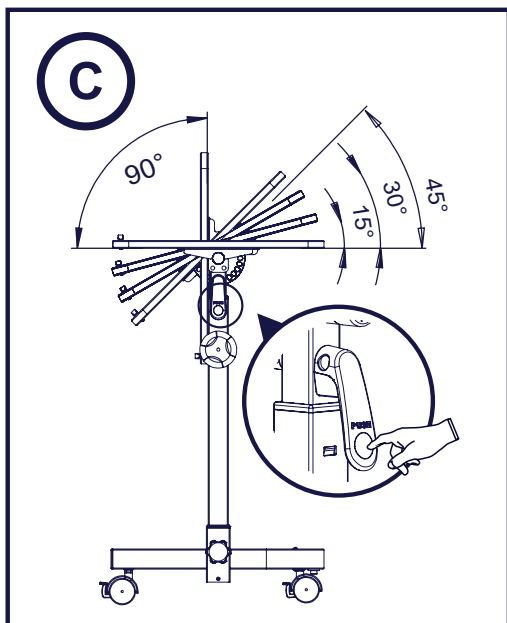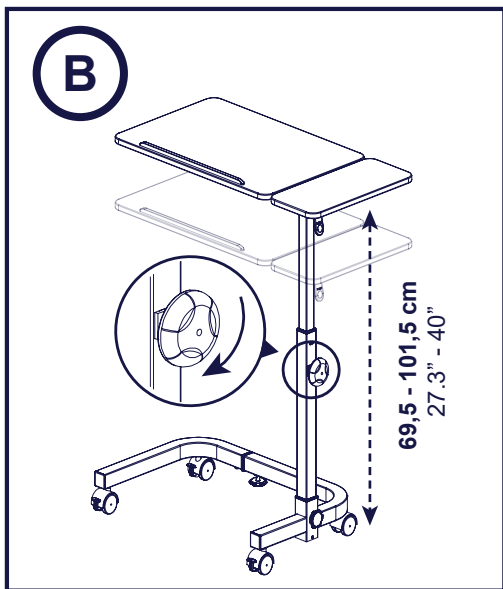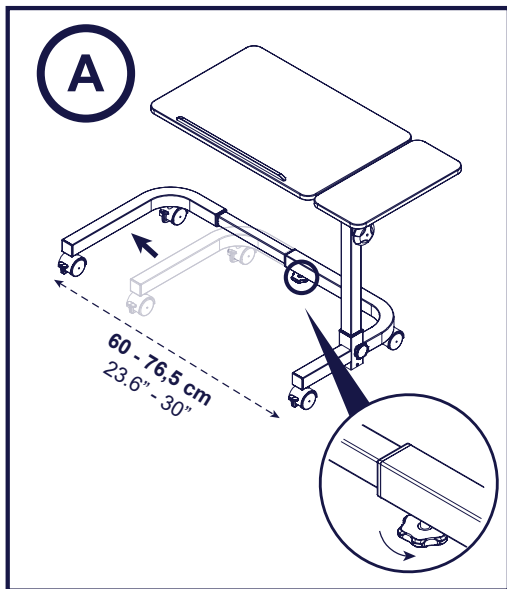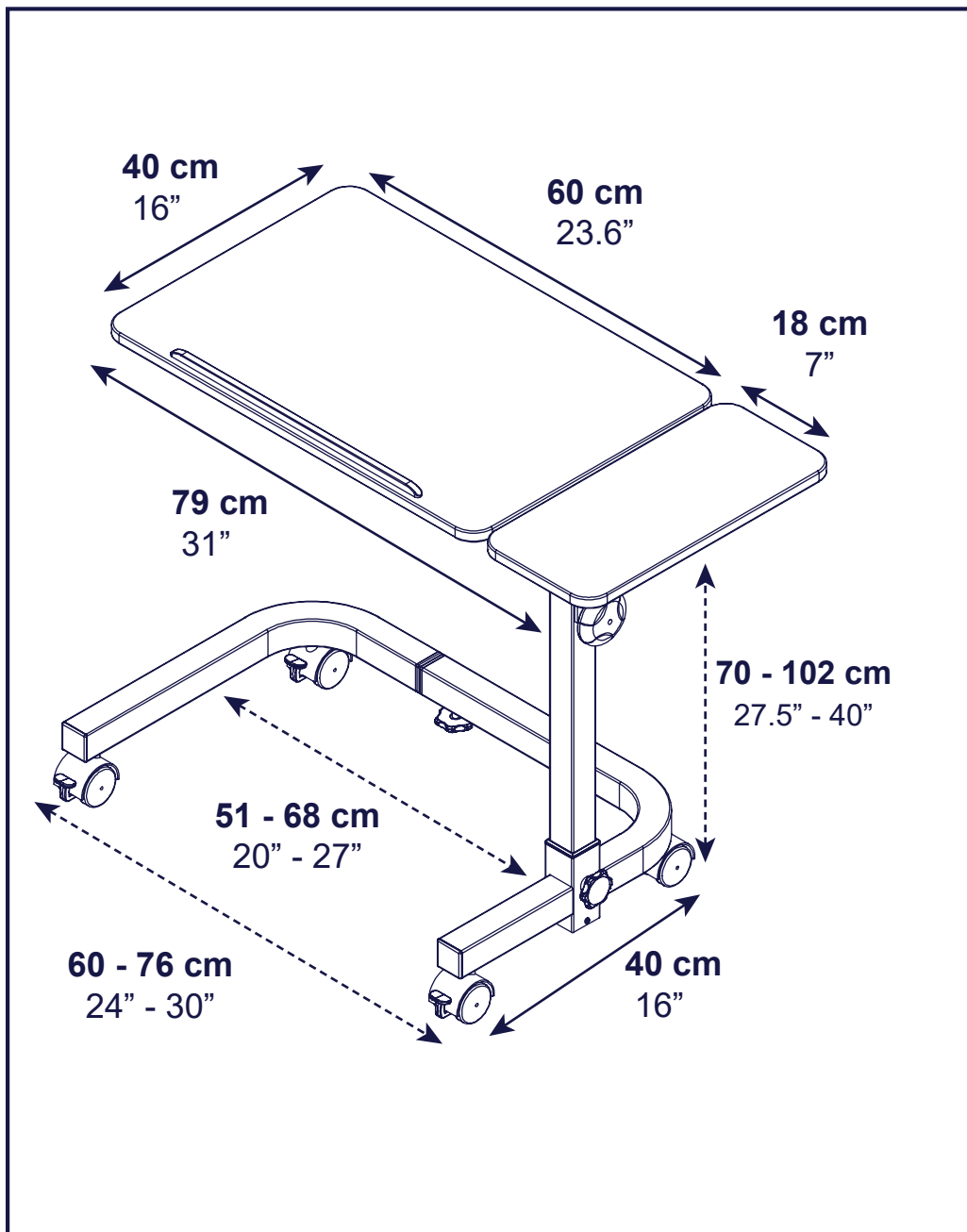
**KMINA - OVERBED TABLE  
KMINA - MESA AUXILIAR CAMA  
KMINA - BETTTISCH  
KMINA - TABLE DE LIT  
KMINA - TAVOLINO DA LETTO**

The **KMINA OVERBEDTABLE** is designed to make everyday life easier for people who are bedridden or in a wheelchair. It is ideal for reading and writing in bed, using the computer or eating. Thanks to its folding table, adjustable height and width, it can be adjusted in different positions and adapted to the user.

### COMPONENTS (Page 1)

Large board (A), Small board (B), L-shaped structure (C), U-shaped base (D), Castors (E), Washers (F), Fixing clip (G), Wrench included for assembly (H). **MATERIALS:** steel, wood, metal and plastic.

### ASSEMBLY (Page 2)

1. Screw the castors (E) with the washers (F), to the U-shaped base (D). Use the adjustable wrench (H) to screw them in correctly.
2. Insert the L-shaped frame (C) into the U-shaped base (D) with the handle facing outwards. Tighten the side thread of the U-shaped base (D) correctly to ensure a firm and secure structure. Do not turn the height adjustment thread of the L-shaped frame (C) until you have placed the table top in step 4.
3. Insert the large board tube (A) into the L-shaped frame (C) until the centre frame clip pops out again.
4. Then place the small board (B) into the centre frame tube from the crank side. To adjust it, use the adjustment clip (G) by pressing it until it is securely fastened.

### ADJUSTMENT (Page 3)

**A. To adjust the table width:** unscrew the nut in the middle of the U-shaped base (D), set the desired width, and tighten the thread until it is secure. The table width adjustment is: 23.6" - 30".

**B. To adjust the height of the table:** unscrew the nut in the middle of the L-shaped frame (C), set the table to the desired height, and tighten the thread until it is securely fastened. The height adjustment of the table is: 27.3" - 40". Adjust the height of the table carefully and after placing the small table, otherwise the frame rises sharply and may cause damage.

**C. To adjust the tilt of the table:** press the side handle while adjusting the angle of the table, when you have found the right tilt, release the handle. The table has 9 settings and can be tilted up to 90° in both directions and is completely vertical. Only the large table can be tilted. The small table remains fixed on the table.

**D. To lock the castors:** push the brake lever downwards until you hear a "click". To unlock the wheels, push the same lever upwards until you hear the same sound.

### WARNINGS

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

#### Safety precautions. Pre-use checks:

- Read the instructions before installation.
- All threads must be tight and snug.
- Confirm that all components and parts are properly assembled and screwed together before use.
- Be sure to mount the product on a completely flat surface.
- After installing the castors, activate the braking system for safer mounting.

#### Safety precautions. During the use:

- Brake the castors for better use of the table and to prevent the table from rolling away.
- Do not exceed the maximum weight supported by the product, which is 22 lbs.
- Do not lean on the suspended end of the table top; this may cause the table to tip.
- Be careful not to get your fingers caught in the moving part of the table when adjusting the tilt of the table.
- Adjust the height carefully to avoid a sharply rise. Do not adjust the height until the top board is in place.

### CLEANING AND MAINTENANCE

- Clean regularly by hand with a damp cloth, do not use abrasive cleaning agents or rinse with water.
- Use mild detergents to clean the table.
- For disinfection in hospital environments, please follow the instructions of a hygiene specialist.

### WARRANTY

KMINA warrants this device to be free from defects. The warranty shall remain valid for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorised dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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