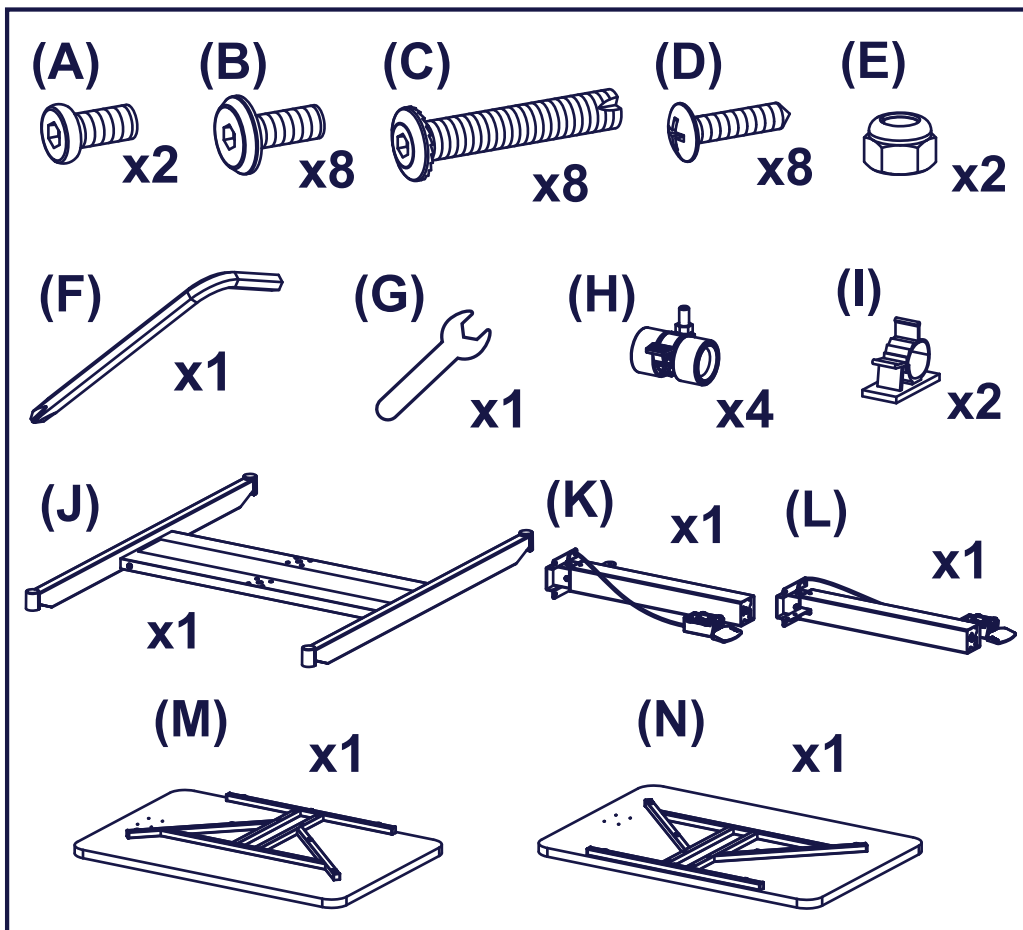
KMINA


USER MANUAL
MANUAL DE USUARIO
BENUTZERHANDBUCH
MANUEL D'UTILISATION
MANUALE UTENTE

KMINA - DOUBLE SIDE TABLE
KMINA - MESA AUXILIAR DOBLE
KMINA - DOPPELBEISTELLTISCH
KMINA - TABLE D'APPOINT DOUBLE
KMINA - TAVOLINO DOPPIO

	ENGLISH	ESPAÑOL	DEUTSCH	FRANÇAIS	ITALIANO
A.	6x13 mm Allen Bolt	6x13 mm Tornillo Allen	6x13 mm Inbus-Schraube	Vis Allen 6x13 mm	Bullone a brugola 6x13 mm
B.	6x16 mm Allen Bolt	6x16 mm Tornillo Allen	6x16 mm Inbus-Schraube	Vis Allen 6x16 mm	Bullone a brugola 6x16 mm
C.	8x45 mm Allen Bolt	8x45 mm Tornillo Allen	8x45 mm Inbus-Schraube	Vis Allen 8x45 mm	Bullone a brugola 8x45 mm
D.	4x18 mm Wood Screw	4x18 mm Tornillo Estrella	4x18 mm Holzschraube	4x18 mm Vis à étoile	Vite a stella 4x18 mm
E.	Nut	Tuerca	Schraubenmutter	Écrou	Noce
F.	Allen Wrench	Llave Allen	Inbusschlüssel	Clé Allen	Chiave a brugola
G.	Wrench	Llave inglesa	Schraubenschlüssel	Clé de serrage	Chiave inglese
H.	Caster	Ruedas	Nachlauf	Roulettes	Ruote
I.	Cable management clip	Clip sujeción del cable	Kabel-Halteklammer	Clip de maintien du câble	Clip di fissaggio del cavo
J.	Base	Base	Basis	Base	Base
K.	Front leg	Pata delantera	Vorderes Bein	Jambe avant	Gamba anteriore
L.	Back leg	Pata trasera	Hinteres Bein	Jambe arrière	Gamba posteriore
M.	Front top board	Tabla delantera	Vorderes oberes Brett	Panneau frontal	Pannello frontale
N.	Back top board	Tabla trasera	Hinteres oberes Brett	Panneau arrière	Pannello posteriore

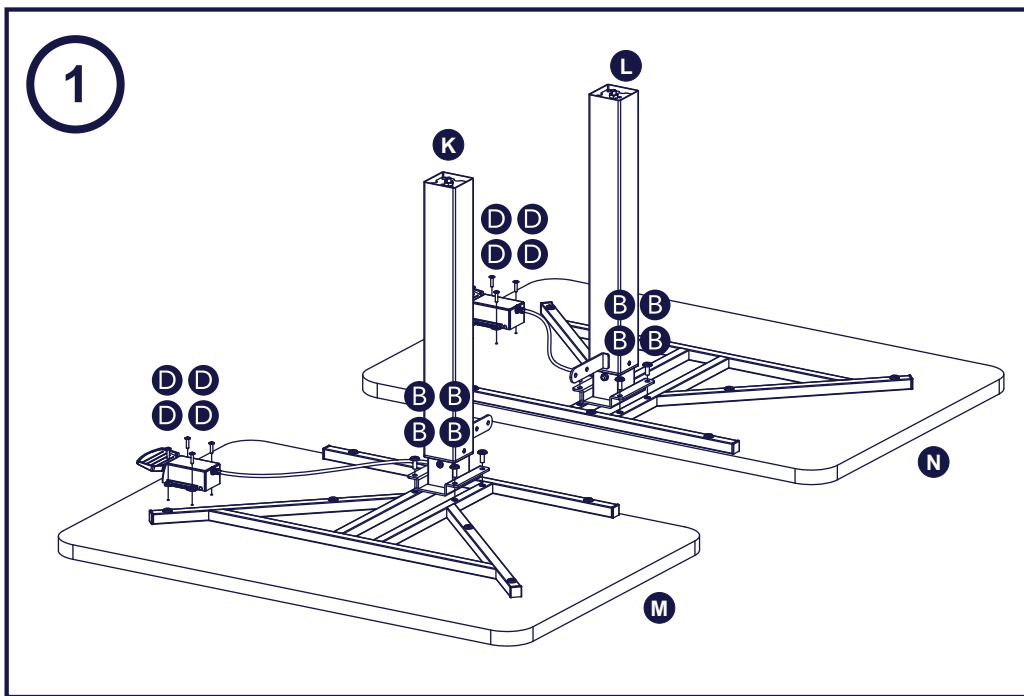
EN 1. Place the front top board (M) upside down and place the front leg (K) on top of it. Use the 6x16 mm Allen Bolt (B) and screw them into the board. Use the star tip of the Allen key (F) to fix them tightly. Then place the lever attached to the front leg (K) and screw it into the holes in the middle of the board. To do this, use the 4x18 mm wood screws (D) and use the star tip of the Allen key (F). Repeat the procedure with the back top board (N) and the back leg (L). In the hardware bag you will find spare parts (B) and (D) in case you need them.

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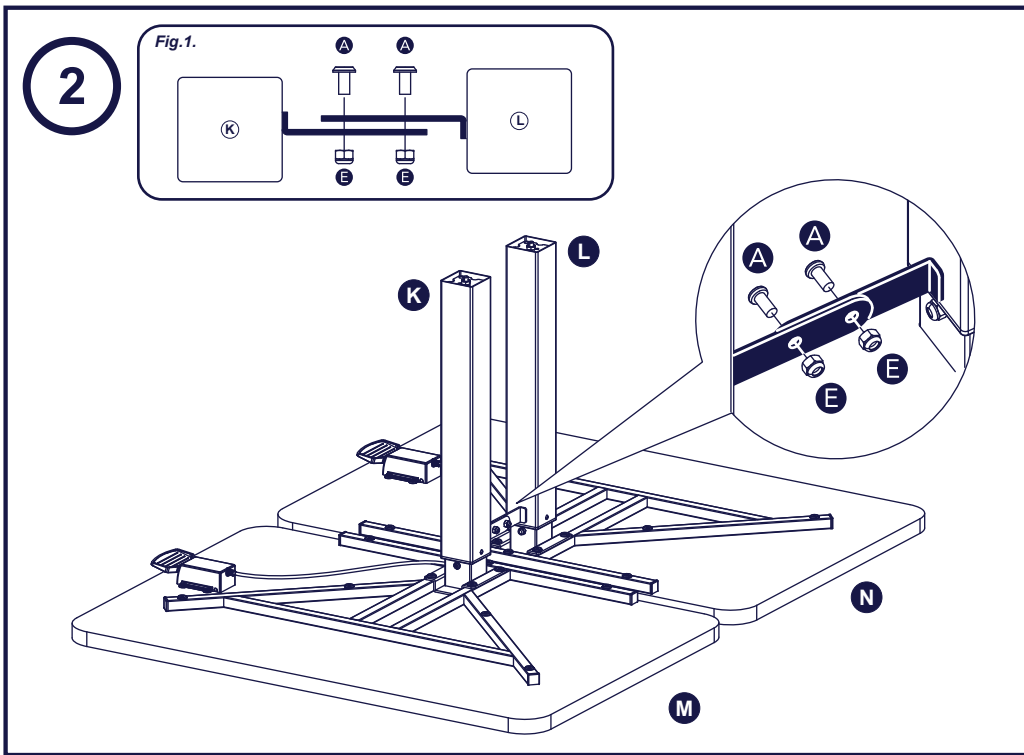
EN 2. Join the projections of the legs and fix them using the 6x13 mm screw (A) and the nuts (E). If desired, you can use the star tip of the Allen key (F) and the wrench (G). Make sure that you place the legs in the order shown in Fig. 1.

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
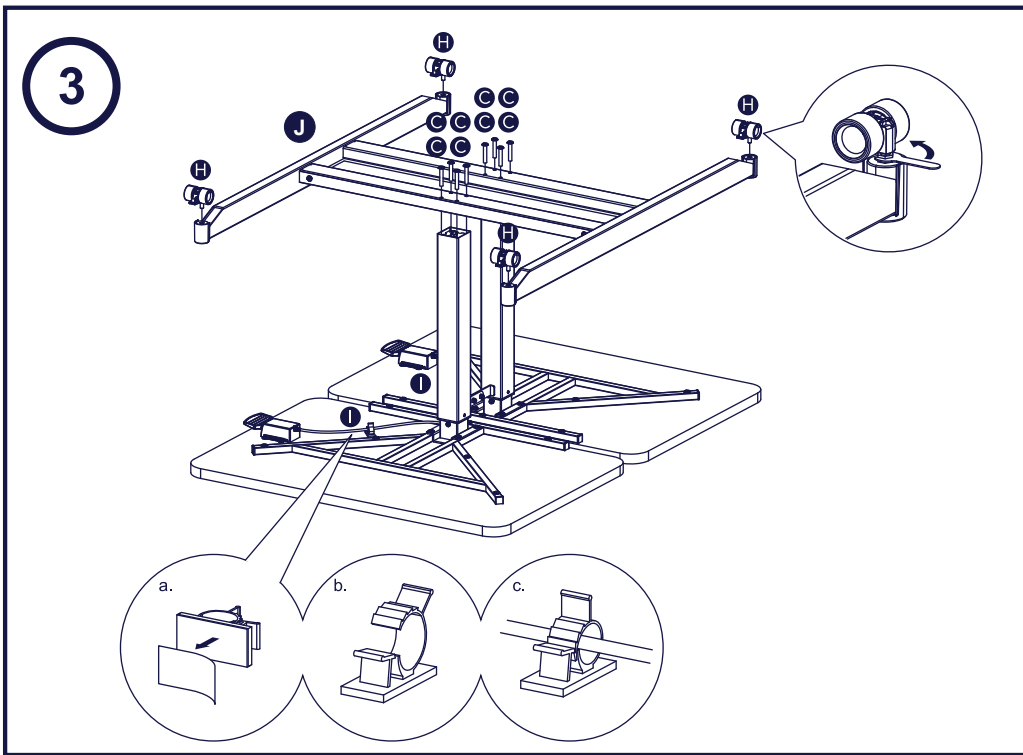
EN 3. Place the base (J) upside down on top of the legs. Proceed carefully because it is a heavy element. Screw in the 8x45 mm Allen Bolt (C) using the Allen key (F). Then screw each of the castors (H) onto the ends of the base (J) using the wrench (G). Finally, peel the adhesive off the cable management clip (I) and place them on the wooden boards. Pass the cable protruding from the legs through the clip (I) for a tighter fit. Rotate the Double Side Table and it is ready for use.

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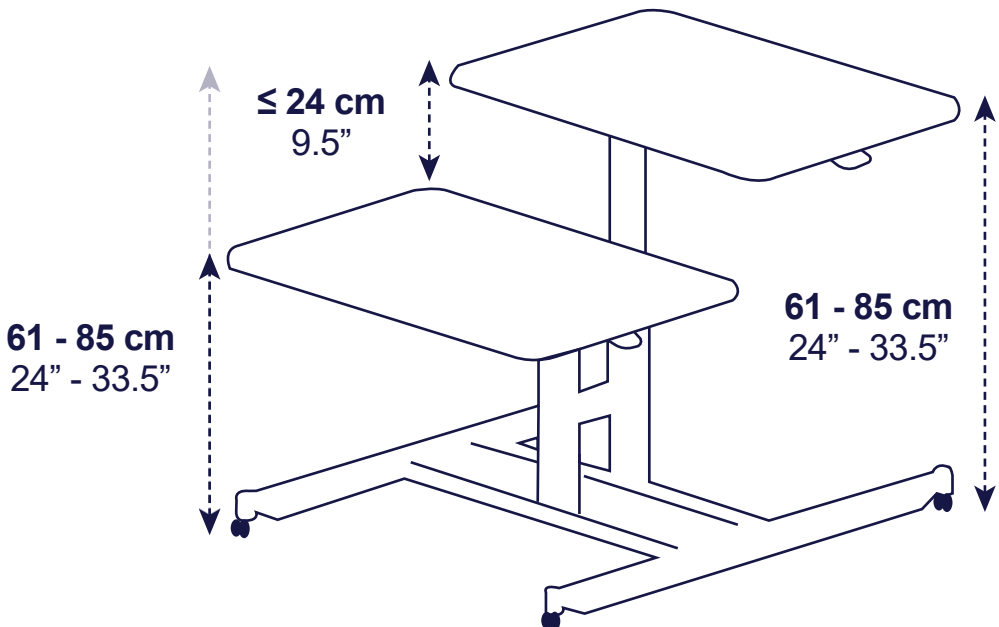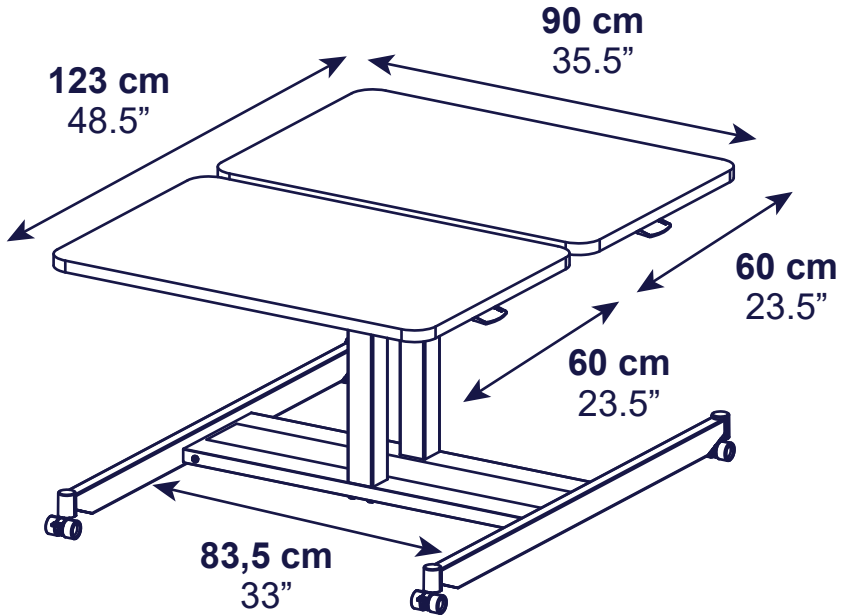The **DOUBLE AUXILIARY TABLE** is designed to facilitate the daily life of patients in wheelchairs. It is ideal for reading and writing, using the computer, or eating. It has two folding wooden tables whose height can be adjusted individually, which allows to adapt to the most comfortable height for the patient and his companion. Weight: 92 lb.

MATERIALS

Wooden tabletop (grade E1). Steel structure.

HEIGHT ADJUSTMENT

To adjust the height of each of the boards:

- **To lower the board:** press the lever located on the side of the board and, with the help of the other hand, push the board down to the desired height.
- **To raise the table:** press the lever on the side of the wooden table top and adjust the table to the desired height.

WARNINGS

Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

Pre-use checks:

- Read the instructions before installation.
- Confirm that all components and parts are properly assembled and screwed together before use.
- Be sure to mount the product on a completely flat surface.
- Be careful when mounting the base because it has considerable weight and can injure you. Preferably install it with the help of another person.

During use:

- Brake the casters for better use of the table and to prevent it from rolling.
- Do not exceed the maximum weight supported by the product, which is 25 kg.
- Be careful not to get your fingers caught in the moving part of the table when adjusting the tilt of the table.

CLEANING AND MAINTENANCE

- Clean regularly with a damp cloth. Do not rinse with water.
- Use mild detergents to clean the table. Do not use abrasive products.
- For disinfection in hospital environments, follow the instructions of a hygiene specialist.

WARRANTY

KMINA guarantees that this device is free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear requiring periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

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