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REFERENCE

KMINA FIBRE CARBON CRUTCH K10008

The **KMINA CARBON FIBRE CRUTCH** is foldable and is made of black carbon fibre tubes. Its soft grip is made of high quality fiberglass reinforced plastic and has reflectors on the front. It also has an elastic band for quick assembly when folding and unfolding. Includes storage bag, ideal for easy portability and the rubber ends are 3/4".

HEIGHT ADJUSTMENT

- Unfold the folding crutch by simultaneously separating the parts.
- Press the top clip and slide the carbon fibre tube until the crutch is secured.
- To establish the correct height, use the lower tube choosing the most convenient one. Stand upright on a firm, flat surface, letting your arm hang naturally at your side. The arms should be slightly flexed with the hand on the handle (no more than 15°) at the level of the trochanter.
- Make sure that the height adjustment clip, is correctly positioned and fits into one of the holes in the tube according to the chosen height.
- Make sure both clips are in place to start using the crutch. To fold, proceed in the reverse order of unfolding, removing each of the tubes after pressing the clips and inserting the crutch into the transport bag.

WARNINGS

Do not use this product without reading and understanding the instructions. If necessary, contact a healthcare professional, dealer or technical personnel before using this product to avoid possible injury.

Before using, we make sure that:

- All parts are tight, nothing is loose or uneven, and there are no rattles or cracks in the metal.
- The user does not suffer from impediments, disabilities or is under the influence of any medication, etc. that the doctor, therapist and / or salesperson considers interferes with the safe use of this product.

During use, we ensure that:

- When using the crutch, we make sure that it is adjusted to the same height and that all parts are correctly adjusted.
- We do not hold the crutches in case of sudden impact or jolts. If that is the case, we release our hands and put them on the ground.
- When folding, hold the entire rod so that it does not jump and cause damage or scares, keeping your fingers away from the folds so that they are not trapped.
- We do not exceed the maximum weight capacity for these crutches, which is 300 lb.
- We have correctly adjusted the height of the crutch. It has 10 adjustments through adjustable pressure. The minimum height is 38" (29" to the handle) and the maximum height is 47" (39" to the handle).
- We avoid any safety risk including slippery, uneven or smooth surfaces and objects on the road.
- Using these crutches does not eliminate the risk of falls.
- Inspect the ferrule or plug often to verify that it is not damaged or worn.
- Although these crutches have been designed to be more comfortable than others, we will always use them with extreme care.
- Crutches must be kept in clean condition.
- Replace any broken, damaged, or worn parts. Use only manufacturer's parts.
- Failure to follow the above instructions may result in serious injury.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

The product can be cleaned and disinfected with commercially available products, as well as with a damp cloth. The padding can be hand washed or machine washed in cold water. Always keep the product clean and wash it regularly.

TECHNICAL SPECIFICATIONS

ADJUSTABLE HEIGHT	FOREARM WIDTH	FOLDING HEIGHT	
38" - 47"	3.75"	21.6"	
N° ADJUSTMENTS	TIPS	USER MAX. WEIGHT	CRUTCH WEIGHT
10	5/8" - 3/4"	300 lb	1.28 lb

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