USER MANUAL

MANUAL DE USUARIO MANUALE UTENTE BENUTZERHANDBUCH MANUEL D'UTILISATION

KMINA PRO CRUTCH

MULETA KMINA PRO
STAMPELLA KMINA PRO
KMINA PRO KRÜCKEN
BÈQUILLE KMINA PRO

EN HOW TO ADJUST

REFERENCE	
KMINA PRO CRUTCH	K10001 / K10002

1 ADJUST THE HEIGHT

1° Introduce the interior tube.
2° Adjust the crutch height with the clip.

Straight shoulders.

Shoulders are too far down.

Shoulders are too far up.

2 ADJUST THE HANDLE POSITION

1° Place the elbow at the suggested zone.
2° Adjust it to the right position (see examples).
3° Secure the position with the screw.

The handle position is correct.

The handle position is too close.

The handle position is too far away.

*** MAKE SURE THAT THE FASTENING STRAP IS PROPERLY PLACED**
We recommend to place the opening part of the fastening strap on the outside.

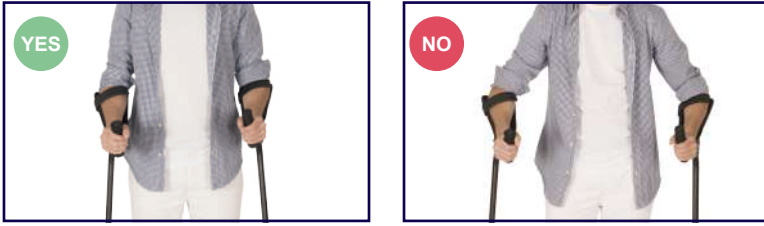
Right crutch (K10001).

Left crutch (K10002).

EN HOW TO USE


1 PLACE YOUR ELBOW CLOSE TO YOUR BODY

2 WALK WITH THE TIP 12" CM FROM YOUR BODY

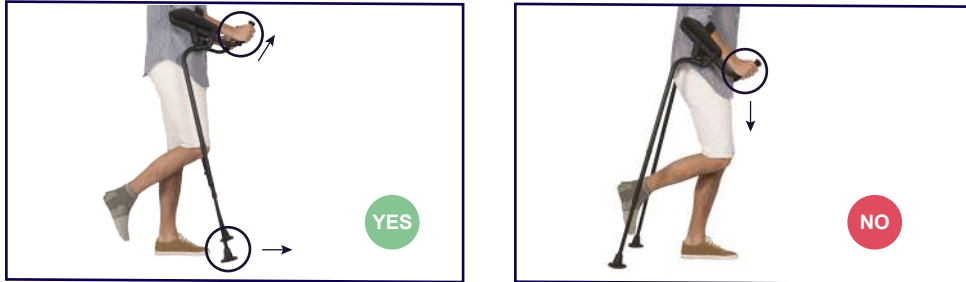3 WALK WITH YOUR ELBOWS CLOSE TO YOUR SIDES, RAISE YOUR HANDS UP AND MOVE THE TIP FORWARD

4 TAKE SHORT STEPS

5 TAKE OFF YOUR CRUTCH CORRECTLY

The flexible fastening strap allows you to take off your crutch from behind or from the side. For a better grip and greater safety you may experience some pressure.

EN WARNINGS

These crutches are recommended for active crutch users and with common pathologies such as: hip operation, knee operation, back problems, stroke, amputees, spinal cord injury, hemiplegia or multiple trauma.

Before using, make sure that:

- Your elbow is placed in the suggested zone.
- The handle padding is well adjusted and does not slip out of place.
- Nothing is unadjusted or uneven and there are no cracks in the metal.
- All the springs and screws are tight and well regulated.
- The forearm support is at a 30 degree angle with respect to the horizontal axis.
- The grip of the handle is well regulated, placing the elbow in the indicated area.
- The maximum forearm length for the crutches should not exceed 9.5". The minimum forearm length requirement is 7.8" measured from the elbow to the wrist.

When using, make sure that:

- You do not have any disability or are under the influence of any medicine, etc., that a medical professional or therapist would deem to be in conflict with the safe use of the product.
- You have consulted a doctor or therapist and have been informed of the proper and safe way to walk with these crutches.
- You have correctly adjusted the height of the crutch, which has 17 adjustments every 0.6". The minimum height of the crutch is 38" and the maximum height is 48" approx.
- If you use two crutches, ensure they are both set to the same height and all springs are working correctly.
- Body weight should be evenly distributed throughout your forearm and not on the hand.
- In the event of a sudden accident or shock, do not hold the crutches.
- Release your hands and try to place them on the ground.
- The maximum weight capacity for this crutch should not exceed 220 lb.
- The maximum height capacity for this crutch should not exceed 6'1".
- The minimum height requirement is 4'11".
- Try to avoid potential risks and hazards that would put your safety in jeopardy eg, slippery surfaces, irregular surfaces and objects on the ground.
- Using these crutches does not eliminate the potential risk of falling.
- Whilst these crutches have been designed to be more comfortable for the patient, careful use should be carried out at all times.
- These crutches should be kept clean at all times.
- Any serious incident related to the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

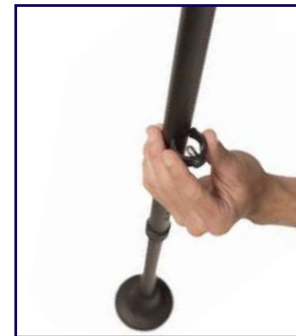
The **product can be cleaned** and disinfected with commercially available products, as well as with a damp cloth. The padding can be hand washed or machine washed in cold water. Always keep the product clean and wash it regularly.

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