PEPE - KIDS CRUTCHES offer the user support when walking. They have been designed to be used by children who have a physical deficiency or injury, or who are undergoing convalescence or rehabilitation periods.

HEIGHT ADJUSTMENT

- Stand upright on a firm, flat surface, allowing your arm to hang naturally at your side.
- Handgrip height adjustment. Slide the aluminium tube B (Fig. 1) until the handgrip is at the height of the trochanter.
- Forearm adjustment. Slide tube A (Fig. 1) to adjust the forearm. The arms should be slightly bent with the hand on the handle (no more than 15°).
- Fit the height adjustment clips to start using the crutch.

FEATURES

- It has 11 settings, 8 crutch height settings and 3 forearm height settings. The minimum height is 28.2" (up to the handle 22.4") and the maximum height is 36.2" (up to the handle 29.3").
- The tip used in this crutch is for tubes with a diameter of 5/8" - 3/4".
- The approximate weight of each crutch is 16 oz.
- Aluminium tubes. Polypropylene (PP) elbow and handle. Rubber tips. Steel and polypropylene pin.

WARNINGS

Do not use this product without reading and understanding the instructions. If necessary, contact a health care professional, distributor or technical personnel before using this product to avoid possible injury.

Before use, you must ensure that:

- All parts are tight, nothing is loose and there are no rattles or cracks in the metal.
- The user does not suffer from impairments, disabilities or is under the effects of any medication, etc. that the doctor, therapist and/or seller considers to interfere with the safe use of this product.

During use, make sure that:

- When using two crutches, they are adjusted to the same height and that all parts are adjusted correctly.
- Do not hold the crutches in case of sudden impact or jolts. If this happens, release your hands and support them on the floor.
- Do not exceed the maximum weight capacity for these crutches, which is 220 lb.
- Avoid any safety hazards including slippery, uneven or soft surfaces and objects in the path.
- Using these crutches does not eliminate the risk of falls.
- The tip or cleat is often inspected for damage or wear. In case of damage, any broken, damaged or worn parts are replaced with parts from the manufacturer.
- Although these crutches are designed to be more comfortable than other crutches, they should always be used with extreme care.
- The crutches should be kept in a clean condition. It is recommended that the handle and forearm be washed regularly with neutral soap.
- Failure to follow the above instructions may result in serious injury.
- Any serious incident involving the product should be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

This product **can be cleaned and disinfected** with commercially available products as well as with a damp cloth. Keep the product always clean and wash it regularly.

KMINA guarantees that this device is free from defects. The warranty shall remain valid for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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