EN	REFERENCE	
	PEPE - CANE WITH SEAT (BLACK)	P10017
	PEPE - CANE WITH SEAT (BRONZE)	P10018

The **PEPE - CANE WITH SEAT** is designed for people who need extra support to walk. It also has a folding seat to sit down and rest. It is designed for the elderly, people with mobility problems, with knee pain or in rehabilitation. Materials: aluminum tube, plastic handle and seat.

INSTRUCTIONS FOR USE

- As a walking support, the cane with seat can be used like any other cane.
- As a seat, the cane is used in the following way:
 1. *To unfold the chair:* Pull the lower part of the seat until the legs are completely open. The user should sit so that the handle of the cane is at the front. **IMPORTANT:** do not use the handle as a backrest.
 2. *To fold the chair:* pull the upper part of the seat and hook the clip at the bottom of the seat with the cane tube, leaving the seat completely upright.

WARNINGS

- Regularly check the cane with seat for damage or wear. Periodically check that all screws are tightened securely.
- Be careful when using the cane on slippery surfaces to avoid possible falls. Use it on flat surfaces.
- Do not apply excessive weight to the cane and never exceed the maximum indicated weight, which is 220 lb.
- Do not stand on the seat.
- Before using the cane with seat, check that all parts are securely connected and fastened. In case of signs of breakage or damage, do not use the cane.
- Do not store the cane with seat at temperatures below freezing point.
- Use the cane with seat only for the purpose for which it was designed. The company is in no way responsible for any misuse of the cane, or for injury or damage resulting from its use.
- Do not bring the cane with seat close to abrasive materials.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

CLEANING AND MAINTENANCE

- Keep your cane with seat clean using a damp cloth. Do not use abrasive cleaning materials.
- Do not keep in the sun for long periods of time.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement. This warranty does not cover:

- Replacement parts supplied by anyone other than KMINA or the authorized dealer.
- Any failure of the unit during the warranty period that is not caused by a defect in material or workmanship or if the failure is caused by misuse.
- KMINA is not liable for accidental damage or damage to property.

SEAT HEIGHT	SEAT DIMENSIONS	WEIGHT	Ø TIP TUBE	USER WEIGHT
20.5"	8.5" x 7.7"	2.2 lb ± 1.7 oz	1"	220 lb

REFERENCE	HEIGHT	FOLDED DIMENSIONS
P10017	34.3"	34.3"x8.3"x2"
P10018	32.3"	32.3"x8.3"x2"

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