

USER MANUAL

PEPE
MORBIDITY

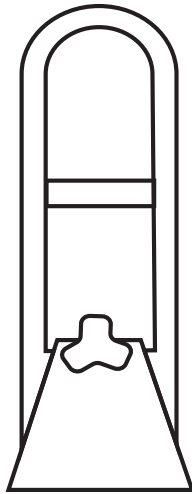P30008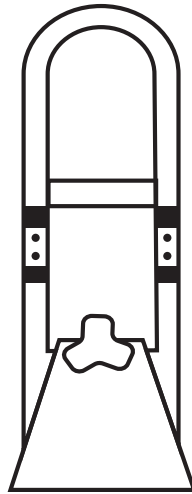

P30035

EN

REF	
PEPE - BATHTUB GRAB BAR	P30008
PEPE - BATHTUB GRAB BAR ADJUSTABLE	P30035

The **PEPE - BATHTUB GRAB BAR** is ideal for people with difficulty in getting up, risk of falling or reduced mobility, providing help and safety for access and exit from the bath, as well as a support point during use. Do not use this product without reading and understanding the instructions. If necessary, contact a healthcare professional or technical personnel.

ASSEMBLY

Examine the Access Bar Kit containing the Bracket (1), Grab Bar (2), and Screw Key (3). *If your grab bar is height adjustable **P30035**, Grab Bar is divided in base and bar and it is necessary the adjustment clips for assembly (4).

1. Insert the long part of the bracket (1) into the access bar (2). If your bathtub grab bar is height adjustable, the Access Bar is divided in base and bar, insert
2. Use the screw wrench (3) to tighten the three pieces so that they are held in one.
3. Determine the installation location on the edge of the bathtub.
4. Place the bracket on the edge of the bathtub and adjust to the width of the bracket with the screw wrench, so that it is supported and secure.
5. Try hard and pull the bar to test its supporting power.
6. If your bathtub grab bar is height adjustable use the adjustment clips (4) in the holes as required.

CARE AND CLEANING

- Cleaning with a microfiber cloth with hot soapy water.
- For the most stubborn stains you can use a sponge, but it is essential not to use scourers, bleaches or abrasive products.
- We recommend cleaning it before the first use, after each use and whenever there is contact with body fluids.

GENERAL WARNINGS

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. Do not use the product if it is damaged!
- The grab bar supports 220 lb.
- We recommend using only one user at a time.
- Check the setting each time you use the access bar.
- Use the device for its intended purpose. The company is not responsible in any way for any misuse that is made of the bathtub rail grab, or for injuries or damages caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

MATERIAL	USER MAX. WEIGHT	Ø BAR	WEIGHT
White lacquered steel	220 lb	1"	6.6 lb

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