CODE	
PEPE - BATH TRANSFER BOARD 27.5"	P30010

The PEPE - BATH TRANSFER BOARD is ideal for people with difficulty getting up, risk of falling or reduced mobility, providing help and safety access to the tub. Do not use this product without reading and understanding the instructions. If necessary, contact a healthcare professional or technical personnel first to avoid possible injury.

ASSEMBLY / USAGE

Board assembly (1):

1. Attach the 4 plastic legs (A) with the corresponding washers (B).
2. Drive the plastic screws (C) through the side slots so that they fit with the plastic feet (A) and washers (B). Do this one by one.
3. Once fitted, turn each of the 4 legs clockwise without forcing the threading until they are tight to the table.

Handle assembly (2):

1. Insert the handle at the top of the table and into the holes. It is advisable to wet the handle with water or lubricant so that it enters under pressure and after that, tighten the small plastic screws (x2).

Bathtub assembly:

1. Slightly unscrew the plastic legs on the inside of the board to allow them to be moved forward or backward. In this way they will be in direct contact with the inside of the bathtub wall.
2. Adjust the brackets to fix them to the desired size, by screwing.
3. The board must not extend beyond the wall of the bathtub by more than 1".

Use:

1. Sit on the widest part of the board located on the outside with your legs out. The handle should be positioned on the inside of the tub with room to place your hand.
2. Once seated, grasp the handle and insert your legs to enter the bathtub.
3. Sit in the center part safely.
4. Do the same procedure to get out of the bathtub.

CARE AND CLEANING


- Cleaning with a microfiber cloth with hot soapy water. Maximum 140 °F.
- Do not use scourers, bleaches or abrasive products.
- We recommend cleaning it before the first use, after each use and whenever there is contact with body fluids.

GENERAL WARNINGS

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. Do not use the product if it is damaged!
- The bath board supports 242 lb.
- We recommend using only one user at a time.
- Check the plastic supports before use.
- Use the device for its intended purpose. The company is not responsible in any way for any misuse that is made of the bath transfer board, or for injuries or damages caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

TECHNICAL SPECIFICATIONS

MATERIAL	USER MAX. WEIGHT	FIT MEASURES	COLOR
Polypropylene	242 lb	15.8"-24.5"	WHITE
HANDLE HEIGHT	TOTAL MEASURES	BRACKETS HEIGHT	WEIGHT
2.5"	27.5"x13.5"x1.5"	2.5"	5.5 lb

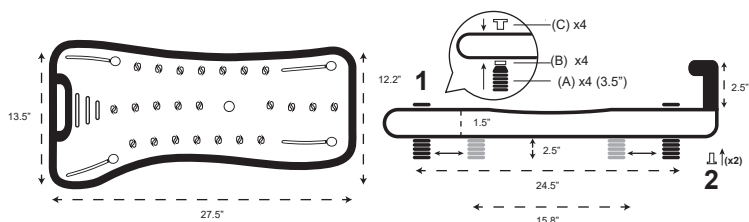REFERENCIA	
PEPE - TABLA BAÑERA 70 cm	P30010

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