CODE	
PEPE - TOILET RISER WITH HANDLES	P30012

The PEPE - TOILET RISER WITH HANDLES is perfect for people with difficulty getting up from the toilet, providing security and firmness, for the elderly, disabled or people with reduced mobility. Don't use this product without reading and understanding the instructions. If necessary, contact a healthcare professional, dealer or technical personnel first to avoid possible injury. Furthermore, it fits most classic oval toilets.

#### ASSEMBLY

The box contains: A. Raised toilet seat with cover, B. Screw-in brackets (2) and C. Armrests.

#### To install the armrests:

- Place each armrest into the numbered holes in the bottom of the toilet seat. The metal buttons should be facing out.
- Slide the armrest tube into position until the button clicks into the hole with its corresponding number and a "click" sounds.
- Repeat the process for the rest of the holes.
- To remove the armrests, press the metal button and pull the handle upwards..

#### To install the toilet riser:

- Place the toilet riser face down on a flat surface and rotate the brackets into the appropriate side locations.
- Tighten and turn the screw handle clockwise to lock the brackets.
- Don't fully tighten the brackets, otherwise it will not be possible to position the seat.
- Lift the cover and place the toilet riser over the toilet. Make sure the inner edge is placed inside the toilet and the front edge it is placed outside. The toilet riser should fit into the toilet in a perfect flat position.
- Rotate the head of the brackets clockwise until the toilet riser is secure and immovable .

#### CARE AND CLEANING

- All parts of the toilet riser can be cleaned with soap and hot water or disinfected with NON-ABRASIVE products.

#### GENERAL WARNINGS

- For a correct use of the device always consult your doctor or therapist.
- Make sure the product is delivered complete and keep it packed away from heat sources. Don't use the product if it is damaged.
- The raised toilet seat must not be used to support more than 220 lb.
- The armrests help you get on and off the seat. Do not use them to support the full weight of the body or with the entire weight on one armrest holding it with both hands or the device may fail.
- We recommend using only one user.
- Check the bolts monthly and, if necessary, retighten.
- The device must be disinfected before reuse.
- DO NOT allow children to play or operate the device.
- DO NOT use the cover as a seat.
- Use the toilet lift only for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet lift, or for injuries or damage caused as a result of it.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.
- Do not remove the sticker provided with the product. If necessary, make a note of the product lot in order to inform the manufacturer in the event of an accident.

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