
PEPE PEPE MOBILITY

MANUAL / INSTRUCCIONES / GEBRAUCHSANWEISUNG / INSTRUCTIONS / ISTRUZIONI


P30016

P30029

1) COMPONENTS / COMPONENTES / KOMPONENTEN / INSTRUCTIONS / ISTRUZIONI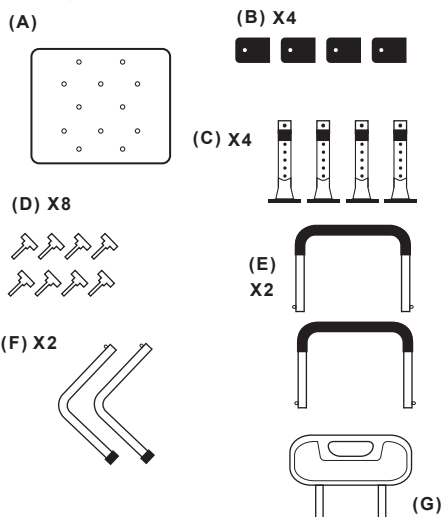

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO

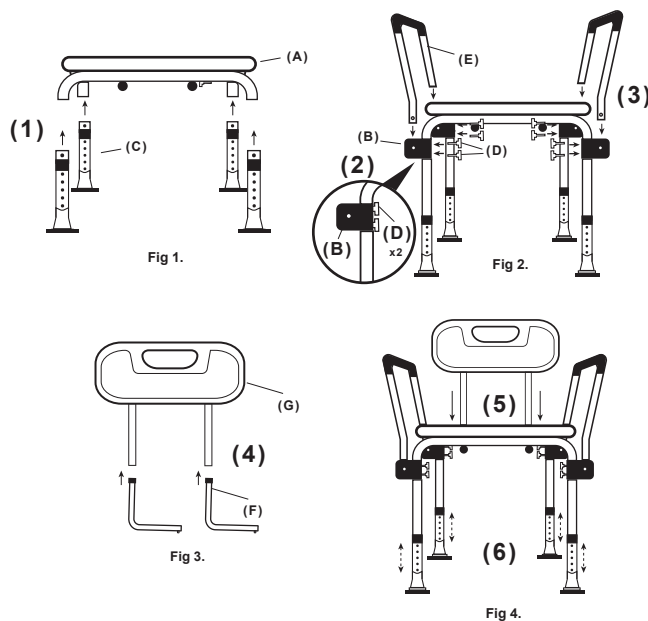EN

CODE	
PEPE - CHAIR WITH HANDLES (BLACK)	P30016
PEPE - CHAIR WITH HANDLES (WHITE)	P30029

CHAIR WITH HANDLES

Manual provided for the user of the product. BEFORE using this product, read this manual and keep it.

INDICATIONS OF USE

The PEPE-CHAIR WITH HANDLES should be used as an aid to sit in the bath or shower and to facilitate the seat for people with movement difficulties in personal cleaning. Its capacity is 220 lb and its weight is 10.5 lb.

FEATURES

- Assembly without the use of tools.
- Removable armrest.
- Quick and easy adjustment from 17" to 21" height in 5 positions according to the regulations used on the legs to suit the user.
- Ergonomic surface with drainage holes.
- Safe and stable thanks to its non-slip tips.
- Corrosion-free thanks to its aluminum chassis and polyethylene back and seat.

ASSEMBLY

1. Turn the seat over (A) and you will see the assembled structure on which the legs, backrest and handles will be installed. Insert the legs (C) into each of the holes in the seat frame until they click, as shown in Figure 1. Make sure they are all at the same height and adjustment.
2. Adjust the brackets (B) as in Figure 2, using the levers (D) to secure them to the frame. Be sure to tighten the two levers on the tube.
3. Insert the armrest (E) into the protruding side holes of the brackets (B) by pressing down as in Figure 2.
4. Take the backrest (G) and insert the L-shaped structures (F) until you hear a click.
5. Once assembled, insert the backrest (G) into the tubes at the bottom of the frame (A) by pressing down until you hear a click.
6. Check that the height is the same on all legs or adjust as required.

WARNINGS!

- Check all parts for all components or if there was any damage during shipping.
- Make sure all four legs are adjusted to the same height.
- Make sure the fasteners click into the holes in the legs.
- Don't use for side transfers or part load.
- Don't sit on it, unless all four legs are fully in contact with the ground. Use the product only on even ground.
- Don't stand up in the chair.
- The user must be able to maintain balance in case of use without an assistant.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with products available on the market. Always keep the product clean and wash it regularly.

1. Remove the legs.
2. Use water and a mild non-abrasive detergent to clean the separated parts.
3. Disinfect by cleaning all accessible surfaces with disinfectant.

WARRANTY INFORMATION

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

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