PEPE - PADDED TOILET GRAB RAIL

P30027

PEPE - PADDED TOILET GRAB RAIL (x2)

P30028

The **PEPE - PADDED TOILET GRAB RAIL** is ideal for people with difficulty in getting up, risk of falling or reduced mobility, providing assistance and safety. Do not use this product without reading and understanding the instructions. If necessary, contact a health professional or technical personnel first to avoid possible injury.

ASSEMBLY

Supplied in bag with screws (4) and plugs (4), for installation in brick wall. Examine the grab bar kit. Take everything out of the box and check that the grab bar is in good condition, as well as all the parts. The dimensions of the screws are M6x70 mm and wall anchors M10x80 mm.

- Determine the location of the installation. In general, the bar should be at the height of the user's wrist in a standing position.
- Mark the installation location on the wall using a pencil on the holes in the base.
- Use a 25/64" (10 mm) drill bit to drill the holes and insert the plugs where the screws will be inserted, using a hammer if necessary.
- Fix the 4 screws through the holes in the base of the support bar, positioning the bar on the wall and screw the 4 anchor points. It is recommended to use an electric screwdriver.
- Test with force and pull the bar to test its holding power.

GENERAL WARNINGS

- For correct use of the device always consult your doctor or therapist.
- Make sure that the product is delivered complete and keep it packed away from heat sources. Do not use the product if it is damaged!
- The support bar supports 220 lb.
- Do not install in hollow walls, wooden walls or walls with a width less than the length of dowels and screws.
- Use by only one user at a time is recommended.
- Check screws monthly and, if necessary, retighten.
- Use the screws that come with the clamping bar.
- Use the device for the purpose for which it was designed. The company is not responsible in any way for any misuse of the toilet bar, nor for injury or damage incurred as a result.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

FEATURES

- Bar weight: 1.6 lb.
- Bar material: aluminum.
- Padding material: foam.

CARE AND CLEANING

- Cleaning with microfiber cloth with warm water and soap. Maximum 60°C.
- For stubborn stains, you can use a sponge, but it is essential not to use scouring pads, bleach or abrasive products.
- We recommend cleaning before first use, after each use and whenever there is contact with body fluids.

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