

1) COMPONENTS / COMPONENTES / KOMPONENTEN /
COMPOSANTS / COMPONENTI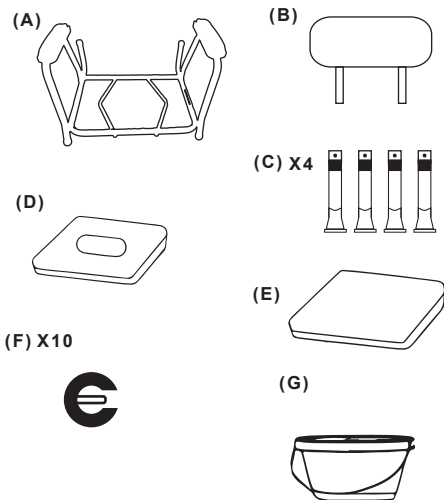

2) ASSEMBLY / MONTAJE / VERSAMMLUNG / MONTAGE / MONTAGGIO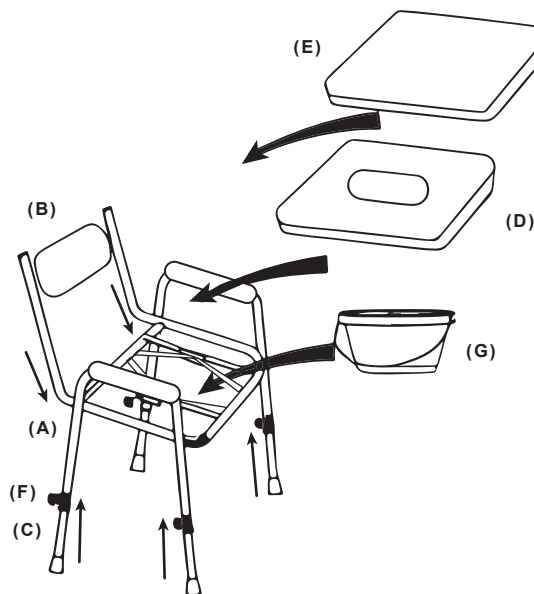

EN

CODE

PEPE - PADDED BEDSIDE COMMODE

P30034

The **PEPE Padded Bedside Commode** is a portable toilet designed to facilitate the need to go to the toilet for people with physical or mental disabilities, whether chronic or temporary. It can also be used by elderly people or people with any mobility problem. It includes a large padded seat, a bucket for bowel movements and is height adjustable to adjust the chair to the user's height.

FEATURES

- Tool-free assembly.
- The height can be adjusted quickly and easily, with 7 adjustments on the legs. Seat: 17" to 23". Cover: 18.5" - 24.5".
- Padded surface.
- Safe and stable thanks to its non-slip tips.
- Frame material: aluminum. Seat material: PVC.
- Maximum user weight: 363 lb.
- Chair weight: 18 lb.

ASSEMBLY

1. Insert the legs (C) into the ends of the main frame (A) by fastening them with the fastening clips (F). Then adjust the height of the legs to one of their 7 heights with another fixing clip (F).
2. Insert the backrest (B) into the holes at the back of the main frame (A) with the fastening clips (F).
3. Place the bucket (G) in the central part of the main structure (A).
4. Attach the seat (D) to the back bar of the main frame (A) by firmly snapping the attachments to the bar.
5. Place the cover (E) on top of the seat by exerting pressure.

WARNINGS!

- Check all parts to verify if all components are present or if there was any damage during shipment.
- Make sure all four legs are adjusted to the same height.
- Do not sit on it unless all four legs are in full contact with the floor. Use the product only on even flooring.
- DO NOT stand on the chair.
- The user must be able to maintain balance when using without an assistant.
- Always sit in the center of the chair.
- The manufacturer is not responsible for any damage caused by using the product in combination with unofficial parts (end caps and/or cover).
- Do not remove the sticker provided with the product. If necessary, make a note of the product lot in order to inform the manufacturer in the event of an accident.
- Any serious incident involving the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

CLEANING AND DISINFECTION

The product can be cleaned and disinfected with commercially available products. Always keep the product clean and wash it regularly.

1. Remove the feet.
2. Use water and a mild, non-abrasive detergent to clean the separate parts.
3. Disinfect by wiping all accessible surfaces with disinfectant.
4. Do not use bleach or any other corrosive product.

WARRANTY INFORMATION

The company warrants this device to be free from defects. The warranty will remain in effect for 3 years from the date of original consumer purchase. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

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