REFERENCE

| | |
|--------------------------------|--------|
| PEPE - ADJUSTABLE BED BACKREST | P40007 |
|--------------------------------|--------|

The **PEPE - ADJUSTABLE BED BACKREST** is recommended for incorporating bedridden patients or people with reduced mobility so that they can perform tasks comfortably. Measurements: 26.3"x23.6". Pad: 11"x6". Material: steel and polyester. Weight: 5 lb.

INSTRUCTIONS FOR USE

1. The product is assembled, ready for use. Unfold the backrest by pulling on the rear tubes.
2. Always put the cover in front. The user will be placed on top.
3. Depending on the regulation you want, it has 5 adjustments with degrees of inclination from 25° to 80° and a height of 13.5" to 23.6".
4. Once stable, position the user in the desired position. Use the pad if you like or remove it easily with the velcro strips.

WARNINGS

- The maximum user weight is 240 lb.
- Inspect periodically for wear and damage. The product should not be stored outdoors.
- Before each use, make sure the product is properly assembled and in a stable position.
- The product can only be used on unobstructed surfaces.
- Risk of injury to fingers may increase during assembly or assembly process.
- Do not modify the product and its components in any way, or use the product for a purpose other than its intended use.
- Do not use the product if it is damaged or altered in any way.

MAINTENANCE

- Can be cleaned with soap and water with a cloth.

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