**ES**

REFERENCIA

PEPE - LAVACABEZAS PORTÁTIL P40009

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REFERENCE

PEPE - HAIR WASHING BASIN P40009

The **PEPE - HAIR WASHING BASIN** is recommended for head and hair washes in bedridden patients or people with reduced mobility. Measurements: 46x35x12 cm. Material: ABS plastic.

#### INSTRUCTIONS FOR USE

1. Insert the tube into the right hole ready for emptied. Make sure it fits snugly and no water comes out.
2. Place the sink in the sink and pour the water with the appropriate temperature, using the stopper to block the water outlet. You can start the wash in the sink or on another surface.
3. Put the wearer's head on the curved edge carefully so that water does not enter the neck and wet the clothing. Comfortable neck support is important as it is designed to conform to the curve of the human neck.
4. After cleaning the hair, drain the sewage with the pipe or empty the sewage directly.

#### WARNINGS

- Check periodically for wear and damage.
- Before use, check the water temperature with the user.
- Any serious incident related to the product must be reported to the manufacturer and the competent authority of the Member State in which the user and patient are established.

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