
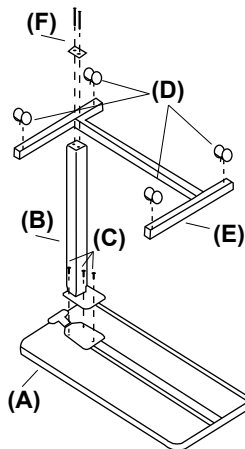
OVERBED TABLE / MESA AUXILIAR CAMA / BETTTISCH / TABLE DE LIT / TAVOLINO DA LETTO

**COMPONENTS / COMPONENTES / KOMPONENTEN
/ COMPOSANTS / COMPONENTI**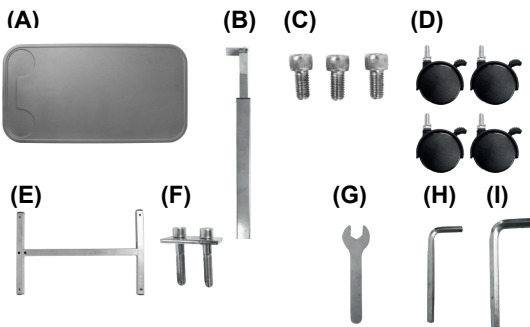

EN	REF.	PEPE - OVERBED TABLE	P40024
----	------	----------------------	--------

The **PEPE Overbed Table** is designed to facilitate the daily life of people who are bedridden or in a wheelchair. It is ideal for reading and writing, using the computer or eating. It has a side lever to adjust the height of the table to the user's needs. **Material:** PVC tray and steel structure.

FEATURES

- 4 castors with brake system.
- Tray dimensions: 28 x 15".
- Adjustable height: 29" - 42".
- Maximum supported weight of the table: 50 lb.

ASSEMBLY

1. Place the tray (A) face down on a flat surface and fit the side frame (B) by screwing the three small screws (C) into the holes of the tray. Use the narrow Allen wrench (H) for a good fit.
2. Place the wheel base (E) on top and fit it to the central structure. Tighten the two large screws (F) using the thick Allen key (I) until it is securely fastened.
3. Finally, install the 4 braked casters (D) into each of the holes in the wheel base frame (E) using the wrench (G) to fully tighten.

HEIGHT ADJUSTMENT

To lower the height of the table: use the lever on the side of the tray and, with the help of the other hand, push the tray downwards.
To raise the height of the table: it is not necessary to press the lever. Just hold the tray by the ends with both hands and push up the tray.

CARE AND CLEANING

Clean regularly by hand with a damp cloth. Do not use abrasive products. Use mild detergents to clean the table.
For disinfection in hospital environments, follow the instructions of a hygiene specialist.

WARNINGS

- Make sure that all screws are fully tightened.
- While the table is in use, brake all four casters for better use and to prevent the table from rolling.
- Do not exceed the maximum supported weight, which is: 50 lb.
- Do not unload all your weight on the table tray.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and patient are established.

WARRANTY

KMINA warrants this device to be free from defects. The warranty will remain in force for 3 years from the date of original purchase by the consumer. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

ES	REF.	PEPE - MESA AUXILIAR CAMA	P40024
----	------	---------------------------	--------

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