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
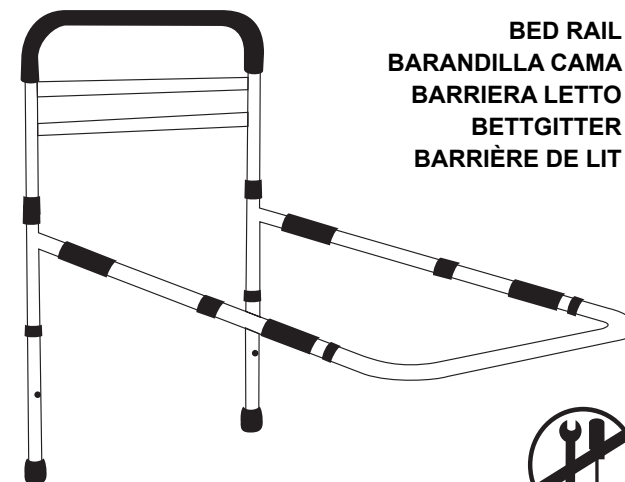
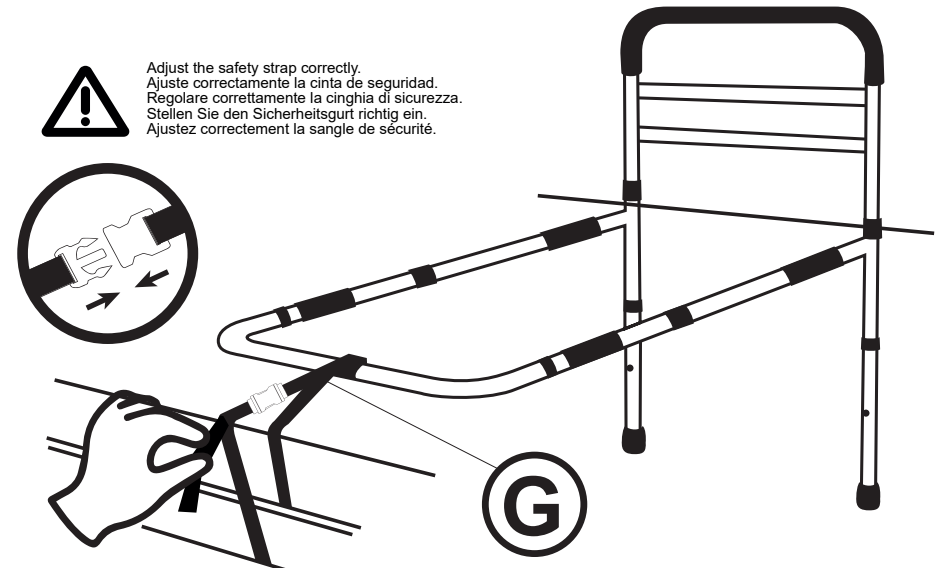
FIG. 5 | DIMENSIONS / DIMENSIONES / MISURE / ABMESSUNGEN / MESURES

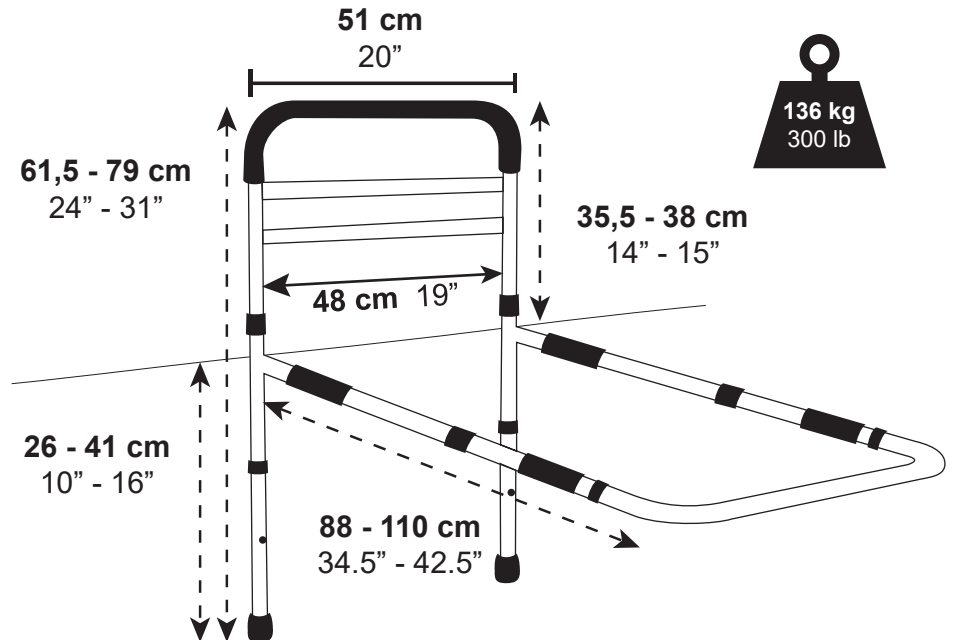REF.	
PEPE - BED RAIL EUROPE (BLACK)	P40025
PEPE - BED RAIL EUROPE (WHITE)	P40026

The **PEPE Bed Rail** is designed to provide users with a secure grip and additional support when getting in and out of bed. It is intended for the elderly, bedridden persons, persons suffering from mental disabilities and persons with a handicap or injury who need extra support. Maximum user weight: 300 lb.

TECHNICAL SPECIFICATIONS

Bed rail weight: 10 lb. Materials: steel structure, foam rubber padding, rubber end caps, nylon and PP safety strap, ABS plastic safety rings and the rest of the parts connecting the tubes are made of PE plastic.

1. COMPONENTES

A. Support rail | B. L-shaped frame | C. Legs | D. Thick tubes | E. Tight tubes | F. End bar | G. Safety strap

2. ASSEMBLY (FIG. 2)

- Insert the support rail (A) into the L-shaped frame (B) with the help of the spring buttons until it clicks and turn the locking ring so that it is securely fastened.
- Fit the legs (C) into the frame and adjust them to the same height in one of their 13 settings.
- Connect the tight tubes (E) into the thick tubes (D) leaving the padding of both tubes at the ends. Adjust the length of the bar to the desired length by turning the locking ring.
- Add to the protruding holes of the L-shaped frame (B) the bar created in step 3 on the side of the thick tubes (D) until you hear it click.
- Finally, fit the end bar (F) into the structure by means of the spring buttons until you hear a click.

3. ADJUSTMENTS (FIG. 3)

- To adjust the depth of the frame under the mattress, turn the thread, adjust as needed and turn again to tighten.
- To adjust the height of the legs, choose the height of both legs from the 13 settings and adjust using the spring buttons.
- To adjust the height of the handle, choose one of the 3 settings and adjust both parts to the same height using the spring buttons.

4. ADJUSTMENT TO THE BED (FIG. 4)

- Slide the bed rail assembled between the mattress and the bed base at the most suitable point of the bed for the user. It may be necessary to lift the mattress to position the rail correctly.
- Push the rail all the way in until the outside of the rail is at the edge of the mattress. Make sure the rail is completely under the mattress.
- FOR EXTRA SUPPORT**, extend the safety strap (G) under the mattress and pull it from the opposite side. Hook it to a support (bed leg or bed base) as shown in FIG. 4 and close the strap buckle. Once closed, tighten it until it is firm and secure.

WARNINGS!

- This bed rail is not designed or intended to prevent a person from falling or rolling out of bed. It is designed to assist the user in getting in and out of bed by providing a secure grip.
- Not suitable for persons suffering from seizures, dementia or other problems resulting in loss of control of body movements.
- Ensure that the spring-loaded spring buttons engage the holes with a click.
- Ensure that the safety strap is firm and taut.
- This product should only be installed by physically and mentally trained adults.
- Do not use two guardrails together on the same side of the bed to avoid being trapped between the two guardrails.
- This guardrail is for adults only. It does NOT serve as a safety system for children.
- Do not put your head, or any other part of your body that could get caught between the rails.
- If the handrail is not used correctly, serious accidents can occur.
- Do not use with mattresses thicker than 11" or less than 7".
- If the gap between the bed rail and the mattress is more than 5 cm, readjust it so that the rail is flush with the mattress.
- This rail must not be used on articulated beds, nor as a backrest.
- Do not use if any parts are missing or broken.
- Any serious incident involving the product must be reported to the manufacturer and to the competent authority of the Member State in which the user and the patient are established.

CLEANING AND DISINFECTION

Clean the structure with soap and water and a dry towel. Check the ferrules periodically for cracks, damage or missing parts. Replace if necessary before use.

WARRANTY INFORMATION

KMINA warrants this product to be free from defects. The warranty will remain in effect for 3 years from the date of original consumer purchase. The warranty does not extend to parts exposed to natural wear and tear that require periodic replacement.

REF.	
PEPE - BARANDILLA CAMA EUROPA (NEGRA)	P40025
PEPE - BARANDILLA CAMA EUROPA (BLANCA)	P40026

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